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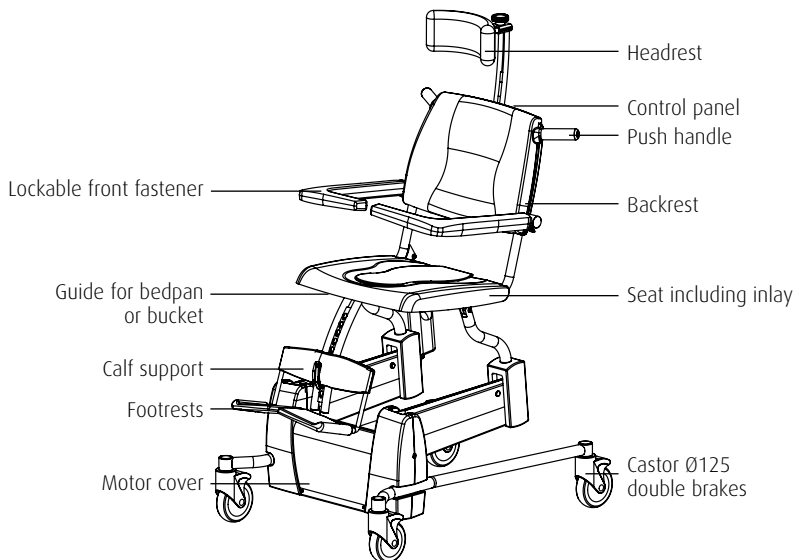
# Index

Introduction	18
General safety	19
Brakes	20
Activation	20
Electronic stop button	20
Sitting down / standing up	21
Foldable/detachable footrests	21
Armrests	22
Raising / lowering the chair	23
Tilting backwards / forwards	24
Adjusting the headrest	25
Using a bedpan / pail	25
Use as a toilet seat	26
Washing	27
Charging the Reflex	28
Maintenance	29
Technical specifications	29
Check list for technical problems	30

## Introduction

The 'Reflex' electrically operated height-adjustable, tiltable shower and toilet chair, is an ergonomically sound aid for the care and cleaning of people with physical handicaps. Comfort is optimum for both carers and users.

Its versatility in terms of adjustment options and the functional design also make it possible to use the Reflex as a toilet seat with the use of a bedpan or toilet bucket, or in combination with a toilet.

The Reflex consists of an upper section, the seat section and the frame in which the drive unit and power supply are housed. The seat section is comfortable and provides effective support for users. The height of the seat section can be electrically adjusted in relation to the frame and can also be tilted electrically. This combination makes the Reflex multifunctional. The Reflex is fitted as standard with arm rests with a front barrier and a removable and height-adjustable footrest. The Reflex is fitted with a stop function. Using this blocks all the functions of the Reflex.

**The chair has a "Stand-by mode" which enables the control functions and the electrical current to be switched off automatically when the chair is not used for an hour. The Reflex also has a failsafe mechanism in case of overloading.**

The Reflex is a product with a CE mark. This mark complies with the Directive on medical aids of 14 June 1993 nr. 93/42 EEG for medical equipment.

The Reflex is tested and manufactured in accordance with EN 12182.

The Reflex complies with the EN 60601-1 norm.

The Reflex has a protection value of IP44, tested in conformity with KEMA report 208446700-QUA/ESG.

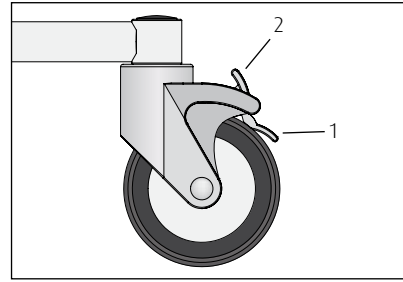
The guideline and standard specifications can be made available on request.

## General safety

- Before working with the Reflex, check that it is no longer connected to the charger.
- Ensure there are no obstacles when raising/lowering or tilting the chair.
- Ensure that the brake on the Reflex is always activated during transfers.
- Close the arm rests with front barrier before moving the user upwards or tilting the user.
- Move and manoeuvre the seat in a low position.
- Do not adjust the seat up or down or tilt the seat while it is moving.
- While adjusting the seat section up or down, do not place the feet on the lower frame. This will prevent the feet from becoming trapped.
- Ensure that there are no obstacles between the moving parts of the Reflex in order to prevent damage to the Reflex and injury to users and carers.
- Preferably, always move backwards over obstacles.
- Adjust the seat to the correct height before using the toilet.
- Always replace the charger or cable if they are damaged.
- If problems arise, contact your dealer.

## Brakes

- The brakes fitted to the Reflex are activated by depressing the protruding lip (1) on the wheel.
- Deactivate the brakes on the Reflex by depressing the upper lip (2) onto which the wheels are attached.

## Activation

Before being able to use the Reflex, it must be activated. To do this, hold down both “up buttons” simultaneously for one second.

## Electronic stop button

- The Reflex is fitted with a stop button that should only be used in an emergency. By pressing the red knob all functions of the Reflex will be rendered redundant.
- By pressing the red stop button the red LED will burn and the chair is in ‘non -operating mode’.
- Unlock the red knob by pressing the two outside buttons for 4 sec to deactivate the emergency stop.

4 sec.

Up button

4 sec.

## Sitting down / standing up

- The carer moves the Reflex towards the user's back, or the user is taken to the Reflex with the aid of a support.
- The brakes on the Reflex must be activated when transfers are carried out.
- The carer sets the correct seat height for the user.
- The carer either removes the footrest or folds them up.
- The carer opens the arm rests and folds them up if necessary.
- The user can now sit down.
- Standing up involves the same sequence of actions in reverse.

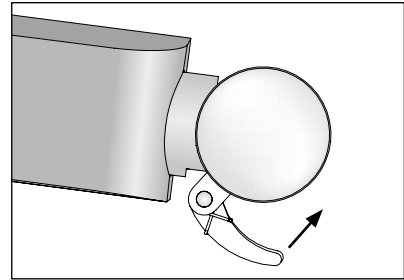
## Foldable/detachable footrests

- The footrest on the Reflex can be removed from the frame simply by pushing the locking lip upwards. The footrest can then be pushed upwards and removed from the frame. Replacing the footrest involves the same sequence of actions in reverse.
- If removal of the footrest is not required, the carer can simply fold up the footplate. When the user is seated in the seat, the carer can place the legs on the footplates after folding them back into position.
- The height of the footrest is adjustable through 23 cm in 10 increments.

## Armrests

- If the user is seated on the Reflex, the armrests with front barrier can be folded down and then positioned appropriately for the user.
- The armrests with front barrier are secured in the horizontal position.
- The armrests can be unlocked by pressing the safety catch at the hinge point in the frame (see illustration), after which the armrest can be turned away by the user and folded upwards.
- Always protect the user when moving the armrests up or down.
- The armrests can also be used as supports when sitting down and standing up. Take account of the fact that the armrests are not locked in all directions of rotation.
- Straight armrests are available as options for the Reflex (option 02,5650).

**Note: When lowering the armrests into place, take care not to get fingers trapped in the hinge.**

## Raising the chair

- When the user is seated on the Reflex, first activate the brakes and then raise the seat.
- The seat section is moved upwards by pressing the knob on the back with the arrow facing upwards. The seat frame of the Reflex will then steadily move upwards.
- You can stop this movement when the desired height is reached by releasing the knob.
- Maintain contact with the user while this upward motion is taking place.

**Up button**

**Controlpanel**

## Lowering the chair

- The seat section is moved down by pressing the knob on the back with the arrow facing down. The seat frame of the Reflex will then steadily lower.
- You can stop this movement when the desired height is reached by releasing the knob.
- If an obstacle blocks the downward movement of the chair, this downward movement is automatically cut out. The chair can only be raised.
- Maintain contact with the user while this downward motion is taking place.

**Down button**

## Tilting backwards / forwards

- The Reflex is fitted with a tilt function to a maximum of 30° backwards. This function can be operated by pushing the knob with the arrow facing backwards. The seat section will then tilt.
- We recommend activating the brakes before tilting the Reflex.
- When the seat section has reached the maximum tilt angle, the movement will stop automatically. The carer can stop the motion at any time by releasing the button.
- To tilt the Reflex forwards, push the knob with the arrow facing forwards.
- The movement will stop automatically when the seat section has returned to the horizontal position.
- We recommend that you use the armrest with front barrier when tilting the seat.
- Maintain contact with the user when tilting the seat.

**Backwards button**

**Forwards button**

**Control panel**

## Adjusting the headrest

- The Reflex is fitted with a height-adjustable headrest that can be operated with a rotary knob that is placed on the back and on the top.
- Unscrew the knob and move the headrest up or down. When the correct height has been reached, tighten the knob.
- Completely removing the knob allows you to remove the headrest. The depth can be adjusted by 7 cm and is adjusted by unscrewing the top.
- The Reflex has a bowl-shaped headrest, which means that the user's head is supported optimally, also at the side.

## Using a bedpan / pail

- When the Reflex is used as a toilet chair, a bedpan or toilet pail can be slid into the special guide under the seat.
- The bedpan or toilet pail is slid in from the back of the seat. In order to prevent injury to the user, we recommend that the carer place the bedpan or toilet pail before the user is seated.

**In order to avoid injury to the user, we recommend that the carer remove the cover from the seat and put the bedpan or pail in position before the user sits down.**

### **Use as a toilet seat**

- The Reflex can also be positioned over the toilet. The carer adjusts the Reflex to the correct height after the user is seated and the armrests are closed.
- The Reflex can now be moved backwards over the toilet.
- If necessary, lower the Reflex to the correct position.

**In order to avoid injury to the user, we recommend that the carer removes the cover from the seat before the user sits down.**

**Note:**

**The Reflex is not compatible with all toilets.**

## Washing

- The fact that the Reflex is height-adjustable improves the ergonomic aspect for the carer while washing the user.
- Adjust the Reflex to a good working height for washing the upper body and hair.
- The carer can then reset the height to wash the lower body and legs.
- Users with poor upper body balance can use the tilt function.
- The height setting depends on the height of the carer.

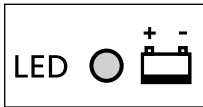
## Charging the Reflex

- The Reflex is an electrically driven high/low shower-toilet seat that operates on battery power. These batteries must be charged on a daily basis. The charging point is located on the back of the seat and is covered by a flap.
- If the batteries are almost empty the LED flashes red: batteries need to be recharged soon. When the batteries have been drained of most of their power, the LED on the control panel will be lit up red continuously, which indicates that they must be charged immediately (Chair can no longer be used).

Charger connection point

- Open the cover and connect the charger to the Reflex (Stand-by mode).
- **When connecting the charger to the chair all functions are non operating.**
- If the signal light on the charger is green, the batteries are charged sufficiently. The charger can remain attached once the batteries have been sufficiently charged.
- Disconnect the charger from the Reflex and close the flap. Only use the original charger supplied with the chair.
- The reflex is ready for use after resetting (see led burning green).

LED not burning:

Chair is in non-operating mode and cannot be used.

LED is burning green:

Chair can be operated (after pressing down the raise buttons for 1 second).

LED flashes red:

Batteries need to be recharged soon (chair is still usable).

LED lights up red continuously: Batteries are empty and need recharging immediately.

**We recommend that you install the charger at a permanent location in a dry room (therefore not the bathroom or shower room).**

**Connect the charger to the chair after use to conserve batteries.**

## Maintenance

- The Reflex must be cleaned after each use with water and normal household cleaning agents. Do not use abrasives or other aggressive solvents (such as chlorine).
- When the Reflex must be disinfected, we recommend using Neoquat combi®. Do not use an alcohol solution.
- Regularly check the wheels for hairs and dust, remove if necessary.
- Remove excess soap remnants.
- Lopital Nederland B.V. recommends concluding an annual maintenance contract for the Reflex.
- Maintenance and repairs must only be carried out by authorised staff.

## Technical specifications

### Construction

Frame from steel tubes and panels with a dual layer of powder coating

Stainless steel hinges

Stainless steel and plastic sealing caps

PVC castor wheels, Ø 125 mm, double blocking option

PUR (polyurethane) body support

Reflex weight 55kg, maximum load 135kg

### Drive unit

Electric motors 24 V

Stainless Steel spindle for up/down movement

24 V actuator for the tilt adjustment

Battery capacity 7Ah, charger 220 V/50/60 Hz

### Dimensions

Adjustable seat height of 490-1010 mm

Backrest height 930-1450 mm

Seat depth 485 mm, seat width 480 mm

Total length of the basic frame 880 mm, total width of the basic frame 660 mm

Foot support adjustment 230 mm

Headrest adjustment depth/height 70/120 mm

Continuously variable tilt 30° to rear

# Check list for technical problems

Problem	Cause	Solution
Reflex will not move up/down and will not tilt	Chair is still connected to the charger	Disconnect the chair from the charger
	Emergency stop activated	Check emergency stop
	Chair is not charged	Charge chair
	Chair is in non-operating mode	Activate chair
Reflex stops suddenly	Emergency stop activated	Check emergency stop
	Technical fault	Contact your dealer
Reflex tilt function does not work	Defect in the actuator	Contact your dealer
Reflex moves with difficulty	Dirt and soap between the wheels	Clean the wheels
Charger operating light is not activated	Charger is not connected	Connect the charger
	Charger is defective	Contact your dealer

**Comment:**  
**If all the functions of the Reflex fail, the seat can be lowered manually with the key supplied.**

For all other faults please contact your dealer.

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Lopital Nederland B.V. Laarakkerweg 9, 5061 JR Oisterwijk, Postbus 56, 5060 AB Oisterwijk