Aquatec® Pico / Pico TS

- EN** Shower stool / Commode chair
User Manual
- DE** Duschhocker / Toilettenstuhl
Gebrauchsanweisung
- FR** Tabouret et chaise de douche / Chaise percée avec accoudoirs et seau
Mode d'emploi
- IT** Sgabello per la doccia / Sedia da comodo
Istruzioni per l'uso
- ES** Taburete de ducha / Silla WC
Manual de instrucciones
- NL** Douchekruk / Toiletstoel
Gebruiksaanwijzing
- DA** Badetaburet / Toiletstol
Brugsanvisning
- FI** Suihkujakkara / WC-tuoli
Käyttöohje
- NO** Dusjkrakk / Toalettstol
Brukerveiledning
- SV** Duschpallen / Toalettstol
Bruksanvisning
- PT** Banco de duche / Cadeira de casa de banho
Manual de instruções
- ET** Duši-/tualetitool
Kasutusjuhend
- LT** Dušo / tualetu kėdutė
Veikimo nurodymai
- LV** Dušas krēsls / tualetes krēsls
Lietošanas instrukcija

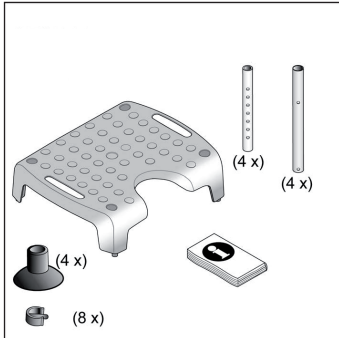
Fig. 1

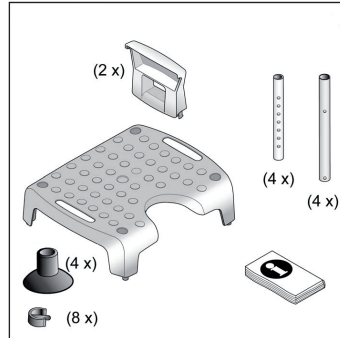
Fig. 2

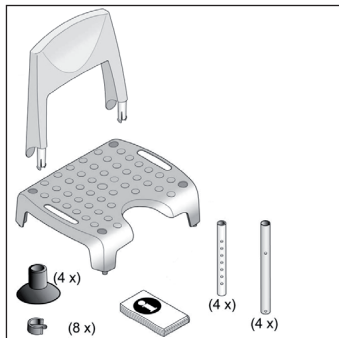
Fig. 3

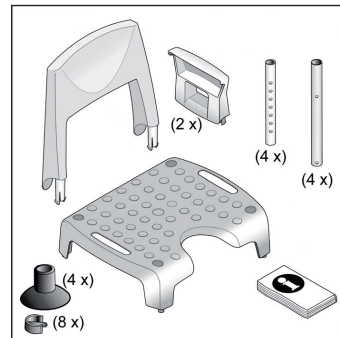
Fig. 4

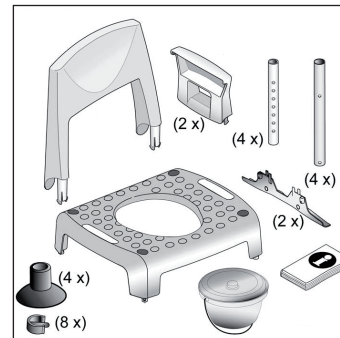
Fig. 5

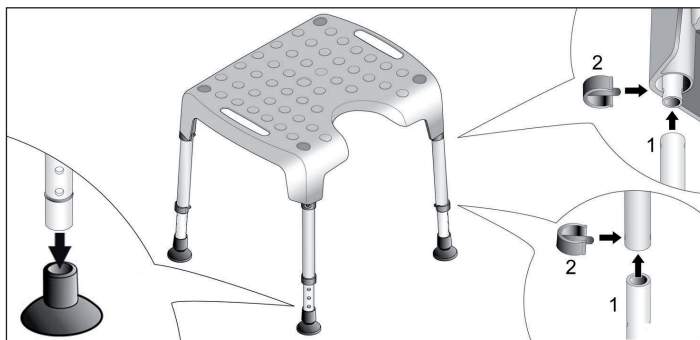
Fig. 6

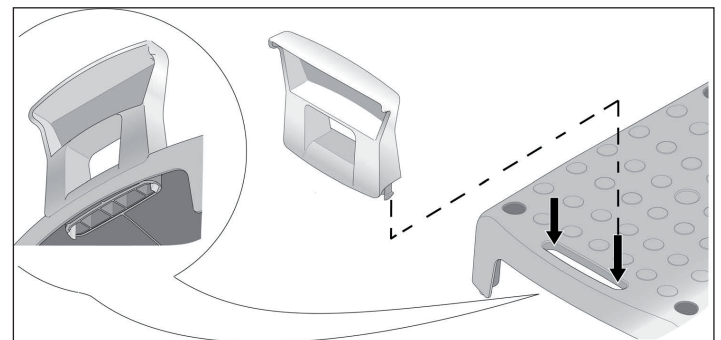
Fig. 7

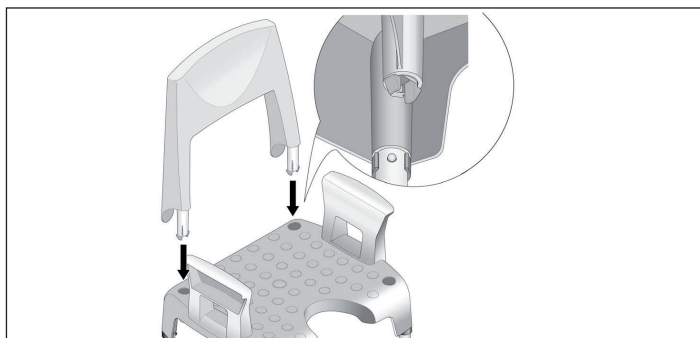
Fig. 8

AQUATEC PICO komplett
with arm- and backrest
2012-05 160 kg 40286984033804
Made in Germany (01) 04028698083904 (10) 12EDA0001

AQUATEC PICO TS
2012-05 160 kg 40286984116999
Made in Germany (01) 04028698116999 (10) 12EDA0001

This manual must be given to the user of the product. Before using the product, read this manual and safe it for future reference!

Yes, you can.

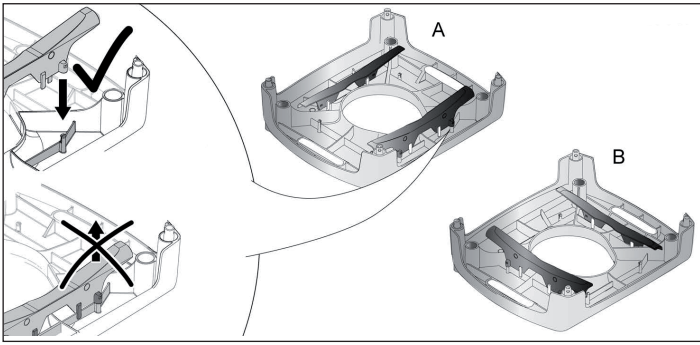
Fig. 9

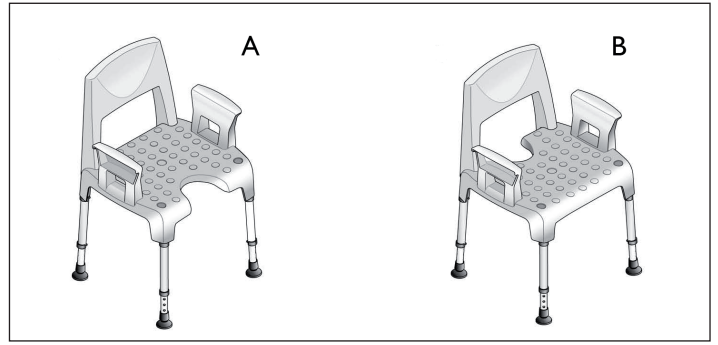
Fig. 10

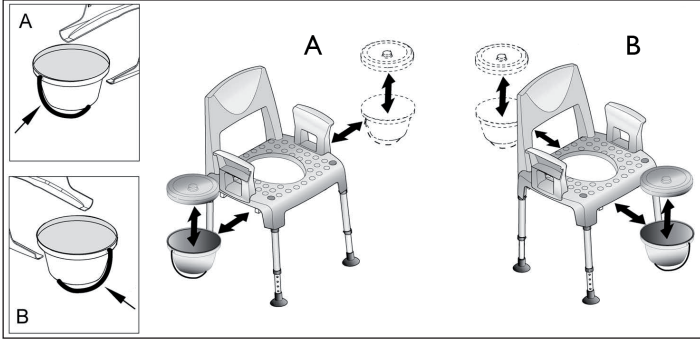
Fig. 11

Intended Use

The Shower stool / Commode chair is to be used solely as an aid for showering or going to the toilet. Any other use is prohibited.

WARNING Risk of falling!

- Do not stand on this product.
- Do not use as a climbing aid.

Temperature for use: 10 - 40 °C

Max. user weight: 160 kg

Service Life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

Safety

WARNING Risk of injury!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.

Scope of delivery

- Fig. 1 - 5.

Assembly

CAUTION Risk of pinching!

- Take care during assembly.

WARNING Risk of falling!

- Ensure that all legs are adjusted to the same height.

IMPORTANT

- The commode pan bracket can only be mounted once and cannot be dismantled. (Fig. 9)

- Fig. 6 - 9.

Usage

- Fig. 10, 11.

Cleaning and disinfection

IMPORTANT

- Take off any fitted accessories.

The product can be cleaned and disinfected using commercially available agents.

- Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

IMPORTANT

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG002 and your local infection control policy.

- Clean by hand.
- Clean without scouring agent.
- Max. cleaning temperature 85 °C for 3 minutes.

Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined above are adhered to.

WARNING

- Check product regularly for damage and to ensure that it is securely assembled.
- Check the arm and backrest fixings and the clips regularly to ensure they are sitting securely.

Re-Use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly and check the safety functions, and reset if necessary.

Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulations.

Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business. Guarantee claims can only be made through the relevant specialist dealer.

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
Technical Data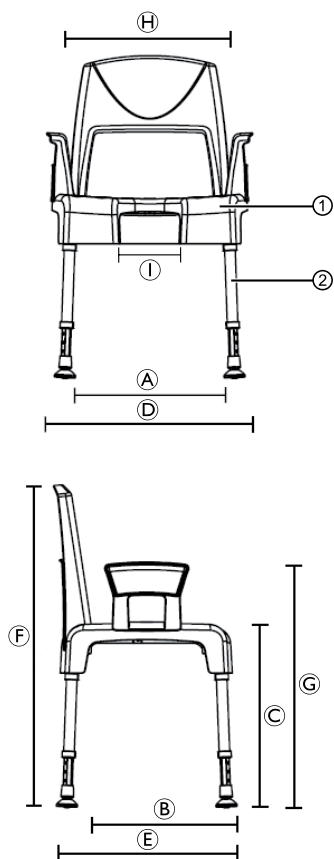

Order numbers	<i>Aquatec® Pico</i>
Shower stool	9.20.001
Shower stool with armrests	9.21.001
Shower chair with backrest	9.23.001
Shower chair complete	9.22.001
Backrest	16469
Armrests (pair)	16468
Measurements	
A	430 mm
B	420 mm
C	425 - 575 mm
D	575 mm
E	520 mm
F	825 - 975 mm
G	605 - 755 mm
H	430 mm
1	180 mm
Depth, hygiene recess	145 mm
Graduation, height adjustment	25 mm
Weights	
Shower stool	3.2 kg
Shower stool with armrests	4.1 kg
Shower chair with backrest	4.0 kg
Shower chair complete	4.8 kg
Max. User Weight	160 kg
Color	White
Materials	
1	PP
2	Aluminium, corrosion-free

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Eastern Europe, Middle East, Russia & CIS:

European Distributor Organisation:
Invacare, Kleiststraße 49
D-32457 Porta Westfalica
Tel. (49) (0)57 31 754 540
Fax. (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

AQUATEC Operations GmbH
Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0
Fax: (49) (0)75 62 7 00 66
kontakt@invacare.com / www.invacare.de

AQUATEC®

Yes, you can.