
# BeSafe®

## HANDICAP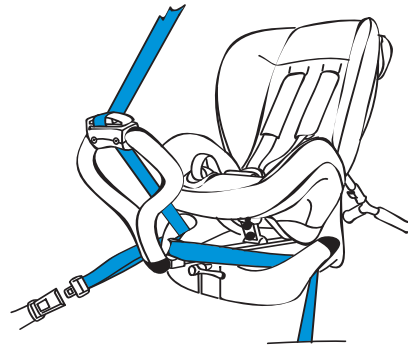

### (N) - Monteringsanvisning - side 2

#### BeSafe iZi Kid Handicap

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### (GB) - Assembly instructions - page 4

#### BeSafe iZi Kid Handicap

BeSafe iZi Kid original assembly instructions must be used when installing the seat in a car, and putting a child in the seat.

Aids in the seat for special needs children are described in this instruction manual.

#### BeSafe iZi Comfort Handicap

BeSafe iZi Comfort original assembly instructions must be used when installing the seat in a car, and putting a child in the seat. Aids in the seat for special needs children are described in this instruction manual.

#### BeSafe Multi Handicap

BeSafe Multi original assembly instructions must be used when installing the seat in a car, and putting a child in the seat. Aids in the seat for special needs children are described in this instruction manual.

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BeSafe iZi Comfort and BeSafe iZi Kid original assembly instructions must be used when installing the seat in a car, and putting a child in the seat. You will find the latest instructions at [www.hts.no](http://www.hts.no). Both of the seats feature the same equipment, a) Pommel b) adjustable side support c) extra head support.

### Adjustment and adaptations

#### Head support

The height of the support is adjusted using a Velcro system. See fig. 1.

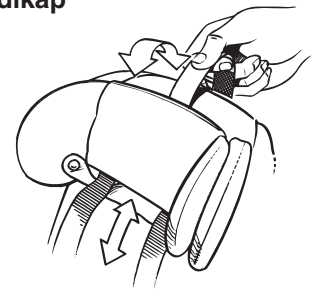
fig. 1

#### Lateral adjustment of side support

Before you begin the adjustment, try the child in the chair and see how much adjustment is needed. A snug fit is an advantage, so that the child is well supported.

1. Withdraw the side padding, (Black plastic part) see fig.2.
2. Press in the lock block, see fig 3, you are now allowed to choose the desired side position.
3. Make sure that the spring-loaded mechanism clicks into place, and that you cannot move it laterally.
4. Replace the side padding, see fig. 2, and pull down the zip fastener.

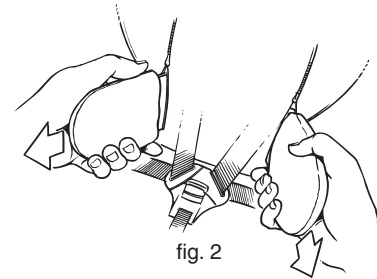
fig. 2

#### Height adjustment of side support

1. Open the zip fastener
2. Loosen the square in PP (stiff wedge under the cover), which is fitted with Velcro. Be careful not to break the plate, see fig 4.
3. Grip the whole adjustable side support, and loosen it from the Velcro.
4. Now you can adjust it upwards and downwards to the desired position.
5. Replace the adjustable side support firmly and check that it is well in place. You cannot change any of the fixing details.
6. Replace the plate. One of the ends of the plate is thin. When the adjustable side support is in the upper position, the thin part must be up, and when the side support is down, the thin part must be down.
7. Close the zip fastener.

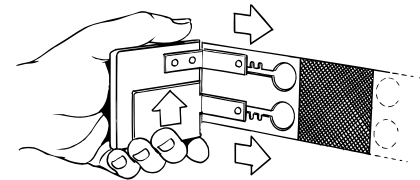
fig. 3

#### Pommel

This is removed for washing. Be careful so that the Velcro does not loosen when you remove this.

#### Cleaning

When removing the cover, follow the BeSafe iZi Comfort and BeSafe iZi Kid original assembly instructions, except for the washing instructions.

**Washing: Hand wash 40 degrees, drip dry**

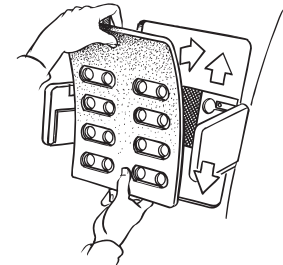
fig. 4

### BeSafe Multi Handikap

BeSafe Multi original assembly instructions must be used when installing the seat in a car, and putting a child in the seat

### Adjustment and adaptations

#### Lateral adjustment of side support

Before you begin the adjustment, try the child in the chair and see how much adjustment is needed. A snug fit is an advantage, so that the child is well supported.

1. Withdraw the side padding, (Black plastic part) see fig.2.
2. Press in the lock block, see fig 3, you are now allowed to choose the desired side position.
3. Make sure that the spring-loaded mechanism clicks into place, and that you cannot move it laterally.
4. Replace the side padding, see fig. 2, and pull down the zip fastener.

#### Height adjustment of side support

1. Loosen the adjustable side support from the Velcro.
2. Set to the desired height, see fig. 5, do not go outside the Velcro area. it is not possible to change any part of the product.
3. Replace the adjustable side support firmly and check that it is well in place.

fig. 5

#### Cleaning

When removing the cover, follow the BeSafe iZi Comfort and BeSafe iZi Kid original assembly instructions, except for the washing instructions.

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