
Instruction for use

IMM1068_IFU.Version4.2023-08-08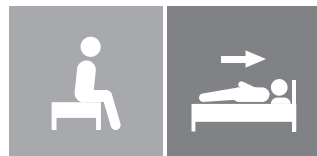

4WayGlide - midi

immedia
by Etac


Index

Illustrations	3
User manual - English	4
Bruksanvisning - Svenska.....	10
Bruksanvisning - Norsk.....	16
Brugervejledning - Dansk.....	22
Käyttöohjeet - Suomi.....	28
Gebrauchsanweisung - Deutsch	34
Gebruiksaanwijzing - Nederlands.....	40
Instruction d'utilisation - Français.....	46
Istruzioni per l'uso - Italiano.....	52
Instrucciones de uso - Español	58


See www.etac.com


Illustrations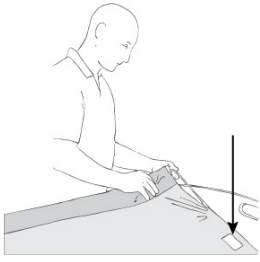

1

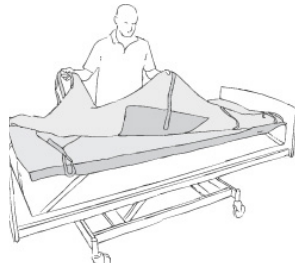2

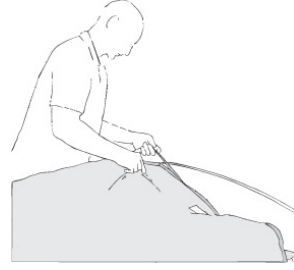
3

4

5

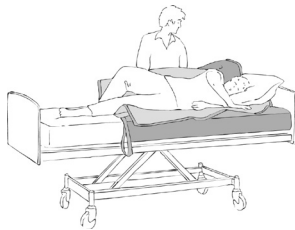6

7

8

9

10

11

ILL.

GB

SV

NO

DA

FI

DE

NL

FR

IT

ES

General

Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, please read this manual carefully.

This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the user or carer.

These products comply with the standards applicable for Class 1 products in the Medical Device Regulation (EU) 2017/745 on medical devices.

At Etac we strive to improve our products all the time and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid. If in doubt, please contact Etac.

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person/company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

Expected lifetime: The device has an expected service life of 1 to 2 years under normal use. The service life of the device varies depending on usage frequency, loads and how often and how it is washed.

For further information on the Etac Patient Handling range, see www.etac.com.

The product can be scrapped in accordance with national regulations.

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Symbols in general

Washing	Manual Cleaning	Warning	Manufacturer
Do not bleach	Never leave on the floor	Production year, month and day YYYY-MM-DD	CE-marking
Tumbledry	Risk of sliding down	Batch no./Lot no.	Keep dry
Iron	Keep out of the sun	User mass limit = maximum rated load	Non sterile
Do not dryclean	Read the user manual	The product can be scrapped according to national regulations	Article number
Medical device	Do not add fabric conditioner	UKCA marked	

Introduction	The person referred to as "the user" is the person who is lying in the product. "The carer" is the person who is handling the product.
Intended Purpose	The in-bed system is an assistive device intended for alleviation of, or compensation for, a functional impairment due to an injury or disability. The device is designed for an individual lacking the ability to reposition or turn themselves in bed due to reduced mobility or physical strength.
Intended User	The device can be used by children, adults and elderly people with a need for repositioning.
Intended Environment	For indoor use in acute care, long-term care and home care.
Indications	For users who are in need of support to be safely and comfortably positioned and turned in lying positions or who are in need of mobilisation, e.g. users with limited ability to move, and/or users who are sensitive to pain, shear or tactile input. Not an exhaustive list. The user group for the device is based on individual health and mobility function and not on a specific diagnosis or age.
Contraindications	There are no known contraindications.
Precautions	Not to be used without NylonSheet. Not to be locked when transfer is completed if there is an increased risk of pressure injury. Not to be unlocked when transfer is completed if there is an increased risk of involuntary movements.
Training Requirements	Only carers who have read and understood the user manual are allowed to use the device.

Practical handling

Positioning

Position the 4Way sheet on the bed.
The “this side up” label should be at the head end (Illustration 1).
Fit the 4Way sheet. You can fit the corner straps around the mattress at the foot end.

The “this side up” label should be at the head end (Illustrations 2 and 3).

Position the user in the bed so that the hips are within the green tape marking on the mattress.

Use

Alternative turning method

If the user is very self-supporting, just one carer can turn the user (Illustration 10).

Positioning

If the user is to remain on their side after bath/care, the user’s back can be supported using a positioning wedge or pillow. (Illustration 11).

Sliding down

Then two carers grasp the straps under the user at the shoulders (and hips, if necessary) and slowly pull the user upwards using weight transference (Illustration 12).

Raising the bed headboard

Use the knee bend before elevating the headend to avoid the user to slide. The bed headboard can now be raised without the user sliding down. If reduced freedom of movement in the upper body is required in the seated position, you can lock the glide system at both sides at the shoulders by fitting corner straps around the mattress.

The “5 smiles” technique

To ensure the most comfortable position (seated or lying), it is imperative that the carers remove any tension on the skin occurring when the user’s head or feet are raised. Loosen the shoulder blades (Illustration 14).
Loosen the legs (Illustration 15). Finally, the carers must make sure that the user’s pillow is correctly placed under the user’s head and neck – first remove the pillow, shake/plump and replace.

Seated position

Once the carer has ensured that any tension on the skin is released, the carer can support the user in the seated position with positioning wedges and pillows (Illustration 15).

Notice

Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that carers receive instruction in manual handling. Etac offers advice and training for carers. For further information, contact Etac.

Always use the correct manual handling techniques

Encourage the user to assist where possible.

To ensure that the user feels safe and that every manual handling is smooth, always plan the manual handling in advance.

There is a risk that the user may slide off. Never leave the user alone on the edge of the bed.

Never leave the product on the floor.

Always conduct a risk assessment, and ensure that the assistive product can be used for the individual user and, in combination with other devices, that it is safe for the user and carers.

It is recommended that the guard rail is in place when the user is left on the glide system, unless a risk assessment has found that the user can be safely left on the glide system without the need of guard rails.

Always lock the glide system when you have completed the transfer.

Do not use the product as a sling.

To reduce sliding to the side if the head end is elevated fit the corner straps around the bed mattress.

If in any doubt - please contact Etac.

Material and cleaning

Material:

GlideMattress:

Top side: Polyester/cotton

Filling: Polyester fibre

Back side: Nylon

Handles: Polyester.

NylonSheet: Nylon, non-slip.

The product should be checked regularly, preferably each time it is used, and especially after washing.

Check that there is no damage to seams or fabric.

Never try to repair a product yourself.
Never use a defective product.

Washing instructions:

Recommended to wash the product before use.

Do not use fabric softener.

This product can be cleaned with a solvent-free cleaning agent with a pH level between 5 and 9, or a 70% disinfection solution.

Products and accessories

Article no.	Product Description	Size (mm)
IM140/150C	4WayGlide Mattress no/lock - midi	W1400xL1500
IM140/150NS	4WayGlide NylonSheet - midi	W1700xL1500

Set

Article no.	Product Description	Size (mm)
IM4328	4WayGlide - midi no lock set (incl. IM140/150C + IM140/150NS)	

Combinations

Article no.	Product Description	Size (mm)
IM53NS	Positioning Wedge S	W300xL530xH200
IM53/80NS	Positioning Wedge L	W300xL800xH200

Service information

Inspection

First inspection:

- Is the packaging intact?
- Read the label on the packaging and check the article no. and product description
- Check that Short Instruction is enclosed – updated User Manual/Instruction for use can be downloaded from www.etac.com or contact your Etac Customer Service or the local dealer.
- Check the label on the product - does it include article no., product description, lot/batch no., cleaning instructions and supplier name?

Periodic Inspection:

- Make sure that your Manual transfer system always is in a perfect condition.
- After wash – always check materials, stitching, handles, buckles.
- If the product shows signs of wear and tear, it must be removed from service immediately.


Visual inspection/Check the product:

Ensure that the materials, seams, stitching, handles, buckles are intact/faultless.

Stability test:

Try to bend the product verify that it feels firm, stable and solid.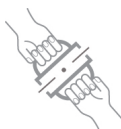

Mechanical load or stress/Test handles:

Draw hard in the handles in opposite directions and control the material, seams and stitching.

Test buckles:

Lock the buckle and draw in opposite directions, control the buckle and the stitching.

Test low/high friction:

Place the product on a firm surface or a bed, place your hands on the material and test the friction by pushing your hands down into the product.

Low friction – slides effortlessly

High friction – no sliding/moving

Check:

Manual transfer aids with handles, straps and buckles:

Manual transfer aids with for sitting and lying transfer and with high or low friction:

Production year, month and day

The product can be scrapped in accordance with national regulations

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Etac Ltd, Unit 60
Etac Immedia A/S