
SEQINETIC™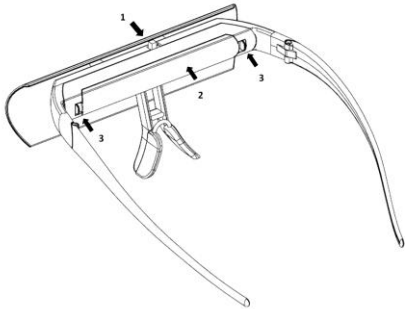

Instructions for use (UK/US)

Daily use

The SEQINETIC™ has 6 light emitting diodes (LED's) that produce a powerful light field reflected into the eyes by the curved reflector. The SEQINETIC™ is worn like a pair of glasses. The lower edge of the reflector should be at eye level or slightly higher when worn correctly. It is not necessary to look upwards towards the reflector. Recommended use per day is 15-60 minutes. (1) On/off switch. (2) Battery cover. (3) Release pin for battery cover.

Replacement of batteries

Gently pull both pins (3) until battery cover (2) can be removed. Use the tip of a pencil if necessary. Remove batteries. Place new batteries in holder. Orientation as shown in the plastic part. Replace the battery cover.

Cleaning

Wipe using a solvent free cloth.

Limits on use

- Your vision may be limited while using SEQINETIC™.
- Do not use while maneuvering a vehicle or using personal transportation such as bikes, skateboards or the like.
- Do not use while operating dangerous machinery or tools.
- Use only in well-lit surroundings.
- Only walk around in familiar areas.
- Do not use on children of 12 years or less.
- Do not expose to or immerse in any kind of liquid.
- If you experience any discomfort during normal use, consult a physician.

Technical specifications

Light intensity:	2,500 Lux
Dimensions:	177mm*154mm*62mm
Weight:	65 g (with batteries)
Batteries:	2*AAA

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SEQINETIC™ is designed and manufactured by: Heveas ApS – Aalborgvej 32 – DK-9575 Terndrup – Denmark – www.seqinetic.com - info@heveas.com © 2012-14 Heveas ApS. All rights reserved.