0,

Manual

# Gewa Control Medi Easy

SE: IR-sändare

GB: IR transmitter

Art.no 429240

ABILIA

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General information about Control Medi .....	22
Symbols.....	22
Inserting Batteries.....	23
Battery Warning Indicator .....	23
Settings.....	23
Alerts when programming.....	23
Cancel Programming.....	23
Locking/Unlocking the Settings Lock.....	24
Recording IR Channels from another Transmitter .....	25
Adding a GewaLink channel.....	26
Adding a 4096 code.....	27
Feedback when pressing buttons .....	28
Setting an acceptance delay.....	28
Direct control .....	29
Setting the channel group.....	30
Button Labelling .....	31
Control Medi Holders .....	31
Cleaning.....	32
Default settings in your Control Medi .....	32
Resetting to Default settings.....	32
Technical data .....	33
Technical data .....	33
Accessories included with your Control Medi Easy .....	33
Other accessories available for Control Medi .....	33
Backups and reinstallation .....	34
Safety and responsibility .....	34
Recycling Waste Electrical and Electronic Equipment.....	34
Conformity with the Medical Devices Directive .....	34
Summary and Quick guide.....	35

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# GB: English Manual Gewa Control Medi Easy

## Introduction

The Gewa Control Medi Easy is a programmable IR transmitter, which replaces ordinary remote controls for TV, DVD, stereo etc. It has totally four different programming positions on one level. Simply press the buttons to send signals. You can also connect an external control switch that can directly control one button on the transmitter.

The Control Medi is a part of the "Gewa Smart House" concept and therefore can be used together with all of these products.

## News and improvements

The Gewa Control Medi transmitter is a further development of previous Gewa IR transmitters. The Control Medi is a new design with an improved transmission range.

The Control Medi is available in three different models. The most advanced feature has eight buttons and four levels giving a total of 28 programmable positions. It also has the possibility to send radio (GewaRadio), and a built-in scanning option for people that cannot press the buttons directly. The simplest model has 4 or 2 buttons (two different keyguards).

When you press a button, confirmation is given as vibration, sound and light.

Gewa Control Medi is available in the following variants:

Art.no	Name	Description
429200	Control Medi Multi	IR/radio transmitter with scanning, 8 buttons
429220	Control Medi Standard	IR transmitter, 8 buttons
429240	Control Medi Easy	IR transmitter, 2/4 buttons (two different keyguards)


## General information about Control Medi

To be able to use your Control Medi there are certain things that you should know:

- Control Medi is delivered preset with GewaLink channels, meaning that it is possible to use the transmitter immediately after the batteries have been inserted. See section *Inserting Batteries*.
- To enter the programming mode, press first Button **4** and then **simultaneously** one of the *Function*-buttons. Different *Function*-buttons are chosen depending on what is to be programmed.
- If you press the wrong button or makes a mistake when programming you can always press Button **4** for two seconds. It works as a *Reset*-button.
- It is important not to wait too long between the different steps when programming. After a certain time your Control Medi automatically leaves the programming mode to save power.
- If a button on your Control Medi or control switch is pressed for more than 1 minute, Control Medi will stop transmitting to save power.
- The settings can be protected by using the Settings Lock.

This picture shows the various terms and functions which are described in the text.

Fig 1

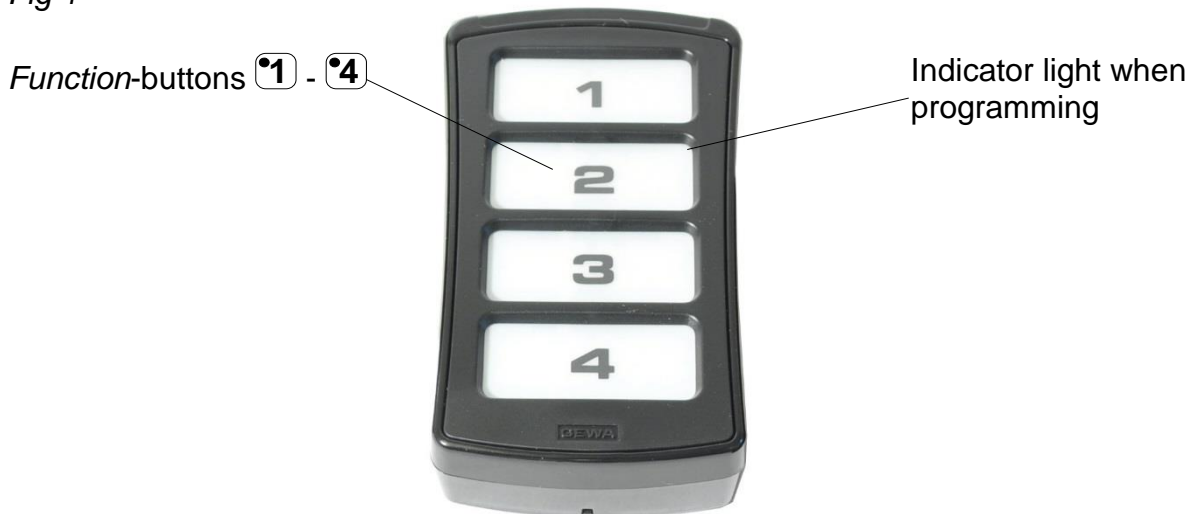## Symbols

The following symbols appear in the text throughout this manual:

- X** and **Y** Any button on Control Medi
- Y** Any button on another transmitter to be programmed
- 1**...**4** Button 1-4
- 4** 2 sec Button **4** held pressed for approximately 2 seconds

## Inserting Batteries

Unscrew the battery lid screws on the back of your Control Medi using a screwdriver and insert three AAA batteries (alkaline LR03 1.5 V). Insert the batteries according to the guidelines at the bottom of the battery compartment. Pay attention to the polarity! It is important to use long life **alkaline** batteries that are protected against leaking battery acid.

## Battery Warning Indicator

Control Medi has a built-in indicator to warn you of low battery voltage. You will hear a sound alert  with one long beep and three short beeps played twice. This occurs about 10 seconds after the transmitter was last used. Replace the batteries as soon as possible.

## Settings

Below is described how to program your Control Medi for different setting options.

To enter the programming mode, you should first press Button  and then simultaneously press the button  for your desired setting option (see below). When you release the buttons a “Setting” alert is heard, and the indication lamp on the button for the setting option flashes. An “Error” alert will sound if you select an invalid setting option.

Press simultaneously	Setting option
 + 	Function
 + 	System
 + 	Reset

This is followed by different button presses depending on what is to be set, see the separate descriptions below.

It is important not to wait too long between the different steps when programming. After a certain time your Control Medi automatically leaves the programming mode to save power.

If your programming is successful, a “Success” alert will sound. If something has gone wrong or takes too long, an “Error” alert will sound.

## Alerts when programming

Whilst you are programming, your Control Medi gives feedback with flashes and different alerts:

“Setting” -  (three increasing tones).

“Success” -  (three positive tones).

“Error” -  (eight negative tones).

## Cancel Programming

To cancel the programming, you can press Button  and keep it pressed for two seconds. It works as a *Reset*-button.

## **Locking/Unlocking the Settings Lock**

In order to protect the programmed settings in your Control Medi, Settings Lock can be activated.

When the Setting Lock is enabled, the settings are protected. If you try to change the settings without first unlocking, the “Error” alert is given.

Do as follows to lock/unlock the Settings Lock:

1. First press **4** and then simultaneously press **1**. When you release the buttons the “Setting” alert sounds and the indication lamp on button 1 flashes.
2. Press **3**.
3. Press **1** to unlock the Settings Lock, or **2** to lock.
4. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

### **Locking/Unlocking the Settings Lock**

Press **4 + 1, 3**, then press your desired option **1** or **2**.

**1** = Unlock. **2** = Lock.

*Default setting: Unlocked*

## Recording IR Channels from another Transmitter

Make sure the batteries in both transmitters are fully charged. Do not use the transmitters in intense light (e.g., sunlight) when programming.

Place the two transmitters on a table with the original transmitter directed towards the rear of your Control Medi, see picture.

1. First press **4** and then simultaneously press **1**. When you release the buttons the “Setting” alert sounds and the indication lamp on button 1 flashes.
2. Press **1**.
3. Press the button **X** where the channel is to be recorded. The indicator light for the selected button glows steadily and the “Press” alert sounds.
4. Press and hold the corresponding button on the original transmitter pressed. Release the button when the indicator light for the selected button goes off and the “Release” alert sounds. This may take up to 10 seconds. If the indicator light glows again and the “Press” alert sounds, you should press the button one or more times until the “Success” alert sounds.

If the “Error” alert sounds, the recording has failed. Try again from step 3.

5. If the recording of the first button was successful, repeat steps 3 and 4 until all buttons are programmed.
6. Finish by pressing **4** for 2 seconds.
7. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.
8. Now test your recorded functions on the device that is to be controlled.

Original transmitter

### Recording IR Channels

Press **4** + **1**, **1**, select button **X**, select button **Y**, then continue or finish with **4** 2 sec

## Adding a GewaLink channel

Your Control Medi Easy comes with GewaLink channels 16-19. If you want to use an other GewaLink channel on a certain button, you can insert a chosen GewaLink channel like this:

1. First press **4** and then simultaneously press **1**. When you release the buttons the “Setting” alert sounds and the indication lamp on button 1 flashes.
2. Press **2** and then press **1**.
3. Select the button **X** where the GewaLink channel is to be saved.
4. Select your **Channel range**, **1** = GewaLink 00-15.  
**2** = GewaLink 16-31.  
**3** = GewaLink 32-47.  
**4** = GewaLink 48-63.
5. Select your **Channel group** according to *Table 2* below.
6. Press the **Button** that corresponds to your desired GewaLink channel in the selected channel group according to *Table 2* below.
7. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

**Table 2: Channel group and button to add GewaLink channel**

	<b>Channel range 1</b> : GewaLink 00-15				<b>Channel range 2</b> : GewaLink 16-31			
Button	Ch. group <b>1</b>	Ch. group <b>2</b>	Ch. group <b>3</b>	Ch. group <b>4</b>	Ch. group <b>1</b>	Ch. group <b>2</b>	Ch. group <b>3</b>	Ch. group <b>4</b>
<b>1</b>	00	04	08	12	16	20	24	28
<b>2</b>	01	05	09	13	17	21	25	29
<b>3</b>	02	06	10	14	18	22	26	30
<b>4</b>	03	07	11	15	19	23	27	31
	<b>Channel range 3</b> : GewaLink 32-47				<b>Channel range 4</b> : GewaLink 48-63			
Button	Ch. group <b>1</b>	Ch. group <b>2</b>	Ch. group <b>3</b>	Ch. group <b>4</b>	Ch. group <b>1</b>	Ch. group <b>2</b>	Ch. group <b>3</b>	Ch. group <b>4</b>
<b>1</b>	32	36	40	44	48	52	56	60
<b>2</b>	33	37	41	45	49	53	57	61
<b>3</b>	34	38	42	46	50	54	58	62
<b>4</b>	35	39	43	47	51	55	59	63

### Adding a GewaLink channel

Press **4** + **1**, **2**, **1**, **---**, choose the button **X**, choose the **Channel range**, choose the **Channel group**, then choose GewaLink channel.

See *Table 2* for your choice of Channel groups and GewaLink channels.

Example: You want to add GewaLink channel 28 onto a certain button **X**.

Press **4** + **1**, **2**, **1**, **X**, **2**, **4**, **1**.

## Adding a 4096 code

A coded channel is normally used for functions that require strict authorization such as apartment or house doors. The code has 4096 different combinations.


If you have to program a new coded channel there are two methods of doing so. You can set the 4096 code on any button.

**Method one:** record the code from another transmitter, as described under *Recording IR channels from another Transmitter*.

**For method two** a binary code is entered. Make up the binary code yourself or else use a code that is already set in an IR receiver or IR transmitter.

There are two types of IR receivers: an older type of receiver where you set the code with a 12-pole switch, or a newer type where the IR receiver is programmable. The principle is the same when it comes to creating the binary code. *Fig 3* shows a 12-pole switch. Switches 1, 4, 6 and 12 are in the “On” position, the others are in the “Off” position.

Fig 3

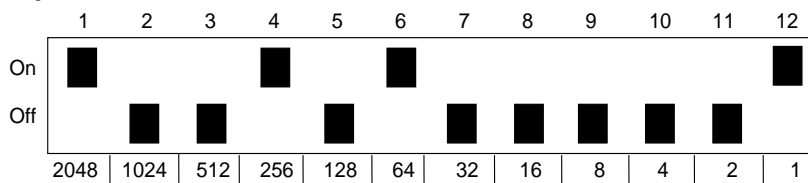The following example describes how the binary code is created according to method two:

1. First press **4** and then simultaneously press **1**. When you release the buttons the “Setting” alert sounds and the indication lamp on button 1 flashes.
2. Press **2** and then press **2** again.
3. Select the button where the 4096 code should be saved.
4. Enter the code by pressing on **1** or **2**.  
**1** = switch in Off position and **2** = switch in On position.  
Press twelve times in turn; **2, 1, 1, 2, 1, 2, 1, 1, 1, 1, 1, 2**, (*Fig 3*).
5. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.
6. The code is now set on your selected button. NOTE! The code above is just an example and should be different for different users. Make a note of the chosen code for future usage.

### Adding a 4096 code

Press **4 + 1, 2, 2**, choose the button **X**, then press **1** or **2** 12 times.

**1** = Off position. **2** = On position.

## Feedback when pressing buttons

Your Control Medi always gives feedback in the form of light signals when you press a button. You can also get feedback as sound alerts and vibration. It is possible to get feedback from all three ways simultaneously, or you can set up what will be on or off (light is always on).

Do as follows to set the levels of sound and vibration.

1. First press **4** and then simultaneously press **2**. When you release the buttons the “Setting” alert sounds and the indication lamp on button 1 flashes.
2. Press **1**.
- 3a. Sound level:  
Press **1** to lower the volume, press **2** to increase the volume. Repeat pressing until the desired volume is set.  
*and/or:*
- 3b. Vibration level:  
Press **3** to lower the vibration, press **4** to increase the vibration. Repeat pressing until the desired vibration is set.
4. Finish by pressing **4** for 2 seconds.
5. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

### Set level for sound and vibration

Press **4** + **2**, **1**, choose the setting, then  2sec.

**1** = Lower volume. **2** = Increase volume. **3** = Lower vibration. **4** = Increase vibration.

*Default setting: Sound alert = 1 (0 – 5) Vibration = 1 (0 – 5)*

## Setting an acceptance delay

An acceptance delay can be set for pressing the buttons. Acceptance delay is the time a button has to be pressed for before a function is activated. An acceptance delay can be set from 1 to 4. 1 is no delay at all (off) and 4 is the maximum delay (1 sec.).

1. First press **4** and then simultaneously press **2**. When you release the buttons the “Setting” alert sounds and the indication lamp on button 2 flashes.
2. Press **2**.
3. Press your desired value for the acceptance delay, **1** is off, and **4** is max.
4. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

### Setting an acceptance delay

Press **4** + **2**, **2**, then choose the setting.

**1** = 0 sec. **2** = 0.3 sec. **3** = 0.6 sec. **4** = 1 sec.

*Default setting: Acceptance delay = OFF (0 sec).*

## **Direct control**

Your Control Medi can directly control a button with an external control switch. You can choose which button that should be activated when you press the control switch.

A control switch with 3.5mm plug will fit in to the side of your Control Medi.

Do as follows to activate/deactivate Direct control:

1. First press **4** and then simultaneously press **2**. When you release the buttons the “Setting” alert sounds and the indication lamp on button 2 flashes.

2. Press **3**.

3a. Activate direct control.

Press **2**, and then press the button **X** that you want to be activated when pressing the control switch.

*or:*

3b. Press **1** to turn off direct control.

4. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

### **Activate/deactivate Direct control**

Activate direct control: Press **4 + 2, 3, 2**, choose the button **X**.

Deactivate direct control: Press **4 + 2, 3, 1**.

*Default setting: OFF (none)*

## Setting the channel group

Your Control Medi has the option to set different channel groups for the transmitter to operate within. When the change of channel group is applied, programmed channels will disappear.

Normally, the channel group does not need to be changed. If several GewaLink transmitters are to be used in the same area and you do NOT want to control the same equipment, it may be necessary to use different channel groups.

Table 3 shows the different settings.

**Table 3: Channel groups**

	Channel range, GewaLink channels			
	00-15 ①	16-31 ②	32-47 ③	48-63 ④
Channel group ①	00-03	16-19	32-35	48-51
Channel group ②	04-07	20-23	36-39	52-55
Channel group ③	08-11	24-27	40-43	56-59
Channel group ④	12-15	28-31	44-47	60-63 <sup>1</sup>

<sup>1</sup> Channel 62 is a stop code and cannot control any function. On the other hand you can store channels/codes on this button which in turn can control a function.

Do as follows to set the **Channel group**:

1. First press ④ and then simultaneously press ②. When you release the buttons the “Setting” alert sounds and the indication lamp on button 2 flashes.
2. Press ④.
3. Choose the Channel range ① to ④ according to table 3 above.
4. Choose the **Channel group** ① to ④ according to table 3 above.
5. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

### Set channel group

④ + ②, ④, choose Channel range, choose **Channel group**.

See *Table 3* for choice of channel range and channel group.

*Default setting: GewaLink channel 16-19*


## Button Labelling

The buttons on your Control Medi are marked with numbers as standard. The numbered label can be replaced with a personalised design. On Abilia's website it is possible to create your own button labels using pre-designed templates with symbols and text.

The document can also be downloaded. The labeling is printed on a clear sheet or white paper and inserted through the slit on your Control Medi. Before the button label is inserted, the keyguard must be removed.

Remove the keyguard by gripping the middle, and then lifting it straight up, see photo.

An A4 sheet for making notes is included with your Control Medi.

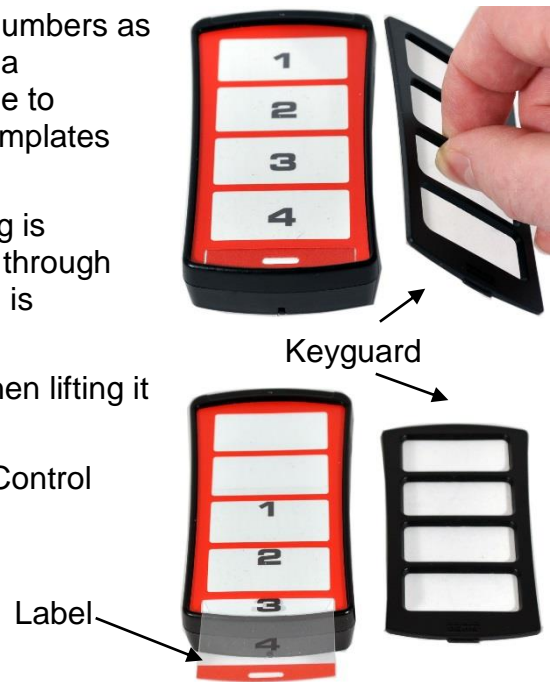## Control Medi Holders

Rubber feet are mounted on the underside so that your Control Medi can lie still on flat surfaces. Your Control Medi can also be equipped with a clip (accessory) which can be attached to a belt or pocket.

As an optional accessory, there is also a holder that fits to your Control Medi. This holder can be assembled for mounting on to a table or attaching to a pipe. The holder can also be mounted on a mobile stand.

Another option is to hang your Control Medi around the neck. A neck strap is supplied with your Control Medi.

Do as follows to attach the neck strap:

1. Unscrew the battery lid.
2. Place the thin loop of the neck strap in the groove around the screw mount.
3. Screw back the battery lid back down.
4. Adjust the length of the neck strap.

### Warning!

The string lock is a safety feature to prevent strangling. Make sure that the ends of the neck strap can be pulled out eg. if the transmitter gets caught on something. Never make knots in the neck strap!

## Cleaning

Your Control Medi can be cleaned with a damp cotton wool pad with surgical spirit, Chlorhexidine or ethyl alcohol.

It can also be cleaned with a damp cloth (weak solution of washing up liquid). Wipe dry after cleaning.

### Warning!

Do not use any disinfectant except surgical spirit, Chlorhexidine or ethyl alcohol.

These liquids are inflammable, handle with care!

## Default settings in your Control Medi

On delivery your Control Medi Easy will be set with certain default settings as follows:

- Acceptance delay is set to value OFF
- Feedback sound level is set to value 1 (0 – 5)
- Feedback vibration level is set to value 1 (0 – 5)
- GewaLink channels 16-19

### Resetting to Default settings

For Control Medi it is possible to reset one button or reset the entire Control Medi to Default settings.

Do the following to restore:

1. First press **4** and then simultaneously press **3**. When you release the buttons the “Setting” alert sounds and the indication lamp on button 3 flashes.
- 2a. Reset **one button**:  
Press **2**.  
Choose the button **X** to be restored.  
*or:*
- 2b. Reset to **factory settings** (all buttons):  
Press **1**.  
Press **1**, **2**, **3** in sequence.
3. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

### Resetting to default settings

To reset one button:

Press **4** + **3**, **2**, choose button **X**.

To reset to factory settings:

Press **4** + **3**, **1**, **1**, **2**, **3**.

## Technical data

### *Technical data*

Power consumption at rest:	Approx. 2µA
Power consumption at transmitting	Approx. 40mA (GewaLink channel)
Input control switch type:	3.5mm jack plug 2-pole
Battery life (button use only):	Approx. 2 years (100 seconds transmitting per day with GewaLink channel)
Programming locations:	4 places
Operating force buttons:	< 160g
Range:	40-50 m (GewaLink channels)
EMC-standard emission: " immunity:	EN 301 489-3 V1.4.1:2002, EN(IEC) 61000-6-2:2005, EN(IEC) 61000-6-3:2007, selected parts from EN 300 220-2 V2.3.1:2010
Computer Connection:	Via 429252 Control Tool USB interface
Measurements (L x W x H):	100 x 55 x 22 mm (incl. rubber feet)
Weight (incl. batteries):	105g
Operating temperature (excl. batteries):	-20°C to +45°C

### *Accessories included with your Control Medi Easy*

Art.no	Name	Description	Qty
429256	Control Medi Labelling 4	For labeling of buttons	1 pcs
429254	Control Medi Labelling 2	For labeling of buttons	1 pcs
-	Note sheet	A4-sheet for noting your programmed channels	1 pcs
429258	Control Medi Keyguard 4	Keyguard for 4 buttons	1 pcs
429259	Control Medi Keyguard 2	Keyguard for 2 buttons	1 pcs
890410	Neck strap		1 pcs
8375	Battery	LR03 1.5V	3 pcs

### *Other accessories available for Control Medi*

Art.no	Name	Description
429249	Control Medi Holder 6 mm Hole	For assembly of Control Medi to different systems with a 6 mm hole
429250	Control Medi Holder 10 mm Pipe	For assembly of Control Medi to 10 mm pipe system
429251	Control Medi Belt Clip	
429252	Control Tool USB interface	Adapter for connecting a Control Medi to PC

## Backups and reinstallation

Your Control Medi can be connected to a computer in order to make a backup of the settings and recordings of your transmitter. The backup copy can be used to reinstall the settings and recordings onto your transmitter. Use the Control Tool USB Interface accessory (sold separately). The software can be downloaded from Abilia's website. [www.abilia.org.uk](http://www.abilia.org.uk). See instructions on the website.

## Safety and responsibility

Abilia has conducted a risk analysis for the product with reference to SS-EN 14971 for Medical equipment. The instructions for use and the technical specifications for the product have been drawn up in respect of the safety assessments carried out in this document.

It is always important to be aware of safety when first using this product and with its subsequent use. It is not possible to describe in the instructions all the situations in which the product must not be used. If you are not sure whether you have started using the product properly or set it up correctly, please contact your supplier.

Abilia is not liable for any loss, damage or consequential loss resulting from any use, correct or otherwise, lack of follow-up, incorrect set-up, incorrect connection or similar misuse. Abilia financial liability is at all times limited to the value of the product itself.

## ***Recycling Waste Electrical and Electronic Equipment***

The Control Medi contains electronics components that must be recycled. When the product is no longer of use, hand it in to a recycling centre for waste electrical and electronic equipment.

## ***Conformity with the Medical Devices Directive***

The Control Medi is marketed as a technical aid for people with disabilities and meets the requirements in the Medical Devices Directive 93/42/EEC including relevant EMC and electrical safety requirements.

## Summary and Quick guide

### Locking/Unlocking the Settings Lock

Press **4** + **1**, **3**, then press the desired option **1** or **2**.

**1** = Unlock. **2** = Lock.

*Default setting: Unlocked*

### Recording IR Channels

Press **4** + **1**, **1**, select button **X**, select button **Y**, then continue or finish with **2** 2 sec

### Adding the 4096 code

Press **4** + **1**, **2**, **2**, select your button **X**, then press **1** or **2** 12 times.

**1** = Off position. **2** = On position.

### Setting the levels for sound and vibration

Press **4** + **2**, **1**, choose setting, **4** 2sec.

**1** = Lower volume. **2** = Increase volume. **3** = Lower vibration. **4** = Increase vibration.

*Default setting: Sound alert = 1 (0 – 5) Vibration = 1 (0 – 5)*

### Setting the acceptance delay

Press **4** + **2**, **2**, then choose setting.

**1** = 0 sec. **2** = 0.3 sec. **3** = 0.6 sec. **4** = 1 sec.

*Default setting: Acceptance delay = OFF (0 sec).*

### Activating/deactivating Direct control

Activate direct control: Press **4** + **2**, **3**, **2**, then choose button **X**.

Deactivate direct control: Press **4** + **2**, **3**, **1**.

*Default setting: OFF (none)*

### Resetting to default settings

To reset one button:

Press **4** + **3**, **2**, choose the button **X**.

To reset to factory settings:

Press **4** + **3**, **1**, **1**, **2**, **3**.

Abilia AB, Kung Hans väg 3, Sollentuna, Box 92, 191 22 Sollentuna, Sweden  
Phone +46 (0)8-594 694 00 | Fax +46 (0)8-594 694 19 | [info@abilia.se](mailto:info@abilia.se) | [www.abilia.se](http://www.abilia.se)