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NORMATIVA

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INSTRUCTIONS FOR USE

NORMATIVE

This product is a Class I medical device and complies with Regulation (EU) 2017/745 of the European Parliament and of the Council of April 5, 2017 on Medical Devices.

This product should not be disposed of in the waste container. There are specific collection points where you can deposit them. For the disposal of the container and the product, strictly comply with the legal regulations of your community.

INTENDED USE

Support device for washing and bathing. It is a technical aid that allows people with reduced mobility, who cannot make conventional use of the shower, to remain seated during the bath.

TECHNICAL FEATURES

- Made of aluminium
- It has holes for good drainage and prevent slipping
- Designed with handles on both sides to increase its user stability.
- Adjusting in Height: 47 - 50 cm
- Seat Size: 31 x 50 cm
- Net weight: 2 Kg
- Maximum weight cap: 120 Kg

LIFESPAN

The useful life of this product is influenced by their conditions of use and storage and by the regularity of their maintenance. By imperatively respecting the data provided in this instruction manual, it is possible to keep the item in perfect condition for 24 months.

WARRANTY

We guarantee our customers the products against any manufacturing defect for two years from the date of the invoice. For more information, see our general conditions of sale.

NOTICE

Any serious incident related to the product must be reported by Sucesores de Manuel García, S.L: and to the competent authority of the Member State in which the user and / or patient are established.

REF 2192

FOLDABLE SHOWER BENCH

Sucesores de Manuel García S.L. CIF B12321162
Carretera de Castellnovo, Km 0, 12400 Segorbe (Castellón), Spain
Tel.: +34 964 71 03 22 - e-mail: comercial@garcia1880.com
www.garcia1880.com

PRECAUTIONS

- Only use the bench on level and firm ground
- Do not modify the bathroom bench structure and do not fix anything on it.
- Do not use the bathroom bench as a ladder. Do not use it on the top of other furniture or fitments.
- Do not exceed the maximum weight.

CONTRAINDICATIONS

- The use of the shower bench is not suitable for people with strong perception disorders, balance disorders or an inability to sit.

HOW TO CLEAN IT

- For cleaning, use neutral detergent and hot water. Do not use abrasive products.
- Do not use materials that can scratch the frame.

MAINTENANCE

- Regularly check the screws to ensure that they are securely fastened. If there is any loose, please contact your dealer, you may need to replace it with a new one immediately.
- Do not make any modifications (mechanical alterations) to the crutch so as not to compromise its safety of use.

SYMBOLS

REF	Product reference number
MD	Medical Device
CE	EC certificate
	Manufacturer
	Manufacturing date
LOT	Lot number

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