




homecraft HOMECRAFT ADJUSTABLE SHOWER CHAIR WITH BACK

Item#: 081706373

Sturdy, yet comfortable and lightweight shower chair makes bathing quick and easy. Rust resistant construction ensures this chair will last as long as you need it. The shaped seat and back provide a large area for sitting, and legs are adjustable from 15" - 19" in 1/2" increments.

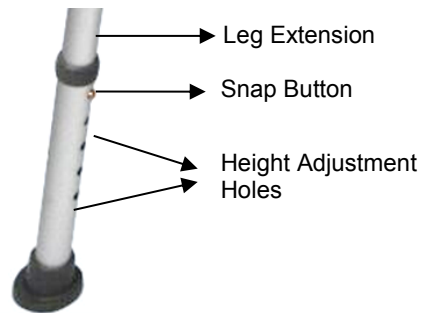

Please read these instructions before attempting assembly. If they are not clear, contact a healthcare professional or Performance Health for more direction.

Before Assembly

Inspect components for any damage that may have happened during shipping or handling. In the case of damage, or missing parts, contact Performance Health immediately at 1-800-323-5547.

Assembly Instructions

Start by setting the chair upside down on an even, secure surface. Attach the back to the chair base by pressing the buttons on the support frame and inserting them into the support brackets on the bottom of the seat. You will hear two "snaps" when the pieces are fastened securely.

Adjusting Height of Chair

Choose the desired height between 15" and 19" in 1/2" increments. Set the height by pushing the snap button to release it from the adjustment hole, then moving the leg extension up or down to the desired height. Release the snap button until it clicks into place, indicating that the button is fully engaged in the hole. Repeat, making sure all the legs are set to the same height. To be sure legs are locked into place, push down or pull on the legs.

Use and Care

Clean with water and a mild soap or non-abrasive cleaning product, rinse with warm water. Wipe bath seat dry after each use.

Precautions

- Before every use, check the button/hole connection and that all four legs are even in height and secure in the shower/tub.
- Ensure that all push buttons are locked in place before use.
- Do not exceed the maximum user weight capacity of 400 lbs.
- Always inspect the shower chair to ensure it is properly positioned and stable before using.
- Do not use if chair is wobbly or unstable.
- On a regular basis, inspect rubber tips to determine if they are ripped or cracked. In the case of wear, replace tips immediately.
- Do NOT stand on chair.

Limited Product Warranty

Performance Health warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PERFORMANCE HEALTH MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANT ABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Performance Health of any manufacturing defect within five days of discovery of the defect. Upon a determination by Performance Health that the product is defective, Performance Health's sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Performance Health, and to reimburse customer for any return freight costs. In no event shall Performance Health be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order

©2018 Performance Health

Distributed by:
Performance Health
28100 Torch Parkway
Ste. 700
Warrenville, IL 60555, USA
USA Phone: 800-323-5547

Authorized European Representative
Performance Health, Ltd.
Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK
Phone: 08444 124 330
International Phone:
+44 (0) 1623 448 706