

sippa


The medical drinking aid for people
with dysphagia or restricted mobility



The sippa drinking system helps people with dysphagia and people with restricted mobility to follow their normal drinking habits unaided.

Through the patented sippa mechanism, the affected person can drink with its head down and avoid swallowing. The cup can be emptied without changing the head position.

The elegant cup shape and color do not make sippa look like an aid, but making it look like a familiar drinking vessel. This raises the acceptance and the quality of life of the user.

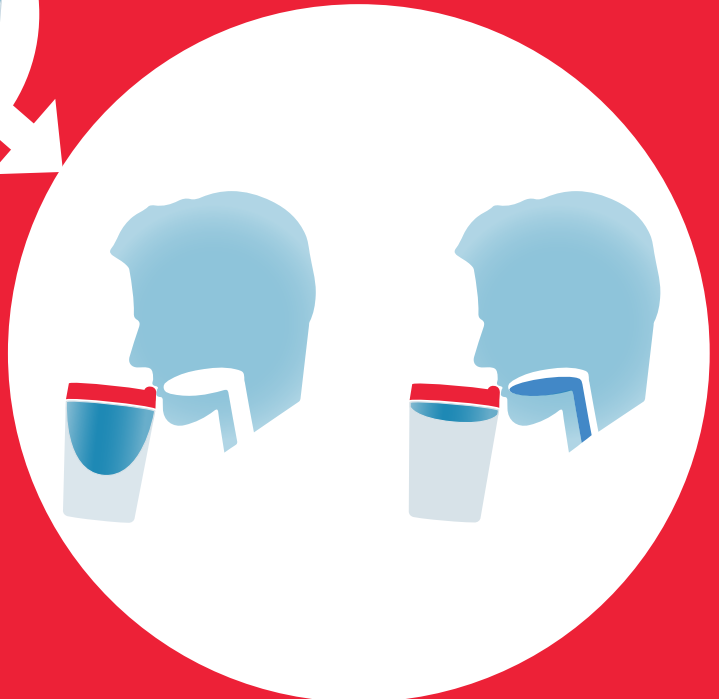


**Explanatory
Video here**




<https://youtu.be/S4YxnyCTtOw>

Application of sippa

sippa can be used with or without compensation mechanism. Using the mechanism, the sippa pad is placed in the cup before filling and screwed tight with the clamping ring. When pouring onto the integrated membrane, this expands to the maximum filling of approx. 200 ml.

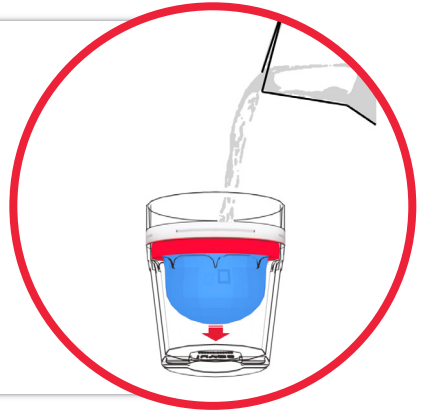

When drinking with sippa you can drink from a static position with your chin towards your chest, the so-called Chin-Tuck-maneuver. The restoring force of the membrane ensures a constant level of liquid at the upper edge of the cup and thus enables this preferred and safe drinking position.

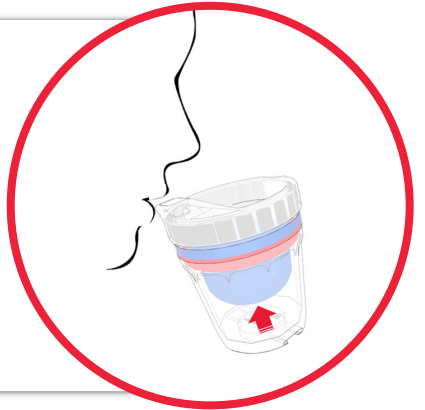

Functions of sippa

sippa has many features which are helpful in the therapeutic and nursing of drinking and swallowing problems.

- Enables drinking position with lowered chin
- Transparent lid with marked drinking opening
- Spill protection with sealing valve
- Optional two-sided handles
- Cleanable in the dishwasher
- Shatterproof material
- Combinable with thickened liquid
- Natural lip approach
- Misuse protection by valve

Medical benefit for patients with dysphagia

sippa promotes an optimal head position for carefree drinking, the so-called Chin-Down- or Chin-Tuck-maneuver, especially for people with dysphagia. A normal wetting of the lips and normal swallowing amounts allow a natural swallowing mechanism.

- Optimal head position with lowered chin
- Secured swallowing mechanism
- Usual wetting of the lips
- Normal swallowboli



Acceptance by normality

sippa is hardly recognizable as an drinking aid from the outside and has the shape and color of an ordinary drinking vessel. As a result, sippa fits easily into any table setting. The user thus feels no exclusion during use. This ensures a higher acceptance and more enjoyment of life for the user.

- Non-stigmatizing design
- Low weight
- Manageable size



Modularity for every situation

An individual problem requires an individual aid. Many aids are not customizable and therefore only limited applicable.

sippa is a completely modular system that adapts to the needs of the user. Spill protection, handle and the patented sippa mechanism can optionally be combined with the standard cup.

- Optional handles
- Optional spill protection
- Protection against misuse in dementia
- Standard cup can be used as standard equipment




More independence

The easy handling and the spill protection allow an independent drinking, without having to rely on nursing staff. Due to the gained independence, dehydration and other nutrient deficiencies can be prevented.


- Protection against spillage
- Also possible with limited mobility
- Facilitates drinking in difficult situations




Lid with spout

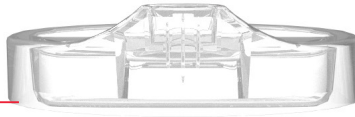

Valve against spillage
and misuse

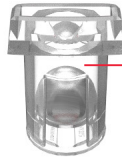

Clamping ring for
sippa pad

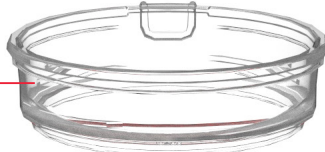

sippa pad with
membrane



Base cup



Handle



iuvas medical GmbH
Martin-Luther-Straße 2
81539 München
Tel +49 89 2154 60 330
contact@iuvas.de

More information at www.iuvas.de

iuvas

