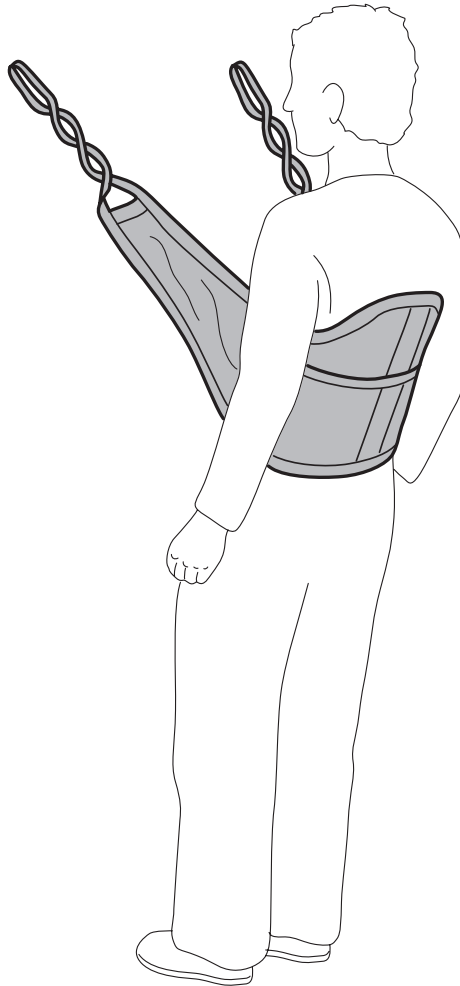
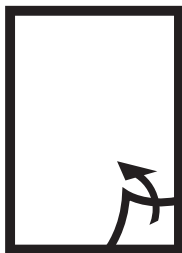
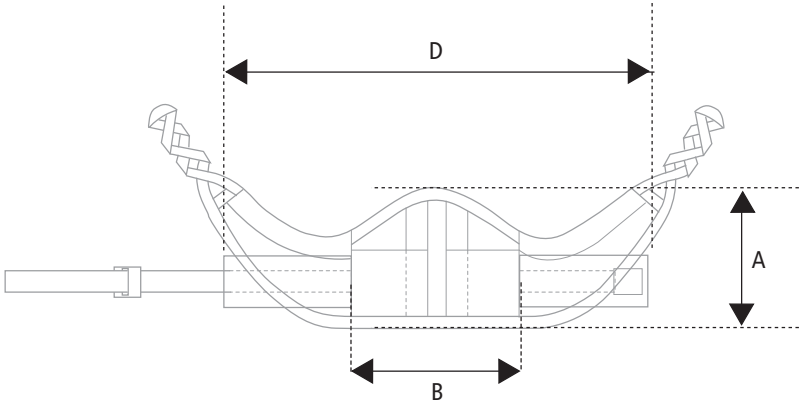
Nova Stand-up Manual

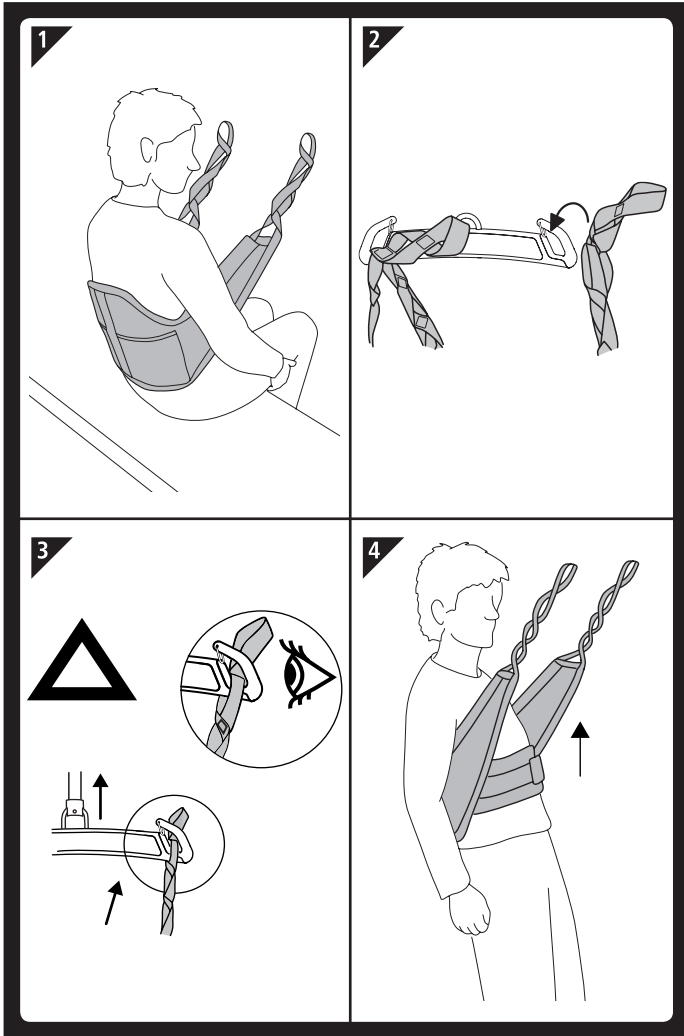
A

	XS	S	M	L	XL
A	28	33	33	34	34
B	12	24	38	54	60
D	71	82	98	108	121

DA Indhold

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General information

Thank you for choosing a sling from Etac.

Etac's slings is tested by accredited and independent testing institutes and comply with the requirements of the Medical Devices Directive Class 1 products (MDD 93/42/EEC).

The slings also comply with the requirements in standard ISO 10535.

In this manual, the user is the person being lifted. The assistant is the person operating the sling.

This symbol appears in the manual together with text. It indicates a need for caution when the safety of the user or assistant may be at risk.

We therefore reserve the right to make changes to our products without prior notice. The dimensions shown in the drawings or other material are for guidance only.

We cannot be held responsible for any printing errors or omissions.

Warranty: Two years for material and manufacturing defects with correct use.

For further information on Etac's range of slings and lifts, see www.etac.com

Properties

The Stand-up sling has a centred band at the back and pockets at the bottom to make it easier to position correctly and to provide good support for the back. The Stand-up sling can only be used with the sit-to-stand lift.

The sling is suitable for users who are unable to get into a standing position but are nevertheless able to stand.

Approved for users up to 350 kg/
770 lbs.

Remember

- The Stand-up sling is only for use with the sit-to-stand lift.
- In order to avoid injury during handling and use, please read through the lift and sling manual.
- It is important to choose the right size. 
- It is important to follow the instructions on how to perform a transfer.
- It is important to follow the instructions on how to perform a transfer. Etac offers help and training for care personnel. Contact Etac for further information.
- Pay particular attention to wear and damage to seams, material, straps and loops.
- Do not use damaged lifting accessories.
- Never use a defective sling!
- The sling should not be stored in direct sunlight.
- If required, consider using one or more co-assistants from time to time.
- Plan the lifting operation prior to the lift to ensure it takes place as securely and as smoothly as possible.
- It is important to check that the straps are correctly fastened to the slingbar once the straps are fully stretched, but before the user is lifted from the support.
- Never leave a user unattended in a lifting situation.
- If the lifting frame which is used has a catch to prevent accidental unhooking, the catch must be in working condition. Lifting frames without a catch or where the catch does not work must not be used.

Practical operation

fig. B

Make sure that the wheels of a wheelchair, bed, bunk, etc. are locked during operation.

1. Fit the sling around the user. Make sure the user's arms are on the outside.
2. Bring the sit-to-stand lift up to the user and position the user's legs on the footrest and suspend the sling from the slingbar hooks.
3. Stretch the straps of the sling without lifting the user. Check that the sling is correctly suspended from the hooks and is correctly positioned around the user.
4. The user should lean backwards during the lifting operation.

Sources of errors/User sitting incorrectly

- Is the lifting sling the right size?
- Is the lifting sling correctly positioned?
- Are all the straps lifting simultaneously?
- Is the sling connected correctly according to individual prescription?
- If the user is wearing clothing made of slippery material, does he/she slide down/out of the sling more easily?

EN

Cleaning

Washable up to 90° C/194 F.

Tumble-dry up to 45° C/113 F

NB!

Do not bleach, autoclave or dry-clean.

The sling must be checked on a regular basis, preferably before each use but especially carefully after washing.

Check there is no wear and damage to seams, material, straps and loops.

NB! Never repair a sling yourself. A worn sling should never be used.

EN

Accessories

Extension strap

For use in situations where a particular seated position, sitting angle or lifting height is required.

Available in lengths 14 cm/ 6 Inches or 18 cm/7 Inches, sold separately in bags.

Marking straps

Marking straps for marking tested loop positions on the sling for lifting.

Two supplied with every sling in plastic bags. Can also be purchased as an accessory.

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Etac Sverige AB