BE1580/BE1370/BE1350
Alarm clock Visit, Pro and Classic

EN Model overview

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DK Modeloversigt

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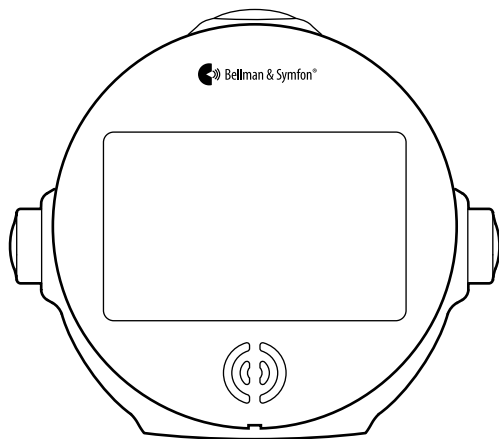
FI Mallikatsaus

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
NL Modeloverzicht

NO Modeloversigt

SE Modellöversikt

Classic

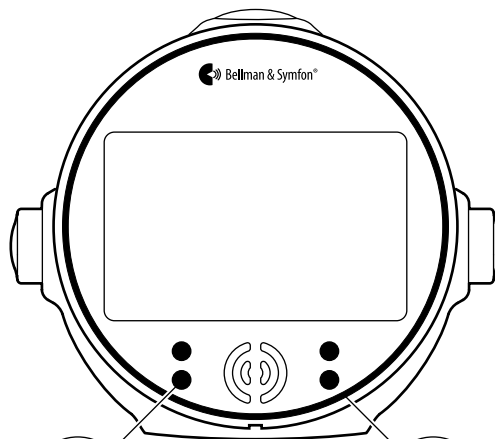
Pro


Visit

- (EN) Remove the battery tab
- (DE) Sicherungsstreifen herausziehen
- (DK) Aftagning af batteridæksel
- (ES) Retirar la lengüeta de las pilas
- (FI) Poista pariston suojauska

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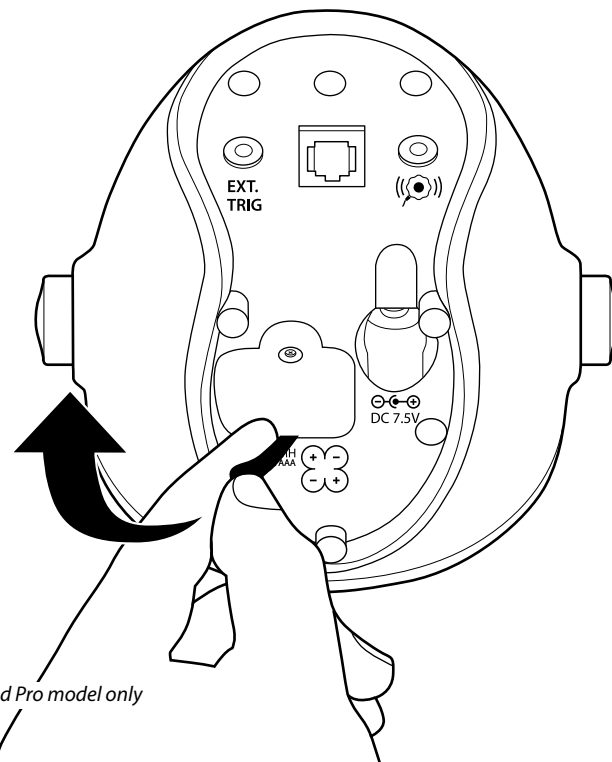
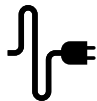
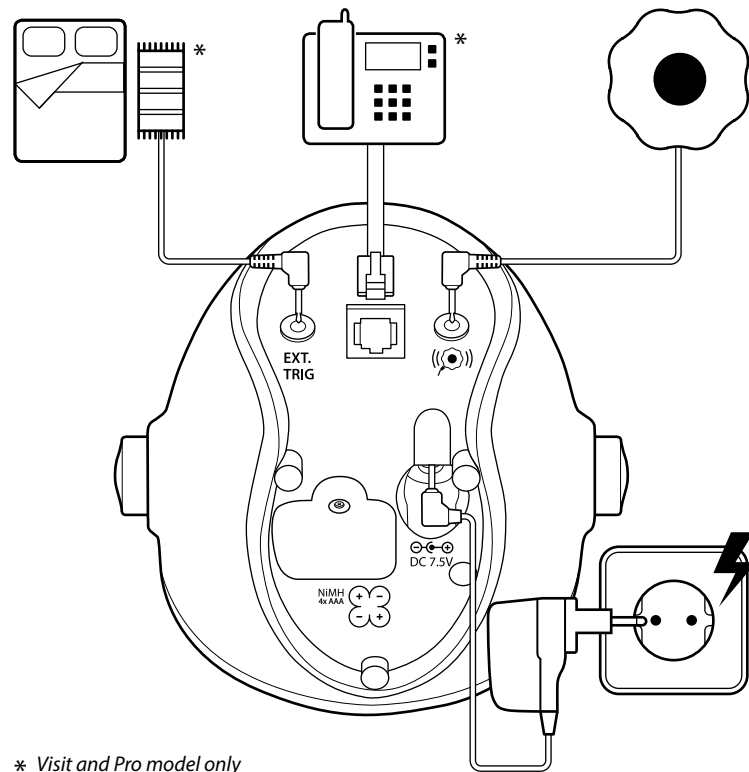
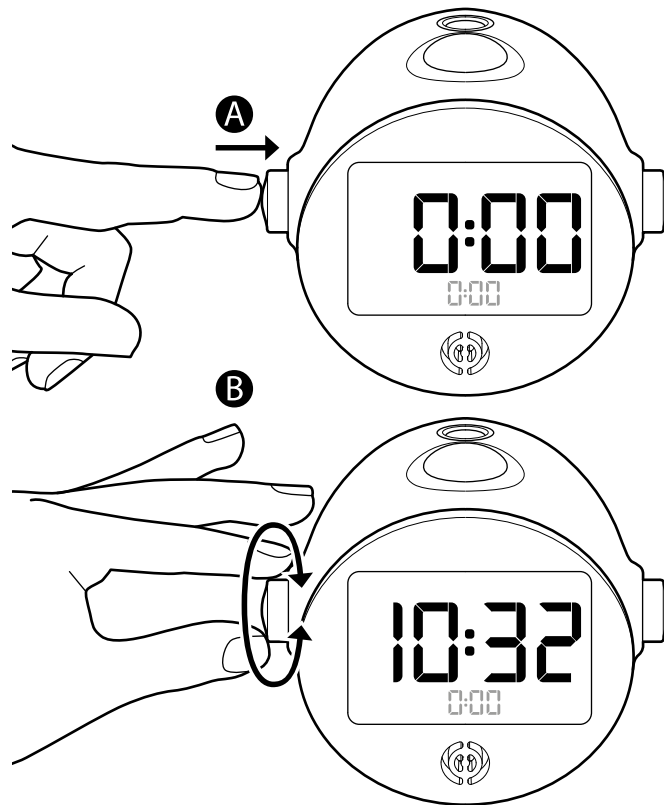
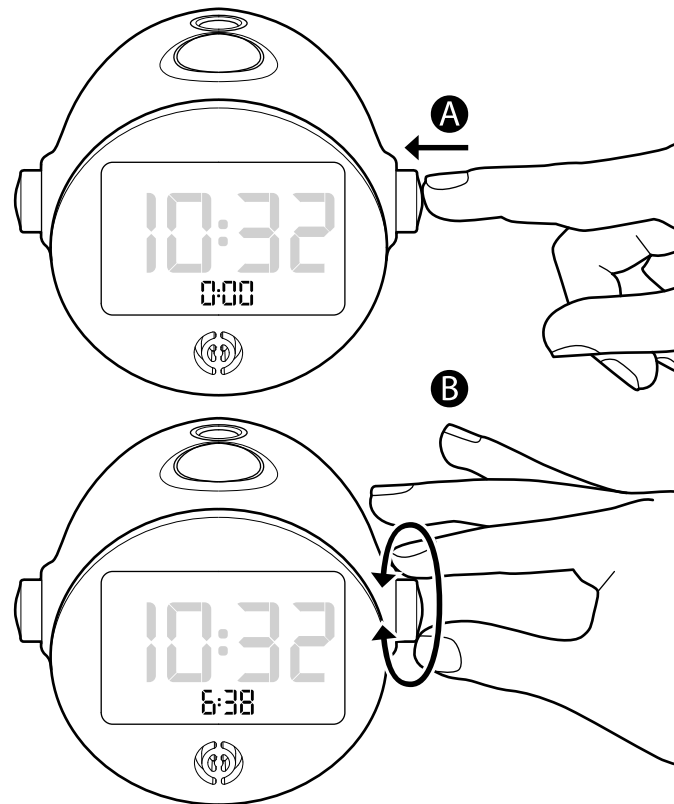
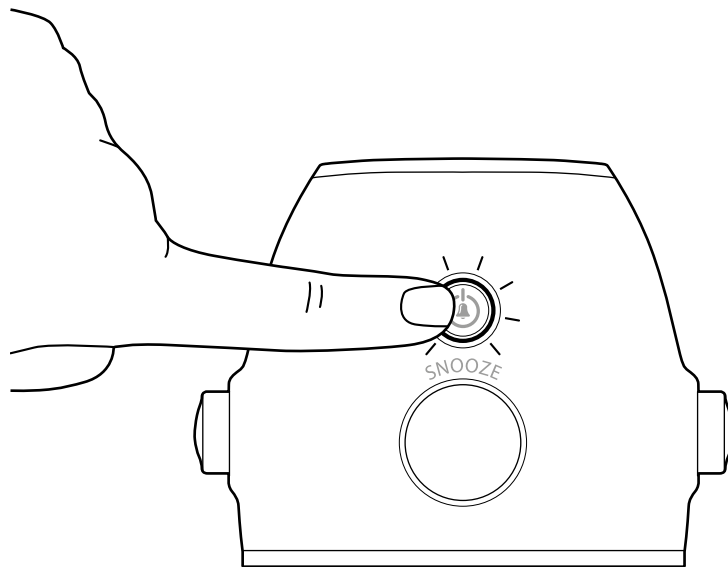
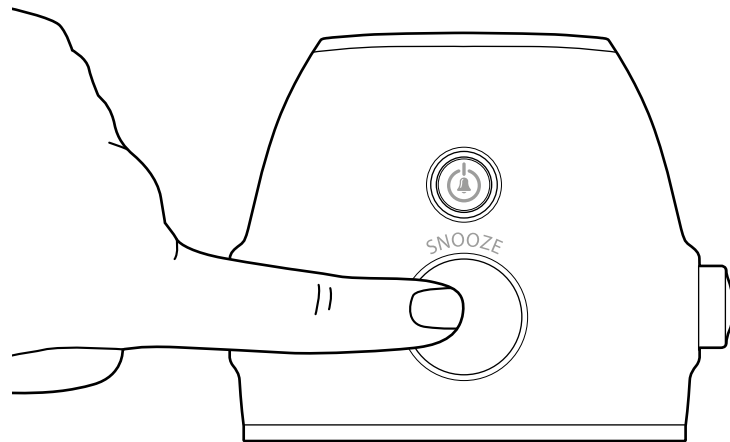
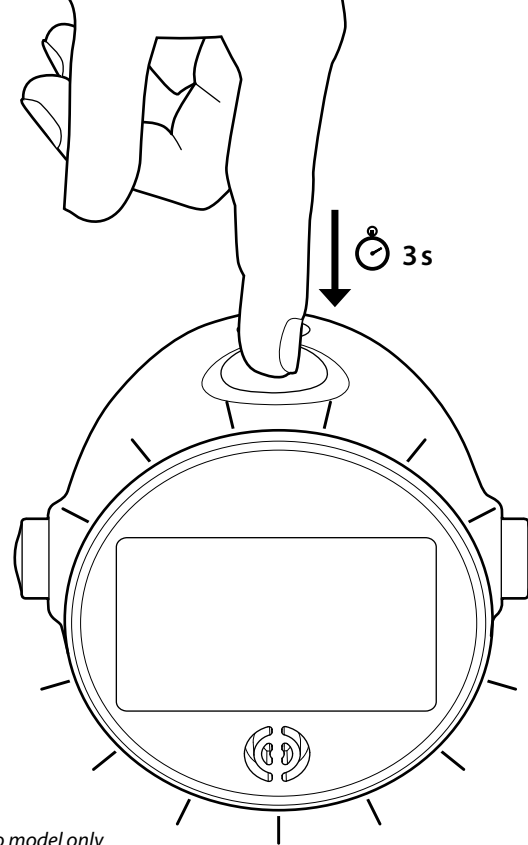
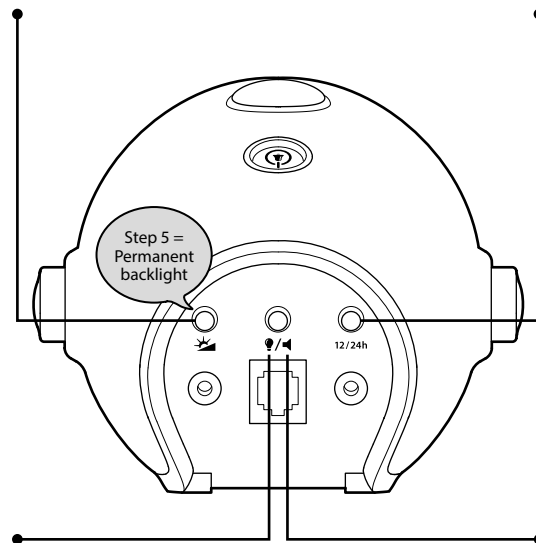
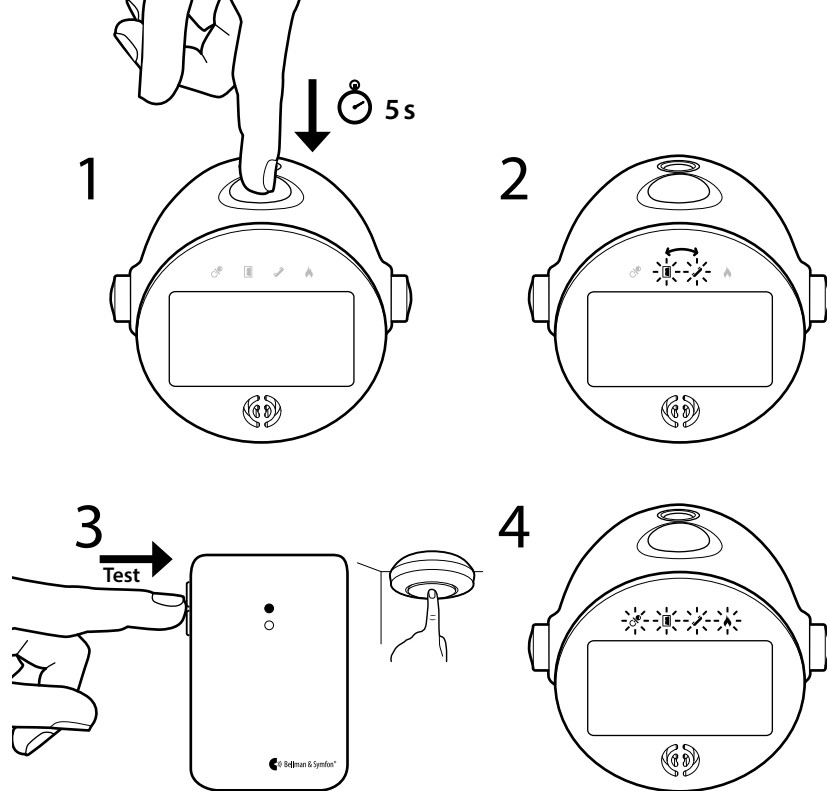
Pairing is only possible on the Visit model.

EN Time	FR Heure	EN Snooze	FR Répétition	EN Alarm	FR Alarme
DE Zeit	NL Tijd	DE Schlummern	NL Snooze	DE Alarm	NL Alarm
DK Tid	NO Tid	DK Slumre	NO Slumre	DK Alarm	NO Alarm
ES Hora	SE Tid	ES Repetición	SE Snooze	ES Alarma	SE Alarm
FI Aika		FI Torkku		FI Herätys	

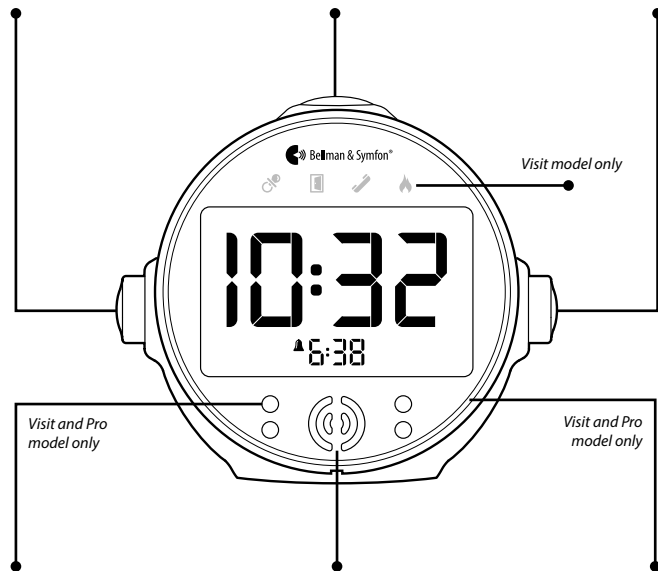EN Flash	FR Clignotant	EN Speaker	FR Haut-parleur	EN Night light	FR Veilleuse
DE LED-Blitz	NL Flits	DE Lautsprecher	NL Luidspreker	DE Nachtlcht	NL Nachtlchtje
DK Blink	NO Lys	DK Højtaler	NO Høytaler	DK Natlys	NO Natllys
ES Destello	SE Blixtljus	ES Altavoz	SE Högaltare	ES Luz nocturna	SE Natlampas
FI Vilku		FI Kaiutin		FI Yövalo	

EN Brightness	FR Luminosité	EN Flash/Sound	FR Clignotant/Son	EN Time format	FR Format
DE Helligkeit	NL Helderheid	DE Blitz/Ton	NL Flits/Geluid	DE Zeitformat	NL Indeling
DK Lysstyrke	NO Lysstyrke	DK Blink/Lyd	NO Lys/Lyd	DK Format	NO Format
ES Brillo	SE Ljusstyrka	ES Destello/Sonido	SE Blixtljus/Ljud	ES Formato	SE Format
FI Kirkkaus		FI Vilku/Ääni		FI Formaatti	

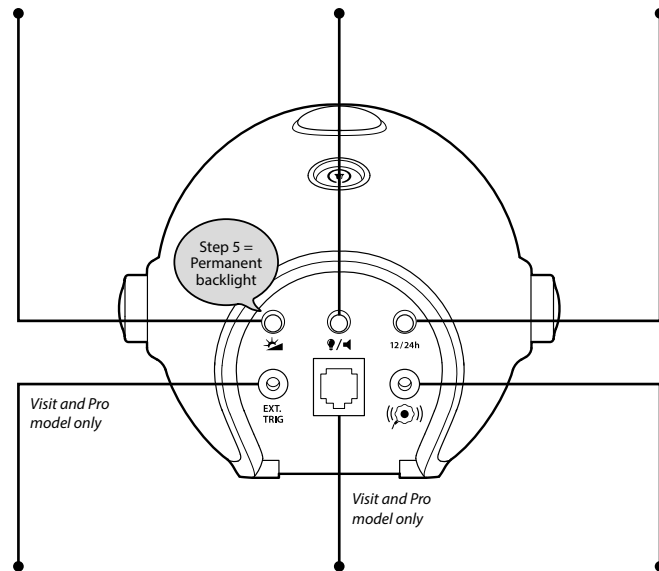EN Ext. trig.	FR Déclench. ext.	EN Telephone	FR Téléphone	EN Bed shaker	FR Vibreux de matelas
DE Trigger	NL Ext. trig.	DE Telefon	NL Telefoon	DE Vibrationskissen	NL Triltschijf
DK Ekst. udl.	NO Ekst. utl.	DK Telefon	NO Telefon	DK Sengen vibrator	NO Sengen vibrator
ES Activ. ext.	SE Ext. trig.	ES Teléfono	SE Telefon	ES Vibrador de cama	SE Sängvibrator
FI Ulk. lauk.		FI Puhelin		FI Tärstin	

Alarm Clock Visit, Pro & Classic

26	Medical device information
34	Informationen zu Medizinprodukten
42	Oplysninger om medicinsk udstyr
50	Información sobre dispositivos médicos
58	Lääketieteellisiä laitteita koskevat tiedot
66	Informations sur les dispositifs médicaux
74	Informatie over medische hulpmiddelen
82	Informasjon om medisinsk utstyr
90	Information om medicinteknisk utrustning

EN

DE

DK

ES

FI

FR

NL

NO

SE

Read this first

This booklet guides you on how to use and maintain your new Alarm Clock. Ensure you read this booklet carefully, including the **Warnings** section. This will help you get the most out of your product. If you have additional questions, contact your hearing care professional.

Intended purpose	The intended purpose of the Visit system is to alert deaf and hard of hearing people of important signals in their home.
Intended user	The intended user is a person with mild to severe hearing loss who needs sound amplification.
Intended user group	People of all ages with mild to severe hearing loss or deafness that need audio, visual or sensory amplification.
Principle of operation	The Visit system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Keep this device out of reach of children under 3 years.
- Flashes can trigger epileptic seizures.

- Be aware that alarms and notifications can be missed if batteries run out.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Use only power adapters and battery types that are specified in this leaflet.
- Protect cables from any potential source of damage.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.
- Do not expose batteries to fire or to direct sunlight.
- Rechargeable batteries should be charged fully before using the device for the first time. Only charge batteries in a temperature between 0°– 35°C, 32°– 95° F.
- Rechargeable batteries should only be changed by Bellman & Symfon or an authorized service center. There is a risk of explosion if the batteries are not replaced correctly.
- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- If you encounter other problems with your device, contact the point of purchase, your

local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.

- This device should not be used on aircrafts unless specifically permitted by flight personnel.
- No company, enterprise or user is allowed to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.

Configuration options

This device can be configured with the following Visit transmitters and accessories:

For BE1580 Alarm Clock Visit

Compatible Visit transmitters

- BE1411 Visit door transmitter
- BE1420 Visit pushbutton transmitter
- BE1431 Visit telephone transmitter
- BE1433 Visit mobile phone transceiver
- BE1481 Visit smoke alarm transmitter
- BE1551 Visit smoke alarm transmitter
- BE1555 Visit CO alarm transmitter
- BE1491 Visit baby monitor

Available accessories

- BE1270 Bed shaker
- BE9105 Telephone cable
- BE9026 Contact mat

For BE1370 Alarm Clock Pro

Available accessories

- BE1270 Bed shaker
- BE9105 Telephone cable
- BE9026 Contact mat

For BE1350 Alarm Clock Classic

Available accessories

- BE1270 Bed shaker

For detailed product information, see the corresponding user manual.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits

stated in this leaflet. If the device gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth.

Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused.

This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Regulatory symbols

With this symbol, Bellman & Symfon confirms that the product meets the Medical Device Regulation EU 2017/745.

This symbol indicates the manufacturer's serial number so that a specific medical device can be identified. It's available on the product and gift box.

This symbol indicates the manufacturer's catalogue number so that the medical device can be identified. It's available on the product and gift box.

This symbol indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.

This symbol indicates that the user should consult the instruction for use.

This symbol indicates that it is important for the user to pay attention to the relevant warning notices in the instruction for use.

This symbol indicates important information for handling and product safety.

Temperature during transport and storage: -10° to 50° C, 14° to 122° F
Temperature during operation: 0° to 35° C, 32° to 95° F

Humidity during transportation and storage: <90%, non-condensing
Humidity during operation: 15% to 90%, non-condensing

Atmospheric pressure during operation, transportation and storage:
700 hpa to 1060 hpa

This device is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in the instruction for use.

With this CE symbol, Bellman & Symfon confirms that the product meets EU standards for health, safety, and environmental protection as well as the Radio Equipment Directive 2014/53/EU.

This symbol indicates that the product shall not be treated as household waste. Please hand over your old or unused product to the applicable collection point for the recycling of electrical and electronic equipment or bring your old product to your hearing care professional for appropriate disposal. By ensuring this product is disposed of correctly, you will help prevent potential negative effects on the environment and human health.

ISO Certification of legal manufacturer

Bellman is certified in accordance with SS-EN ISO 9001 and SS-EN ISO 13485.
SS-EN ISO 9001 Certification Number: CN19/42071
SS-EN ISO 13485 Certification Number: CN19/42070

Certification Body

SGS United Kingdom Ltd
Rossmore Business Park Ellesmere Port Cheshire CH65 3EN UK

Technical specifications

Dimensions	121 × 108 × 92 mm, 4.8 × 4.3 × 3.6"
Weight	Visit/Pro model: 350 g, 12.4 oz. (Including batteries) Classic model: 300 g, 10.6 oz. (Excluding battery)
Mains power	7.5 V DC / 1.5 A
Battery (Not supplied with Classic)	Battery backup: 4 × 1.2 V AAA NiMH rechargeable batteries Operating time: ~ 24 h when fully charged Charging time: ~ 10 h from fully depleted
Connections (Not available on the Classic)	3.5 mm stereo ext. trigger input RJ-11 telephone modular jack, 26 – 120 V RMS, 15 – 100 Hz
Sound	Raises slowly to 100 dBA @ 10 cm, 4" / 94 dBA @ 30 cm, 12" Multiple frequency signal, 950 Hz – 3000 Hz
Flash (Not available on the Classic)	Four high-intensity pulsating LEDs Warning! Flashes can cause epileptic seizures
Bed shaker	Vibrator power: 2.0 – 4.0 VDC
Radio frequency (Visit model only)	868.30 MHz
Coverage (Visit model only)	50 – 250 m depending on the characteristics of the building. Coverage is reduced if walls and large objects block the signal.
Display	LCD display with adjustable backlight intensity in 5 steps
Night light (Not available on the Classic)	Type: Electro Luminescent light Timeout: 1 h on mains power, 15 min. on battery
Alarm and snooze	Alarm timeout: 20 min. Decreasing interval from 9 to 2 min.
In the box, Visit model	<ul style="list-style-type: none">▪ BE1580 Alarm clock Visit▪ BE1270 Bed shaker▪ 4 × 1.2 V AAA NiMH batteries▪ 7.5 V DC / 1.5 A power adapter

In the box, Pro model	<ul style="list-style-type: none">▪ BE1370 Alarm clock, Pro model▪ BE1270 Bed shaker▪ 4 × 1.2 V AAA NiMH batteries▪ 7.5 V DC / 1.5 A power adapter
In the box, Classic model	<ul style="list-style-type: none">▪ BE1350 Alarm clock, Classic model▪ BE1270 Bed shaker▪ 7.5 V DC / 1.5 A power adapter

Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation EU 2017/745 as well as the directives and regulations listed below. The full text of the declaration of conformity can be obtained from Bellman & Symfon or your local Bellman & Symfon representative. Visit **bellman.com** for contact information.

Radio Equipment Directive (RED) (Visit model only)

Medical Device Regulation (MDR)

Electromagnetic Compatibility Directive (EMC)(Not available on the Visit model)

Restriction of Hazardous Substances Directive (RoHS)

REACH Regulation

LVD Directive (Not available on the Visit model)

Waste Electrical & Electronic Equipment (WEEE)

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All rights reserved.
Bellman & Symfon Group AB
TM and © 2023 Bellman & Symfon AB.