



EasyLoading

User manual

Introduction

This manual will provide you with simple step-by-step installation instructions for Easy Loading. The manual is divided into four stages. You will need an Allen key, a screwdriver and a wrench at hand to complete all stages.


Easy Loading

Follow the instructions below for assembly instructions for Easy Loading. In order to complete the stage you need a screwdriver and Allen key at hand.


You need the following parts:

- 2 pcs Handle Aluminum
- 1 pcs Handle Easy Loading
- 1 pcs Easy Loading lifting plate
- 8 pcs Screws

Screw the main handle in Easy Loading lifting plate, 4 screws. See figure 1-2.

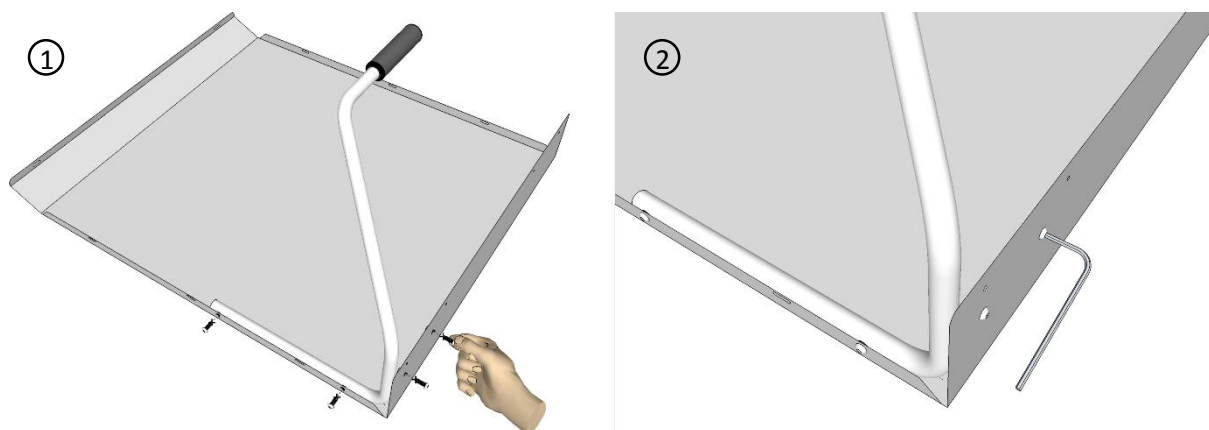

Screw the two smaller handles in Easy Loading lifting plate, 4 screws. See Figure 3-4.

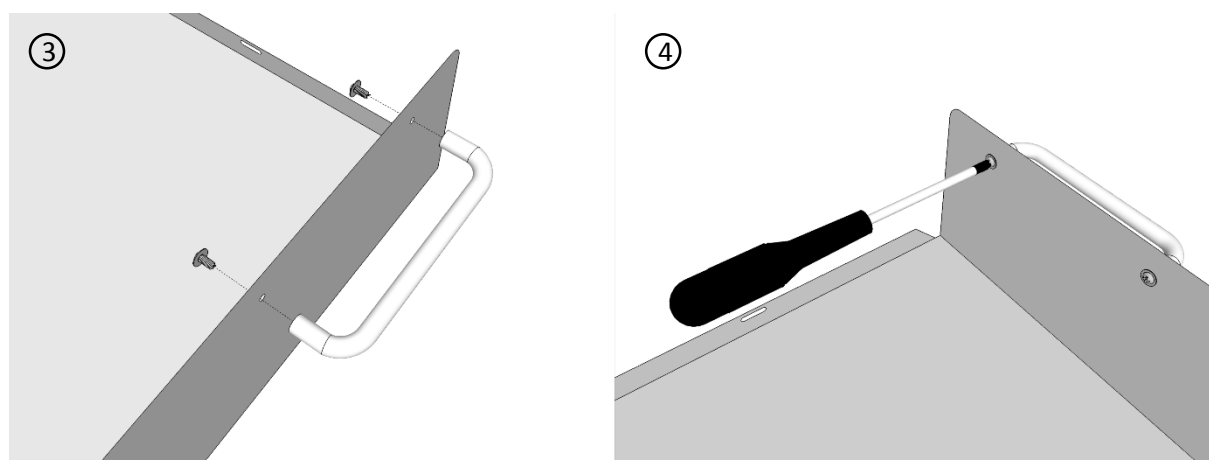



Easy Loading – Guard Plate

The guard plate for Easy Loading is used to protect the interior of your car's trunk. The elevation legs for the guard plate is for you who have a submerged trunk. Follow the instructions below to assemble the guard plate. In order to complete the stage you need an Allen key at hand.

You need the following parts:

- 1 pcs Guard plate
- 4 pcs Elevation legs
- 16 pcs Screws

Fasten each elevation leg with four screws in the prepared holes on the underside of the guard plate. See figure 5-7

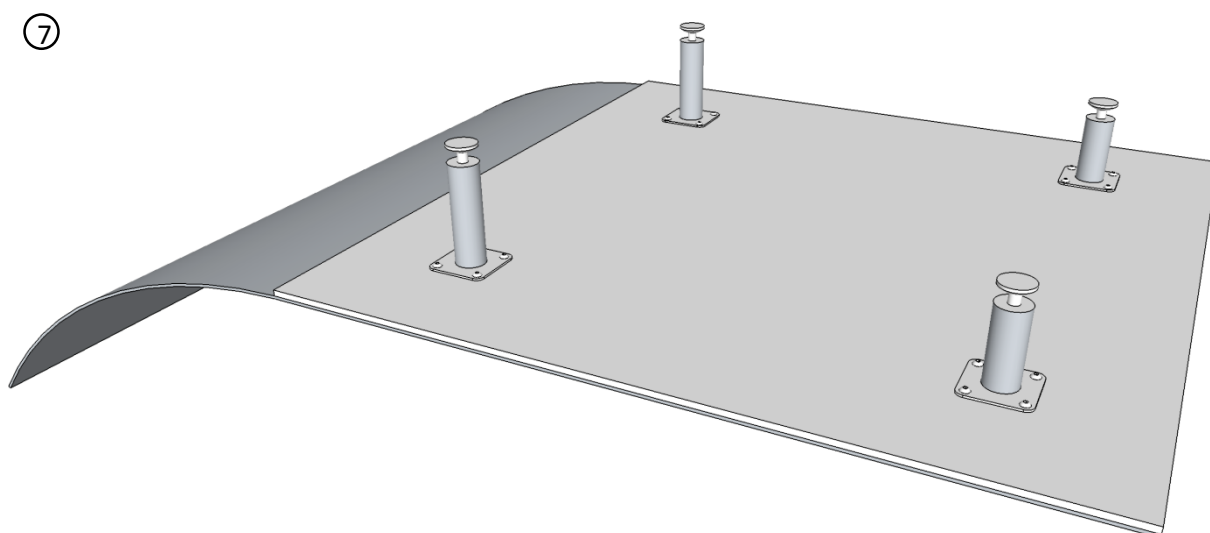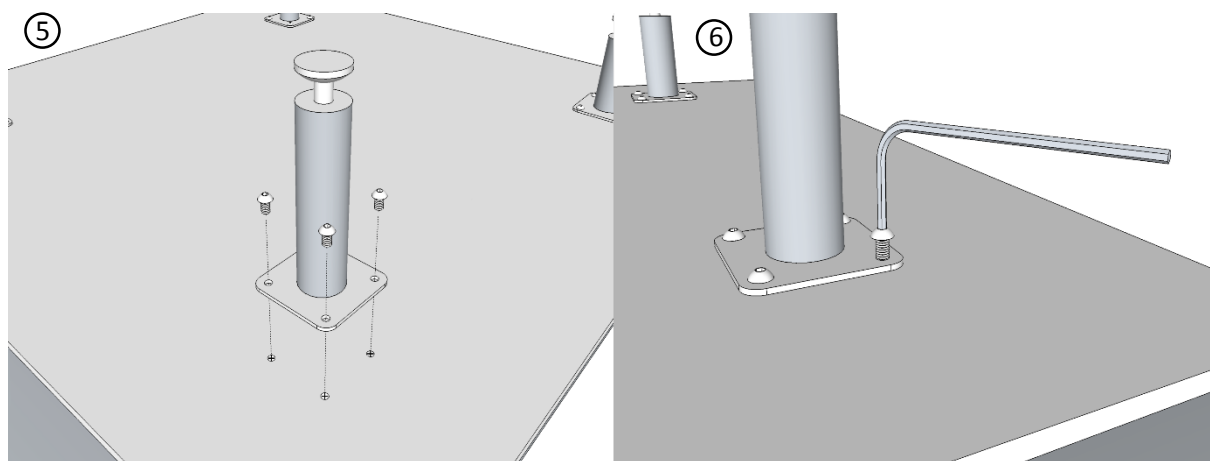



Easy Loading – installation of capstans in car

Follow the instructions below for installation instructions for capstans in car. In order to complete the stage you need a wrench at hand.

You need the following parts:


- 2pcs Capstans
- 2pcs Wire rope
- 2pcs Wire lock

In the car trunk, locate a bracket in the form of a loop or hook that can sustain a load of 5kgs for each of the capstans. The capstans can also be fastened around the headrests.

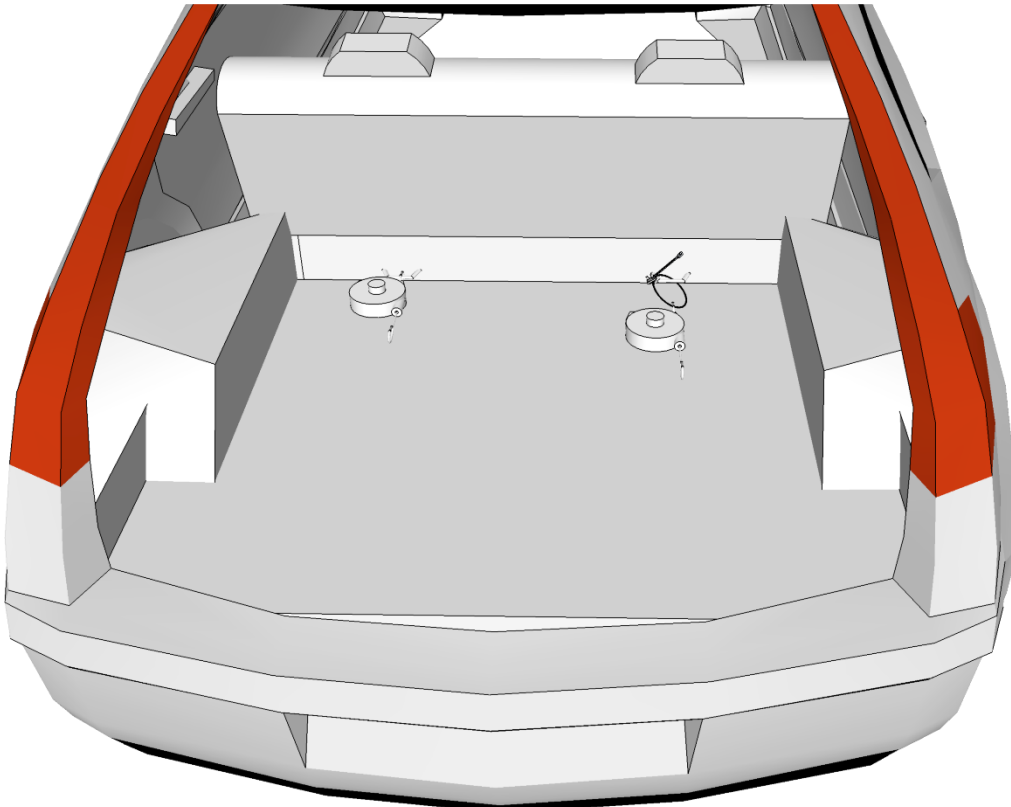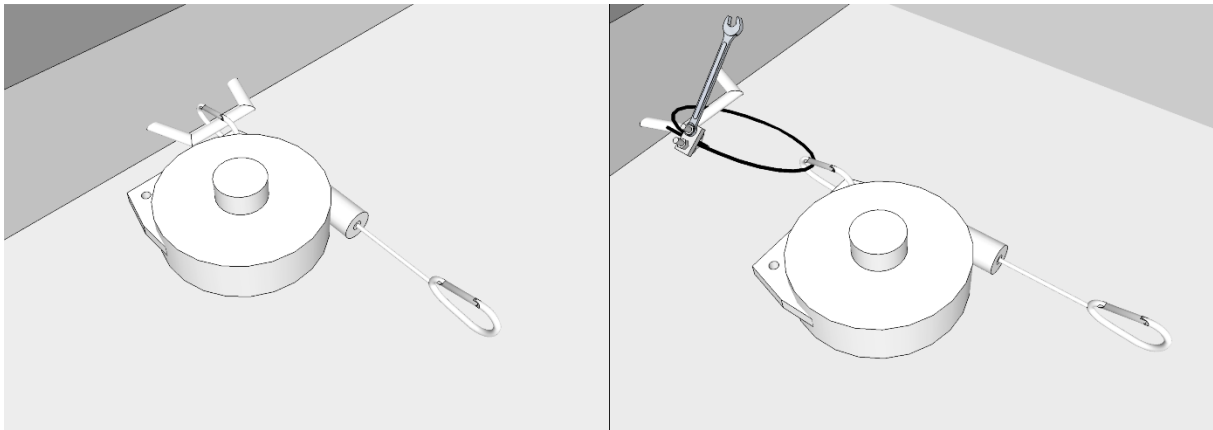


Easy Loading – installation in car

Place the guard plate in the trunk.

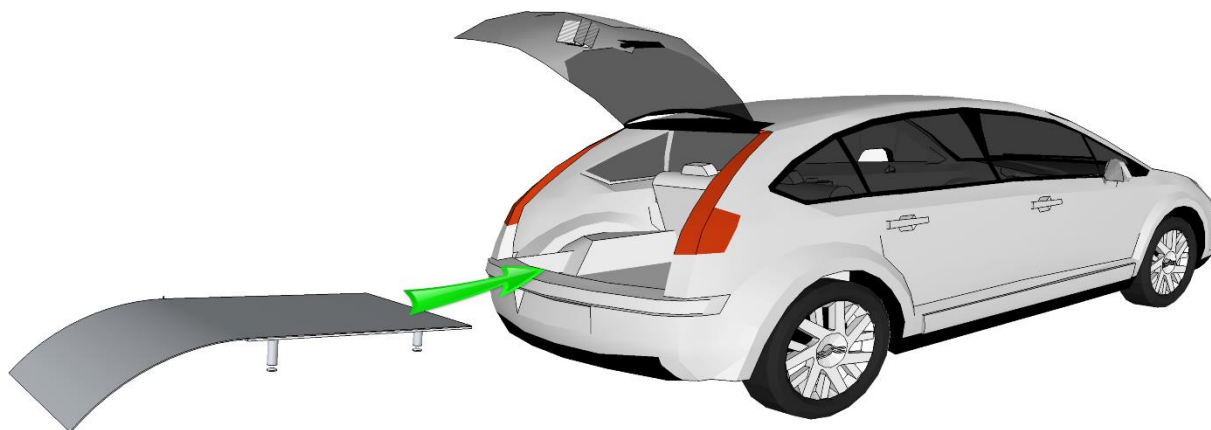

Place Easy Loading on the guard plate and attach the capstans.

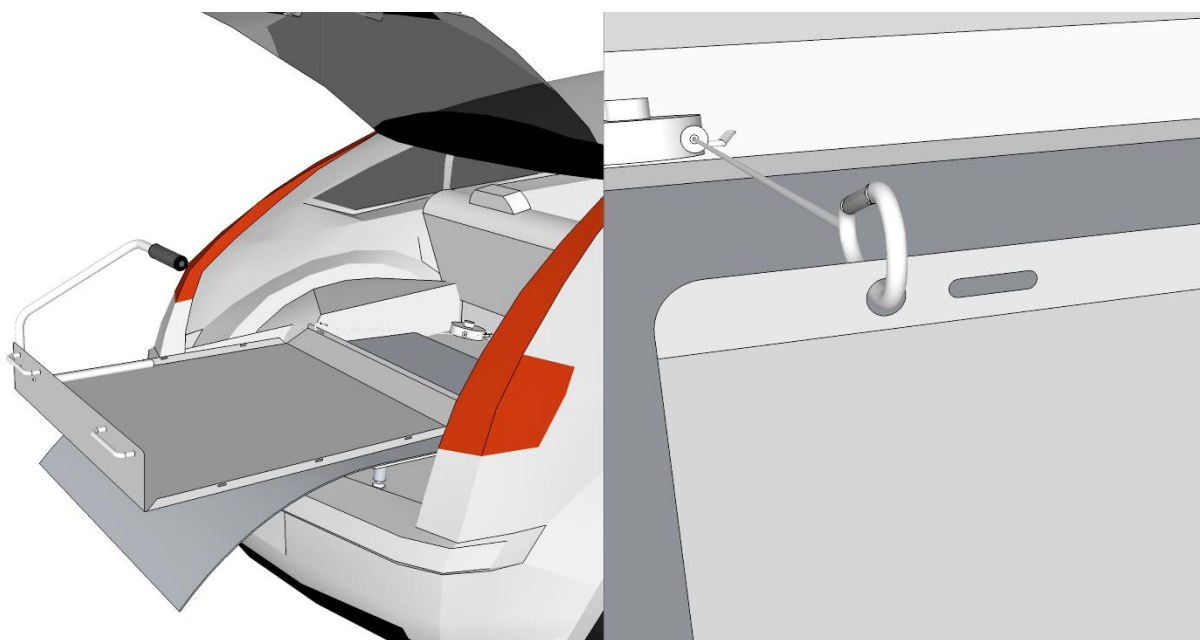

Easy Loading is now ready to be used.



Kalkvägen 3
SE-269 34, Båstad
Sweden

www.novista.se