- Ⓜ **Swivel seat cushion**
- Ⓝ **Dreh-Sitzkissen**
- Ⓚ **Drejesæde**
- Ⓦ **Cuscino girevole**

Item no. **278051, 278052**
Art.Nr. **278051, 278052**
Varenr. **278055, 278056**
Cod.

User manual

MOBILEX A/S
Grønlandsvej 5
DK-8660 Skanderborg
www.mobilex.dk

GB Swivel cushion

Purpose

The product is an aid designed to help people with movement difficulties to get in or out of the car, bed or chair. Use the product only for the maximum allowable weight as indicated on the label. If you have further questions, please contact your dealer or Mobilex A/S directly.

Safety instructions

1. Use the product only in perfect condition and only for the intended purpose.
2. If the product shows damage or defects, please contact your dealer immediately.
3. Read the information on the label.
4. Do not leave the product unattended on the floor - someone could step on it, slip and get seriously injured themselves!
5. Do not make any changes to the product except with the written permission of the manufacturer
6. Do not stand on the product.

Product description

The swivel cushion fits most cars or surfaces and allows the user to get in and out of the car more easily or f.ex. to stand up from a chair. The product can help people with back or knee problems to avoid unnecessary rotation of the back. The product is made of high quality polyester and memory PU foam, as well as a low-friction material that provides good comfort and durability through a silicone anti-slip technology.

Use of the product

1. Place the cushion in the middle of the car seat (or other surface) with the dark, non-slip surface facing down
2. Stand with your back to the cushion.
3. Bend your knees slightly by moving slowly backwards and sit on the cushion.
4. Slowly turn your upper body towards the inside of the car and grasp the steering wheel with your hands.
5. Lift your legs together.
6. Hold the steering wheel firmly and turn your legs and body into the car.
7. Position yourself in the driver's seat and adjust the distance to the steering wheel.
8. Get out of the car: Hold the steering wheel, lift your legs and slowly turn your body out of the car using the cushion. Put your feet on the ground and leave the car.

Maintenance and care

Use a soft cloth only with standard, neutral cleaning agents. Do not use solvents or chlorine!

Recommended service life: Normally 3 years depending on use.

Warning

 **Please note that if the product is bent or broken, the product must not be recycled but must be replaced.**

Reuse

The product is suitable for recycling. Before recycling, where a new user is to take over the product, the product must be checked according to above mentioned advice.

Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

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Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,