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 Mobilex A/S, Nørskovvej 1  
DK-8660 Skanderborg, Danmark

**GB Mobilex A/S****Triangular Shower Stool 302067**  
User's manual

The triangular shower stool is used to give stable and easy seating support, when disabled persons use the shower. It has an aluminum frame with height-adjustable legs and a seat made of white plastic.

**Safety Tips**

Only use the device for the purpose it is intended for. Don't climb on the device. Make sure that the 3 feet are set to the same height. Use device only in good technical condition. Do not overload the device. Use device only on a flat smooth surface. Always check the stability of the device before use.

**Assembly and use**

The device must be assembled by mounting the three legs to the downside of the seat. Adjust the seat height of the chair to the size of the user by pressing the pin-buttons on the stool legs and adjust the legs to the desired height. Make sure that all legs are set to the same height, and that the buttons on all legs are safely locked.

**Maintenance**

Normal dirt on metal and plastic parts can be removed with standard cleaning agents and a soft cloth. Do not use solvents or abrasives!

**Technical data:**

Dimension: 50 x 45cm (W x D)  
Seat height: 47-65 cm  
Max user weight: 150 kg  
Product weight: 2,50 kg

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**I Mobilex A/S****Sgabello triangolare da doccia**