LEJRELET

TUBE 250

MANUAL

DANSK

Anvendelse

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ENGLISH

Application


LEJRELET Tube 250 is a long soft cylinder filled with pressure relieving viscoelastic foam granules.

It is easy to shape and adapt Tube 250 for the individual user and care situation and thus possible to adapt Tube 250 to various situations and conditions.


The viscoelastic foam granules give great adaptability to the body and good support / pressure prevention. Further more, Tube 250 is very comfortable and calming for the bedridden person.

Examples for use of LEJRELET Tube 250:


- Support of the entire body in lateral position
- Positioning in a 30 degree angle
- Pressure relief for scapula and arms in supine position
- Support of legs and pressure relief of heels
- Comfort and calming positioning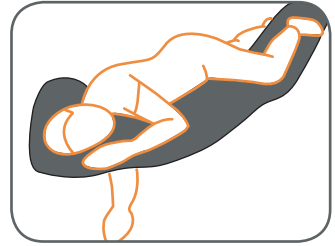


Support of the entire body in lateral position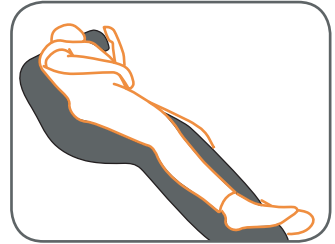

Positioning in a 30 degree angle

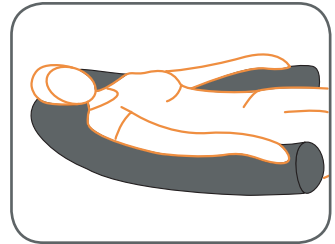Pressure relief for scapula and arms

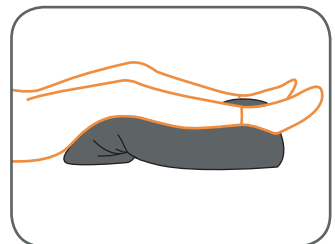Support of legs and pressure relief of heels

ENGLISH

Technical information

Filling:

LEJRELET Tube 250 is filled with viscoelastic foam granules preventing decubitus.

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ENGLISH

Application


The LEJRELET Tube 125 has the same structure and positive qualities as the LEJRELET Tube 250. The great flexibility of the material affords many application options.

It is, however, only half the length, i.e. 125 cm long and therefore weighs less.


The LEJRELET Tube 125 is suitable for smaller adults and large children. It is also suitable for leg amputees who need good support for the shortened leg. The LEJRELET Tube 125 can be folded very tightly, thus providing great support for, inter alia, contractures.

Examples for use of LEJRELET Tube 125:


- To support the body in a lateral position
- For 30-degree positioning for leg amputees
- To support legs and arms while seated
- Stable lateral positioning for flexion contractures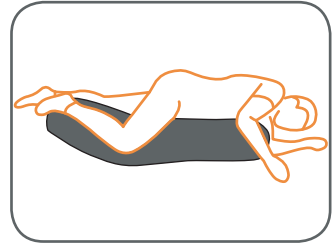


To support the body in a lateral position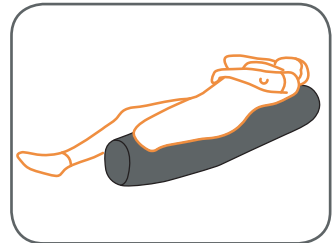

For 30-degree positioning for leg amputees

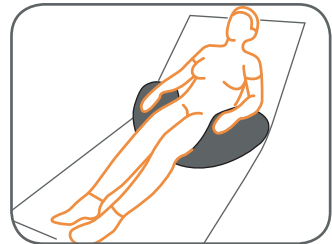To support legs and arms while seated

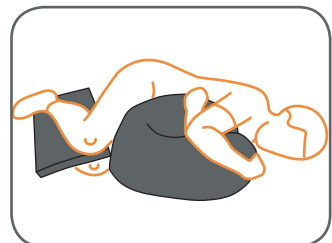Stable lateral positioning for flexion contractures

ENGLISH

Technical information

Filling

LEJRELET Tube 125 is filled with viscoelastic foam granules preventing decubitus.

Size: Ø 25 x L 125

Item no.: 5000000

Weight: 2350 g

Fabric

- Waterproof
- Watervapour permeable
- Two way stretch
- Anti-bacteria treated
- Tight to house dust mite
- Biocompatible
- Oeko-TEX, class 1
- Fire-retardant to crib 5

Be aware that the patient is positioned such that airways are kept clear.

Washing and cleaning

Can be wiped with mild soap and disinfectant fluids or showered with boiling water.

Hand wash

Do not tumble dry

Do not iron

Do not dry clean

Do not bleach

Other symbols

Warning

Item no.

Year and month of manufacture

Name of manufacturer

CE marking:
The product complies with the
Medical Device Regulation
(EU) 2017/745

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Copyright 2019 Vendlet ApS
Vendlet B.V.