
system vario plus




Bedienungsanleitung
Operating instructions
Mode d'emploi
Istruzioni d'uso
Instrucciones de uso
Gebruiksaanwijzing
Brugsanvisning
Bruksanvisning
Käyttöohje
Návod k použití
Instrukcja obsługi
サービスマニュアル



 **ESCHENBACH**
Germany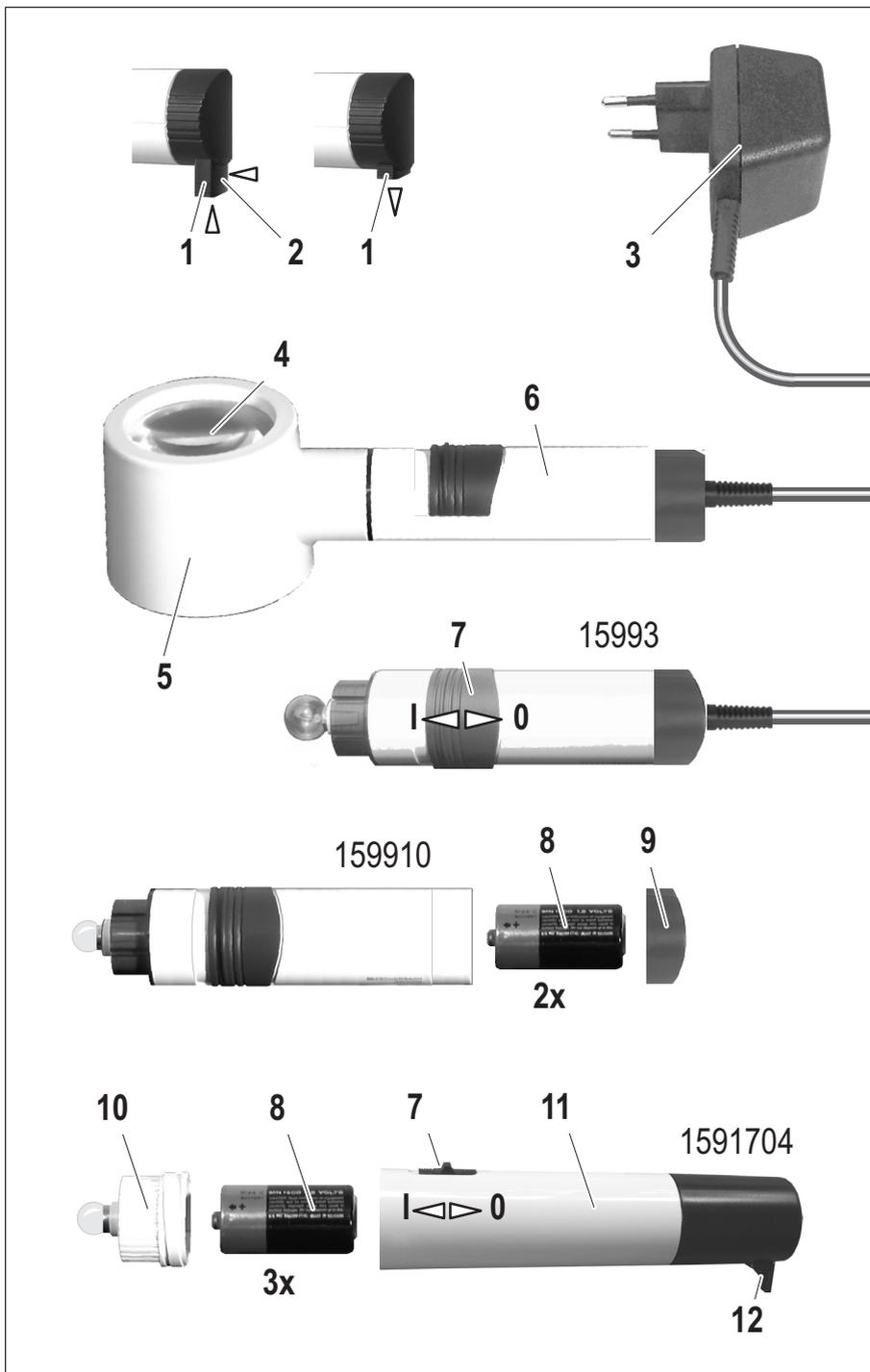

LED

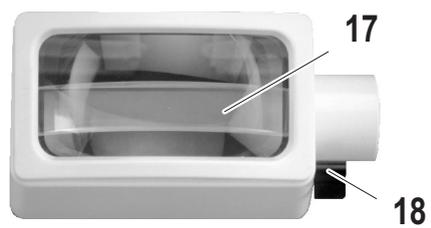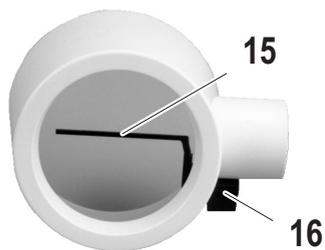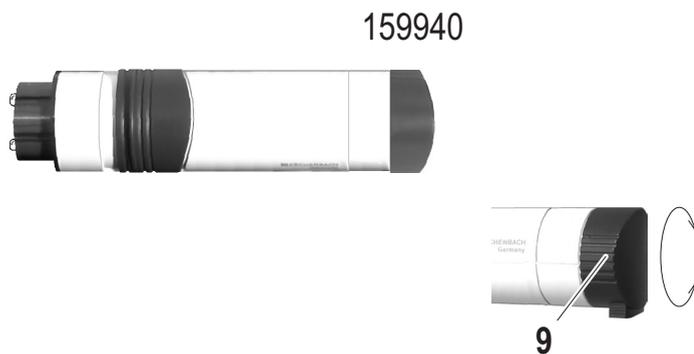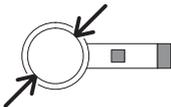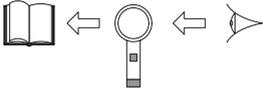				
Ø 35 mm	50,0 Dpt	12,5 x	15 mm	40 mm
Ø 35 mm	38,0 Dpt	10,0 x	20 mm	40 mm
Ø 35 mm	28,0 Dpt	7,0 x	30 mm	60 mm
Ø 50 mm	23,0 Dpt	6,0 x	31 mm	100 mm
Ø 58 mm	20,0 Dpt	5,0 x	36 mm	140 mm
Ø 70 mm	16,0 Dpt	4,0 x	45 mm	150 mm
Ø 80 mm	12,0 Dpt	3,0 x	49 mm	250 mm
100 x 50 mm	7,6 Dpt	3,0 x	59 mm	250 mm
100 x 75 mm	7,0 Dpt	2,8 x	61 mm	250 mm

Deutsch	5
English	11
Français	16
Italiano	23
Español	29
Nederlands	35
Dansk	41
Svenska	46
Norsk	52
Suomi	57
Polski	62
Česki	69
日本語	74

Deutsch

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English

You have selected a quality product made by Eschenbach, which has been produced using the latest manufacturing processes in Germany; a top-class "Made in Germany" product. Congratulations on your choice.

Various magnification ranges and light sources are available for the System Vario Plus illuminated magnifier. The illuminated magnifier that you purchased was individually assembled according to your eyesight. It may therefore look different from the model illustrated.

If your eyesight changes, the illumination and the magnification can be adapted by exchanging the magnifier's handle and the lens barrel.

The lens of your illuminated magnifier is made of high-quality, unbreakable PXM[®] material that is protected against scratches by a hardened coating made of cera-tec[®] – scratch resistance is an important criterion for quality and service life.

Safety instructions

- **Risk of fire!**
Lenses used in optical equipment may cause considerable damage due to their "burning glass effect" if used or stored improperly. Make sure that optical lenses are never left uncovered in the sunlight.
- **Danger of blinding or injury!**
Never look at the sun through any optical equipment.
- **Protect your illuminated magnifier from knocks, bangs, moisture and excessive heat. Never place your illuminated magnifier on heaters or in the sunlight.**

Mains powered models only:

- **Danger of tripping! Ensure that no-one can trip over the connecting cable.**
- **Do not wind the connecting cable around the illuminated magnifier.**
- **Do not use the illuminated magnifier near any containers that hold water (e.g. bathtubs).**

Battery-operated models only:

- **Batteries must not be disposed of in household waste. Ensure that they are correctly disposed of.**
- **Ensure that other people, especially children, are aware of these dangers!**

Proper use

Mains powered models only (handles 15993, 159960):

- ▶ **Insert the electric plug [3] into a mains socket.**
-  It is normal for the plug and the lamp to heat up slightly. The illuminated magnifier can be left plugged into the mains after being switched off.

Models with LED mains handle 159960 only:

- ▶ **Adjust the brightness controller [13] until the brightness level is comfortable.**

Not for handle 1591704:

- ▶ If the magnifier is not level, **pull the support [1] out of the handle until it clicks into place.**
- ▶ If the support gets in the way, **press on the snap-in section [2] and push the support into the handle.**

Handle 1591704 only:

If the illuminated magnifier is not level:

▶ **Push the insertable support [12] (included in the scope of supply) into the slot at the rear cap.**

The distance between the text being read and the lens is optimal when the illuminated magnifier is lying on the text.

▶ **Switch on the illuminated magnifier by pushing the switch [7] towards the lens [4].**

▶ **Place the illuminated magnifier on the text being read.**

▶ **Move the illuminated magnifier by its handle [6/11] over the text being read.**

▶ **After use, switch off the illuminated magnifier by pushing the switch [7] back towards the end.**

Lens barrels with useful additional functions

Swivel-in reading line [15] useful for orientation

The following lens barrels include this function:

Ø 80, 12 dioptr, 3.0 x	Ø 70, 16 dioptr, 4.0 x	Ø 58, 20 dioptr, 5.0 x
---------------------------	---------------------------	---------------------------

▶ **Turn the lever [16] to swivel the orientation aid into position.**

Swivel-in magnifying accessory [17] with 4 dioptr.

To read very small print, or to magnify difficult sections of text.

The following lens barrels include this function:

100 x 50, 7.6 dioptr, 3.0 x	100 x 75, 7.0 dioptr, 2.8 x
-----------------------------	-----------------------------

▶ **Turn the lever [18] to swivel the magnifying accessory into position.**

Changing the batteries

i Always replace all batteries. Ensure that they are inserted correctly (positive end facing the lamp).

▶ **Switch off the illuminated magnifier by pushing the switch [7] back towards the end.**

Handles 159910 and 159940 only:

- ▶ **Turn the grey end cap [9] counter-clockwise and remove it.**
- ▶ **Replace the old batteries with new ones [8].**
- ▶ **Screw the grey end cap back into the handle.**

Handle 1591704 only:

- ▶ **Pull the handle [11] out of the lens barrel [5].**
- ▶ **Unscrew the front cap [10].**
- ▶ **Replace the old batteries with new ones [8].**
- ▶ **Screw the front cap back into the handle.**
- ▶ **Insert the handle straight into the lens barrel.**
Make sure that the slot in the rear cap is facing down.

Lamp bulb replacement

Models with bulbs only (handles 15993, 159910, 1591704):

- ▶ **Switch off the illuminated magnifier by pushing the switch [7] back towards the end.**
- ▶ Handle 159910 only:
Remove the plug from the [3] mains supply.
- ▶ **Pull the handle [6/11] out of the lens barrel [5].**
- ▶ **Change the defective bulb for a new one.** Only use bulbs specified in the technical data.

► **Insert the handle straight into the lens barrel.**

Handles 15993 and 159910 only:

Ensure that the support [1] is pointing downwards.

Handle 1591704 only:

Make sure that the slot in the rear cap is facing down.

Care instructions

When the illuminated magnifier is not being used, it should be kept covered.

Clean the housing with a damp cloth.

Clean the lenses [4] with a soft, lint-free cloth (e.g. spectacle cleaning cloth).

For harder to remove dirt (e.g. fingerprints):

Remove the lens barrel [5] from the handle [6/11] and clean the lenses with lukewarm water. After the lenses are completely dry, assemble the front section and handle again.

Do not use any soap solutions that contain softeners, alcohol-based solvents, or abrasive cleansers. It could damage the lenses.

Technical data

Mains powered models	Bulb	LED
Current supply	230 V, 50 Hz (US version 120 V, 60 Hz)	
Operating current	4.5 V	15 V
Bulb (Order No. 1546)	4 V / 0.55 A / E10	---
Replacement bulbs are available in optical equipment shops.		
Ambient temperature	-10°C ... +40°C	

Battery powered models	
Handle 159910: 2 alkali-manganese batteries/rechargeable batteries Bulb (Order No.-Nr. 1545)	C-size (generally available) 2.5 V / 0.25 A / E10
Handle 1591704: 3 alkali-manganese batteries / rechargeable batteries Bulb (Order No.-Nr. 1546)	C-size (generally available) 4 V / 0.55 A / E10
Handle 159940: 2 alkali-manganese batteries/rechargeable batteries	C-size (generally available)
Ambient temperature	-10 °C ... +40 °C

Replacement bulbs are available in optical equipment shops.

For any problems concerning the warranty or breakdowns, please consult your specialist shop. They will be pleased to help you.

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Guarantee

We are required by law to guarantee the function of the product described in these descriptions in respect of defects caused by manufacturing or material faults. Any damages caused by incorrect handling, including damage caused by being dropped or knocked, are not covered under the guarantee. Guarantee benefits can only be claimed through proof of purchase in the form of a sales receipt!

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