

buckingham pocketeasywipe

Personal hygiene aid

Designed by Occupational Therapist Chris Buckingham



Made in UK

The pocket easywipe has a soft, rounded end (A) to which toilet tissue or moist wipes are attached and a patented cut out (B) to tuck the tissue into to secure whilst using it.

It is simple to use and can be carried discreetly in the travel pouch provided. It is light, sturdy and easy to keep clean.

Simply unfold the pocket easywipe and follow instructions for use below.

Instructions for use

Select the length of toilet tissue which should be approximately 4 sheets long. Open the pocket easywipe to its full length. Take the toilet tissue and wrap it around the soft end (A) and tuck it into the patented cut out (B). The toilet tissue will be held in place for cleaning.

The pocket easywipe may be used to clean the vaginal and anal area following toileting. It is recommended to be used between the legs, from the front (2) or from behind, whilst seated on the toilet.

After cleaning the soiled material is ejected by gently shaking the pocket easywipe over the toilet pan and disposed of the tissue into the toilet. Repeat the above wiping process as required.

Use a moist wipe or a disposable cloth with a mild soapy water to clean the soft end of the pocket easywipe.

Specifications:

The easywipe is ergonomically designed to be comfortable, safe to use and to assist reaching when wiping is difficult. It is latex free.

Please read this:

FOR EXTERNAL USE ONLY.

DO NOT USE THIS PRODUCT IF THERE IS ANY SIGN OF DAMAGE TO THE END MARKED (A) OPPOSITE. YOU MUST DISCARD THE PRODUCT.

Only to be used for the purposes of external, personal cleansing and wiping.

To maintain personal hygiene the easywipe should not be shared between people.

Not to be used on animals or young children.

Only use the easywipe for the intended purpose.

Any excessive force applied to the hinges could result in damage to the product.

If the hinges are damaged then you MUST DISCARD THE PRODUCT

DO NOT use the easywipe if any damage to any part of the product is noticed.

Cleaning

You may immerse the pocket easywipe in water. Dry it thoroughly afterwards.

When cleaning wipe all over with a soft disposable cloth in a mild soapy water.

Dry with a soft disposable cloth.

It is recommended that the pocket easywipe is cleaned after each use.

Due to hygiene concerns the product, once removed from the packaging cannot be returned to us.



Neat and discreet



Instruction video available at
www.pocketeasywipe.com



Registered Office Address - Tower House, Parkstone Road, Poole, Dorset, BH15 2JH
www.buckinghamhealthcare.co.uk



buckingham
healthcare



Helping people to help themselves

Chris Buckingham
LLB BSc SROT