Technische Daten / Technical specifications:

Duschstuhl in Holzoptik / Shower Stool Wood Look

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Intended Use:

The shower chair is a medical device and designed as a sitting and stand-up aid for people who have a limited stability/function of the lower extremities. It can facilitate or enable body care in the shower. The shower chair must only be used by people who can independently control their balance while sitting and should always be used with the support of a caregiver.

Indication:

Limited standing ability, limited stability, general weakness

Contraindication:

Dizziness, impaired balance function, lack of trunk control, states of confusion, insufficient physical or mental abilities to use

Description:

The shower chair is used for a comfortable and safer performance of body care while sitting in the shower. It is made of aluminium and plastic and can be adjusted in five different seat heights. The extra-wide seat combines functionality with modern design. The wood look conveys a comfortable ambience and also adapts visually to modern bathrooms. The selection of the shower chair suitable for the patient should be done together with the specialist staff. Testing, assembly and instruction in use must only be carried out by authorized, trained specialist personnel. The product should only be used with the help of an accompanying person/caregiver. The model is designed for domestic use, must only come into contact with intact, uninjured skin and must not be used across patients. Any use other than that mentioned above is not intended. Sundo GmbH or the manufacturer is not liable for any resulting damages, the risk is borne exclusively by the user. Any serious incidents occurring in relation to this device shall be reported without delay to the manufacturer or to the competent authority.

Prior to use / Maintenance:

First, read the present instruction manual in full and keep it with the packaging until the end of the product life. Please check whether the product is undamaged before first and each use. Also check the safe position of the shower chair before each use, regularly check retaining clips/push buttons and stability. If there are signs of damage or wear, the shower chair must not be used (anymore). Further maintenance is not required in compliance with the safety instructions.

Before using it for the first time, please check for completeness and damage. Defective products may not be used. Please read the instructions for use in full and keep them until the end of the product's life. Note: The minimum footprint is approx. 68 x 54.5 cm.

Assembly (tool-free):

Insert the seat into the designated mounting tubes of the first side panel. Press down the brass-colored buttons and slide the seat until both buttons emerge audibly, visibly and noticeably from the bore. Do the same with the second side panel. According to the same principle, the back tubes can now be attached to the chair. First, choose whether the armrests at the front should be low or high, both variations are possible. Then attach the back tubes to the seat with the click system (grey brackets optionally at front or rear), then the backrest to the back tubes. Three height settings are possible here. Finally, the feet are also inserted into the legs via the click system and attached with the enclosed retaining clips.

Make sure that all four legs are always set to the same height and that all retaining clips and snaps are securely engaged!

Use:

Before use, the caregiver must check the safe status of the Shower Stool. If the showerchair tilts, change its stand so that it stands securely. To sit down and get up, always seek the help of the caregiver. Always sit upright and in the middle of the chair, always use and burden **both armrests evenly**, otherwise there is a risk of tipping and falling!

Cleaning and disinfection:

The shower chair should be rinsed with clear water after each use. If necessary, it can be cleaned with commercially available household cleaners, please make sure that they do not contain oils, greases or abrasives. The aid can be disinfected in the wipe disinfection process with a commercially available, chlorine-free disinfectant. **Important:** When using disinfectants, the application instructions of the manufacturers must be observed.

**Safety Instructions:**

- Maximum body weight of the user: 136 kg!
- Caution, there is a risk of pinching when gripping into the gap between the frame and seat and when assembling
- Do not use the shower chair for persons who cannot control their balance while sitting
- For use of the shower chair when sitting down and getting up, an assistant is necessary!
- If parts of the shower chair are bent, deformed or damaged, it must no longer be used!
- All four feet must be adjusted to the same height and the shower chair must stand securely on them
- Repairs must only be carried out by specialist dealers using original spare parts

Warranty:

According to the statutory warranty claims. Excluded from the warranty: Damage caused by improper use, normal wear and tear and consumption as well as defects that only insignificantly affect the value or usability of the device. Technical changes and additions as well as improper maintenance work lead to the expiry of the warranty and product liability in general. We reserve the right to make changes that serve the purpose of technical improvement.

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Sundo Homecare GmbH,
Homecare Enterprise Co., LTD. No.488, Lunmei Rd., Changhua City, Changhua County 500, Taiwan (R.O.C.)