To watch video instructions, please visit our website: [www.sigvaris.com/videos](http://www.sigvaris.com/videos)

## EN: Composition:

**DND:** Thermoplastic elastomer  
Contents: 92.5% water, 7.5% polyacrylamide, bactericide  
**CONE:** Pole: Polyethylene; Suction Base: Homopolymer, Zinc Alloy  
**Caution:** This product contains natural rubber latex which may cause allergic reactions

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SIGVARIS, INC., 1119 Highway 74 South, Peachtree City, GA 30269 USA  
Phone 1.800.322.7744

SIGVARIS CORP., 8423 Chemin Dalton, Montréal, QC, H2C 2L7  
Phone 1.800.363.4999

SIGVARIS SA de CV, Periférico Sur 4600, Piso 2, Insurgentes, Cuicuilco, Coyoacán, 04530 CDMX, México  
Phone +52 54242077

[www.sigvaris.com](http://www.sigvaris.com)

SIGVARIS and SIGVARIS GROUP are registered trademarks of SIGVARIS AG, CH-9014 St. Gallen/Switzerland, in many countries worldwide.  
© 2021 Copyright by SIGVARIS AG, CH-9014 St. Gallen/Switzerland

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9

**EN:** Once the compression stocking is on the leg, roll the DND down over the heel again.

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10

**EN:** Smooth the compression stocking over the leg with Sigvaris donning gloves for a wrinkle-free fit.

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3

## DOFF N' DONNER

**EN: DOFF N' DONNER – doffing device for compression stockings:** This is how you can use the DND to remove compression stockings.

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## DOFF N' DONNER

**DOFF N' DONNER PRODUCTS consisting of:**  
**A:** DND  
**B:** CONE (sold separately)  
Intended purpose: Aids donning and doffing of medical compression stockings  
Contraindications: No known contraindications

**Indications:**  
- Pain in the back, hands, arms or legs  
- Rheumatism, arthritis, arthrosis  
- Obesity  
- Problems when bending  
**Suitable for the following models:**  
A-D Calf stocking, open / closed toe, with or without grip top  
A-G Thigh stocking, open / closed toe, with or without grip top

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## DOFF N' DONNER

**EN: CAUTION: DO NOT HEAT OR PLACE IN DIRECT SUNLIGHT OR ON HEATED SURFACES!** Heat will damage this product. Exercise **CAUTION** when using pointed or sharp objects. The product is **NOT** toxic.

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**EN: CAUTION:** If used frequently for a long period of time, the DND may become brittle and may wear out in exceptional cases. We therefore recommend storing the DND in a safe place away from light and heat. When used and stored correctly, we guarantee functionality for a period of 6 months from the date of purchase.

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**EN:** DND and CONE have been tested only with SIGVARIS GROUP stockings. SIGVARIS GROUP excludes any liability to the extent allowed by law. In particular, SIGVARIS GROUP accepts no liability for improper use of the DND and CONE, for user errors or for other harm suffered by the customer/patient as a result of its use, especially in conjunction with products not manufactured by SIGVARIS GROUP.

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## CLEANING DOFF N' DONNER

**EN:** After use we recommend that the DND be cleaned and / or disinfected. For private use it is recommended that the DND be cleaned in a commercially available plastic bag filled with y water. For institutional use (e.g. hospitals, doctor's surgeries, medical suppliers etc.), the DND must be disinfected.

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## DOFF N' DONNER

**EN:** All serious incidents that occur when using the product should be reported to the manufacturer.

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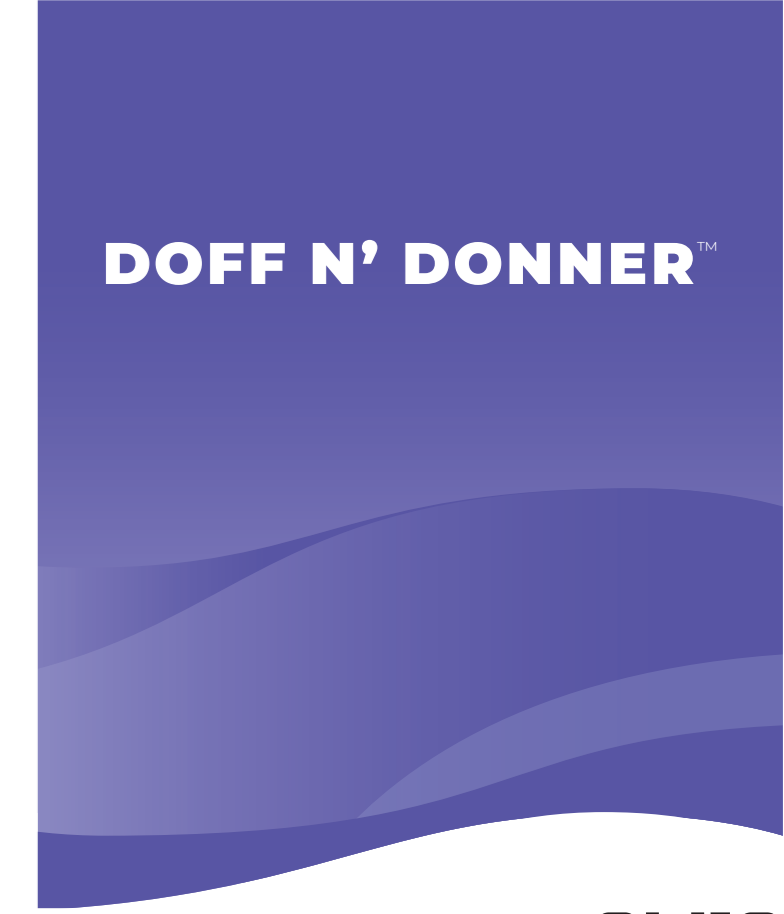
**EN:** Before each use, check the product for signs of wear or deterioration. Optimum functionality of the product can only be ensured if it is completely intact.

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Instructions for use  
Mode d'emploi  
Instrucciones de uso

**EN:** The DOFF N' DONNER (DND) is a soft device that allows you to quickly and easily guide a compression sock or stocking up and down a leg or arm, making it simple to put on and remove all styles of medical compression. The DOFF N' DONNER CONE (CONE) simplifies the rolling of a compression stocking onto the DND. With the aid of the CONE, the medical compression stocking can also be easily unrolled from the DND.

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1

**EN:** Clamp the CONE to a smooth surface (press the lever down, CONE adheres firmly).

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2

**EN:** Ensure compression stocking is turned right side out. Place the compression stocking with the top band (soft top or grip top) over the CONE.

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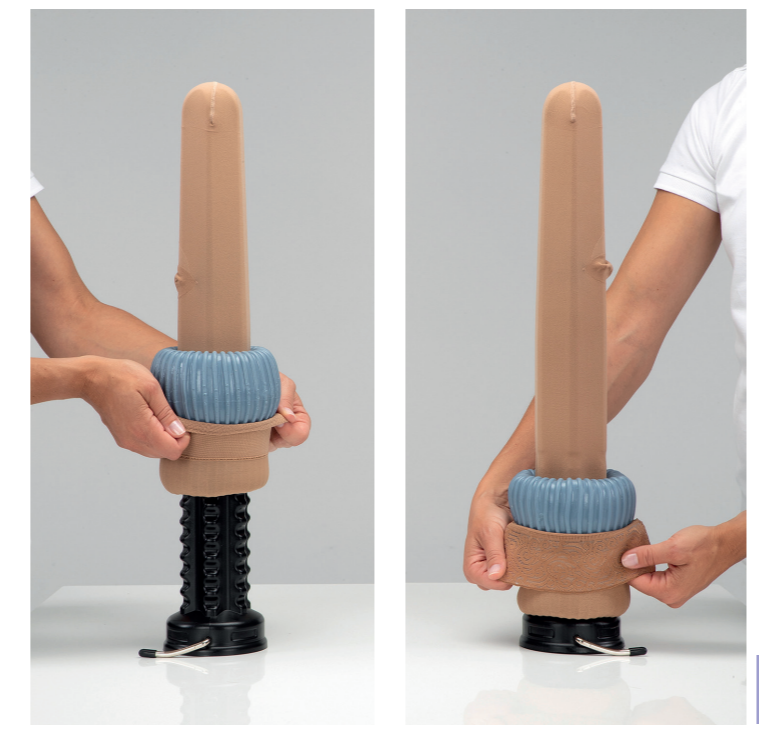
3

**EN:** Roll the compression stocking completely over the CONE.  
**Closed toe:** Position the toe seam flat on the CONE tip.  
**Open toe:** Stocking should end 3 fingers width from the CONE tip.

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4

**EN:** Place the DND over the CONE and roll it downward. Pull the top band of the compression stocking up over the DND.

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5a

**EN:** Calf stocking: Roll the DND upward until the compression stocking is rolled completely onto the DND.

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5b

**EN:** Thigh stocking: Gently pull the top band of the compression stocking beyond the tip of the CONE to avoid bunching. Grasp the DND and roll it upward until the compression stocking is rolled completely onto the DND.

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6

**EN:** Gently roll the DND to flatten the toe seam. Hold the DND so the heel of the compression stocking is visible from the outside and is facing the floor.

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7

**EN:** Place the toe seam on the toes. The heel of the rolled-up compression stocking points downward.  
**Open toe:** Place the cuff according to the arrow.

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