



## USER MANUAL

English

**Quha Zono X**

Enabling | Independent | Life

# Quha Zono X User Manual

## Contents

Quha Zono X mouse	3	Gestures	18
Compatibilty and requirements	4	Troubleshooting	20
Package contents	4	Technical information	22
Product overview	5	Safety	23
Charging the battery	7	Contraindications	24
Bluetooth® pairing	8	Cleaning the mouse and the ear clip	24
Turning the mouse on and off	10	Battery replacement	24
Resetting the mouse	10	Warranty	24
Wearing the mouse	11	Product disposal	25
Using the mouse	12	European compliance	25
Summary of power on/off switch functions	12	Industry Canada compliancce	25
LED indicator	13	FCC compliance	26
Quha Zono X setup software	14	Copyrights and trademarks	27

Quha Zono X User Manual version 1.0 2022-08-29

## Quha Zono X mouse

Welcome to the Quha Zono X mouse. This user manual will guide you through the functions and features of your device. Please read these instructions carefully before you start using your Quha Zono X mouse.

The Quha Zono X computer mouse is intended to be used for computer control for augmentative and alternative communication (AAC) and/or for computer access purposes as an alternative mouse input device for people that have challenges in their ability to access a computer due to injury, disability or illness.

Quha Zono X is a *Bluetooth*<sup>®</sup> computer mouse which uses motion detection. Its user can perform all regular mouse functions without hands. The user wears the mouse using one of the included clips and moves the mouse cursor with simple head movements. Quha Zono X can be used in any position.

### Key features

- Bluetooth<sup>®</sup> connectivity
- Pixel perfect accuracy
- Lightweight, easy to use for hours
- Convenient and comfortable magnetic ear clip for wearing
- Simple set-up software for Windows
- Can be set up on eight devices
- Easy charging with a magnetic charging dock

## Compatibility and requirements

Quha Zono X is compatible with computers and mobile devices that support Bluetooth<sup>®</sup> Low Energy wireless technology. Ensure that your device is Bluetooth<sup>®</sup> enabled.

The mouse requires separate dwell software or a switch for mouse clicks and dragging (selections). The Quha Dwell software for clicking is included. Any available dwell software can be used, and many devices also have a built-in dwell selection functionality as part of their operating system's accessibility settings. NOTE! The mouse device itself does not have a dwell selection function.

## Package contents

In the sales box, you will find the following items:

- Quha Zono X mouse
- Magnetic charging dock
- USB-C charging cable
- Ear clip and replaceable ear tips
- Eyeglass clip
- Quha Dwell software license and quick guide
- User manual and download instructions
- Warranty card

## Product overview

### Quha Zono X mouse:

1. Power on/off switch
2. LED indicator
3. Contacts for charging and external switch
4. Magnets for attaching the mouse to the ear clip and to the charging port
5. Serial number



### Ear clip:

6. Ear hook
7. Magnetic holder for the Quha Zono X mouse
8. Ear tip holder and replaceable ear tip




### Eyeglass clip:

Clip for attaching the Quha Zono X mouse to your eyeglasses

### **Magnetic charging dock:**

1. Slot for the Quha Zono X mouse for charging
2. USB-C port for the charging cable
3. Charging contacts

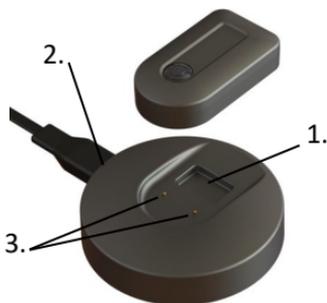

### **Buzzer**

The mouse has a buzzer that plays notification sounds to assist you in using the device:

- Power on/off sound: ascending/descending sequence
- Low battery alarm: 1 beep every 1 minute
- When the mouse connects to the charger: 1 beep
- When the mouse enters pairing mode (see p. 8) : 1 beep
- When the mouse enters reset mode (see p. 10) : 3 beeps
- Notifications that help you perform gestures (see p. 18)

The buzzer is turned on by default. You can turn it off in the Quha Zono X set-up software (see p.14). We recommend keeping the buzzer on when you first learn how to use the mouse.

## Charging the battery

The Quha Zono X mouse has a rechargeable battery that needs to be charged before the first use. To charge the battery

1. Connect the charging dock to the USB port of a computer or to a separate USB charger using the provided USB-C cable. Only use an approved charger.
2. Place the mouse into the slot on the magnetic charging dock. It will snap into the correct position automatically, and charging will begin immediately on contact. A flashing green LED indicates that the battery is charging.

It takes approximately 2 hours to fully charge an empty battery. The LED indicator will turn solid green once the battery is fully charged. The mouse will turn off automatically once you lift it off the charging dock.

You can use the mouse for up to 18 hours on a single charge. You can leave the mouse on the charger overnight. When the battery charge level gets low during use, the LED will start flashing red and the buzzer will start playing a low battery alarm sound (1 beep every 1 minute).

If you have connected the mouse to a computer, you can check its battery percentage in your computer's Bluetooth settings. Check the instructions on your computer.

**NOTE!** If the mouse turns off because of low battery charge, it will not activate again until you connect it to the charger.

## Bluetooth pairing

The Quha Zono X mouse connects to computers and mobile phone via Bluetooth. To start using your mouse, first you need to pair it with your device. You can connect the mouse up to eight different devices.

### **To pair the mouse with the first device**

1. Go to your device's Bluetooth settings and turn on Bluetooth. Keep the Bluetooth settings window/screen open. You may need to make some additional selections on the screen to allow your device to search for a new Bluetooth device. Follow the instructions on your device.
2. To turn your Quha Zono X mouse on, press the power switch for 1 second. The LED will flash turquoise once. It will then start flashing blue and red alternately, indicating that the mouse is in pairing mode and discoverable.
3. To pair, select "Zono X" when it appears on the screen. Follow any additional on-screen instructions to complete the pairing process.

Once the mouse is connected, the LED will flash green every 5 seconds. The mouse cursor will appear on the screen and you will see "Zono X" in your device's connected devices.

After a Bluetooth connection has been established, the mouse will connect to the paired device automatically whenever the device is within range with Bluetooth on.

## **To pair the mouse with a second device**

1. Turn off Bluetooth on any previously paired device that's in the same space, so that the mouse can't connect to it automatically. Turn on Bluetooth on the device you want to pair with your mouse and enable adding a new Bluetooth device.
2. To turn the mouse on, press the power switch for 1 second. Once the mouse is turned on, the LED should start flashing blue, indicating that the mouse is currently not connected to any previously paired device.
3. To place the mouse in pairing mode, press the power switch for 5 seconds. The LED will start flashing blue and red alternately once the mouse enters pairing mode.
3. To pair, select "Zono X" when it appears in the list of available devices on the screen. Follow any additional on-screen instructions.

Once the mouse is connected to the new device, the LED will flash green every 5 seconds. The mouse cursor will appear on the new device's screen and you will see "Zono X" in its connected devices.

If you have two or more paired devices within range, you need to turn off Bluetooth on the devices that you are not using. The mouse will connect to the available device automatically.

If you want to uninstall the mouse on your device, you need to remove "Zono X" from its paired devices. Refer to the instructions on your device for information on how to remove Bluetooth devices.

## Turning the mouse on and off

**NOTE!** When you turn the mouse on for the first time, it will go into Bluetooth pairing mode automatically, so that it's ready to connect to the first computer or mobile device (See p. 8).

To turn the mouse on, press the power switch for 1 second. You will hear a start-up sound and The LED indicator will first flash turquoise once. Once the mouse is connected to your computer or mobile device, the LED will start flashing green every 5 seconds, indicating that the mouse is in normal usage mode. The mouse cursor will appear on the screen.

**NOTE!** If the LED flashes blue, the mouse is not connected to your paired device. Make sure that Bluetooth is turned on, that the devices are close to each other and that there are no obstacles between them.

To turn the mouse off, press the power switch for 1 second. You will hear a power-down sound and the LED will flash red once before the mouse turns off.

## Resetting the mouse

Resetting the mouse clears its pairing history and any other changes to its settings. To reset the mouse, first turn it on. Then press the power switch for 15 seconds. You will hear 3 beeps and the LED will turn red for a while. Once the reset is finished, the mouse will turn off. When you turn the mouse on after a reset, it will go into pairing mode automatically.

## Wearing the mouse

### Using the ear clip

1. Attach one of the included ear tips to the ear tip holder. Handle the ear tip carefully when you attach or detach it to avoid tearing it. Do not wear the ear clip without an ear tip, as doing so may irritate your ear.
2. Place the ear clip on your right ear: place the ear hook around the back of your ear and insert the ear tip into your ear. The front edge of the magnetic holder should point directly forward.
3. Bring the mouse close to the magnetic holder and the magnets will grab it into the correct position automatically. The mouse stays firmly in place, but is also very easy to detach from the holder.



### Using the eyeglass clip

Fold the clip around the right arm of your eyeglasses so that the magnetic slot remains below the arm. Bring the mouse close to the slot to attach it.

## Using the mouse

The Quha Zono X mouse uses a motion sensor to detect movement and transforms the movement it into mouse cursor movement on-screen. Once you have placed the mouse on your ear, just turn your head and the mouse cursor will follow your head movements. Moving the cursor is intuitive and can be adopted in a few seconds.

For clicking and dragging you can use any available dwell clicker software, your computer's or mobile device's in-built dwell clicker function, Bluetooth switches or Quha Zono X compatible switch.

You can adjust the mouse features according to your needs with the Quha Zono X setup software (see p. 14). As Quha Zono X is recognized as a standard Bluetooth mouse, you can also use your device's mouse settings.

## Summary of power on/off switch functions

Press duration	Function	LED indicator
1 second	Power on	Flashes turquoise once
1 second	Power off	Flashes red once
5 seconds	Activates pairing mode	Starts to flash blue and red alternately
15 seconds	Resets the mouse	Turns red for a while

## LED indicator

Color	LED indicator Status	Device Status
Green	Flashes slowly	Battery is charging.
Green	Solid	Battery is fully charged.
Green	Flashes every 5 seconds	Mouse has been turned on, and it is connected and in normal usage mode.
Turquoise	Flashes turquoise once	Power switch has been pressed for 1 second. Mouse will be turned on.
Blue	Flashes	Mouse has been turned on and is waiting to be able to connect to a paired device.
Blue/Red	Flashes blue and red alternately	Power switch has been pressed for 5 seconds and mouse is in pairing mode.
Red	Flashes once	Power switch has been pressed for 1 second. Device will turn off.
Red	Flashes every 5 seconds.	Mouse has been turned on and is in usage mode but battery charge is low.
Red	Turns red for a while	Power switch has been pressed for 15 seconds and mouse is resetting.
-	Turned off.	Mouse is turned off.

## Zono X setup software

You can adjust the features of your Quha Zono X mouse easily with a simple setup application. The mouse will work with the default settings without the setup software, but we recommend downloading it on your device and familiarizing yourself with its functions.

The Zono X setup software will run on Windows.

Download the software from [www.quha.com](http://www.quha.com). Run the Quha Zono X setup software and follow the on-screen instructions. When the installation is complete, there will be a Quha Zono X icon on your computer desktop/mobile device screen. Double click the icon to start the application.

You can use the software when your Quha Zono X mouse is turned on and connected to your device. If the application shows a “Zono X not found” error message, make sure that the mouse is turned on and connected via Bluetooth.

**NOTE!** You can only have one Zono X paired to your computer when using the application.

**TIP!** Select the info ⓘ mark next to a feature name to see more information about the feature.

The settings made with the setup software will remain in the Quha Zono X mouse internal memory and will automatically be active when the mouse is connected to any computer or mobile device.

## Horizontal and vertical sensitivity / speed

With the sensitivity sliders you can adjust the amount of head movement required to move the mouse cursor on the screen. You can adjust horizontal and vertical sensitivity separately, which is useful for users whose head movement is more limited in one of the directions. High cursor accuracy and precision will be maintained automatically even when mouse sensitivity is set to the maximum.



## Tremor filter

The tremor filter reduces unwanted cursor movement caused by shaking. The exact effect depends on the individual movements of each user, so you need to try the filter to see how the different positions suit your needs.



## Gestures on/off

You can turn gestures on and off. Learn more about gestures on p. 18.

Gestures ⓘ

On



## Double-click assistant

The double-click assistant helps you perform a double-click more easily. It stops the cursor for a short time period after the first click, making it easier to click the second time without moving the cursor between the clicks.

NOTE! The double-click assistant will not make the mouse double-click automatically. You can turn the double-click assistant on and off and adjust the length of the delay after the first click.

Double-click assistant ⓘ

On



Short



Long

## Buzzer on/off

You can turn the buzzer on and off. NOTE! Turning the buzzer off will mute all of the notification sounds.


Buzzer ⓘ

On



## Performance settings

You can choose from 3 different performance modes.

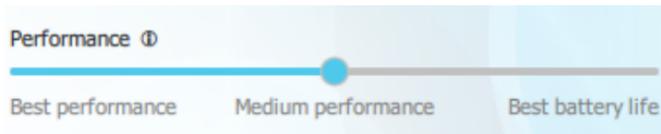

*Best performance.* Highest performance and highest battery consumption.

*Default.* Medium performance and medium battery consumption.

*Best battery life.* Automatic power saving functions are on and the mouse will turn off if not used for 30 minutes.

Changing the mode will have an effect on the Bluetooth® range of your mouse and on how smoothly the mouse cursor moves on the screen (see technical details on p. 22). The high power mode may be the best choice in some situations for tasks requiring constant precise cursor control or if there are obstacles between the Zono X mouse and the computer. In other situations the best battery life mode will provide sufficiently smooth cursor movement and you may prefer to extend battery life. We recommend trying the different modes to see how they suit your purposes.

## **Gestures**

The Quha Zono X mouse has built-in gesture detection for activating three commonly used mouse features hands free. Gestures are turned on by default. You can turn them on and off with the Quha Zono X setup software.

To activate a gesture action, you need to turn your head right and left or up and down a certain number of times. The buzzer will help you perform the movements correctly: you will hear one beep once you have completed one part of the required motion pattern; an ascending sound sequence will confirm that you have completed the entire gesture (except for the exit gesture). All the gestures start by first holding the mouse still for about 1 second.

### **Pause**

To activate: hold still 1 s - move right - left - right - left

Pause stops the mouse cursor and keeps it still until you make the exit gesture. Depending on the settings of your computer or mobile device, the mouse cursor may or may not disappear once the pause mode is activated.

### **Scroll**

To activate: hold still 1 s - move up - down - up - down

With the scroll function you can scroll a window by turning your head up and down. Once you have completed the scroll gesture, you will have 5 seconds for moving the mouse cursor on the window you want to scroll. The buzzer will beep rapidly during the 5 seconds. Once 5 seconds have passed, you will hear a sound sequence confirming that the scroll mode is active,

and you can now scroll the window with head movements. To end scrolling, make the exit gesture.

## Center

To activate: hold still 1 s - move right - left - up - down

Centering moves the pointer in the central area of the screen, making it easier for you to move your head into a more comfortable position before you start moving the cursor again. The pointer path is shown while moving.

## Exit pause/scroll

To activate: hold still 1 s - move right - left

To exit the pause or scroll mode, you need to make the exit gesture. After the exit gesture the mouse cursor will move in the usual way again.

Step	Pause	Scroll	Center	Exit pause / scroll
1				
2				
3				
4				
5				

NOTE! Always remember to first hold the mouse still, otherwise the mouse will not register your head movements as parts of a gesture.

## Troubleshooting

TIP! Answers to many questions regarding the Quha Zono X mouse can be found in the frequently asked questions (FAQ) page at [www.quha.com/support/frequently-asked-questions/](http://www.quha.com/support/frequently-asked-questions/).

### **Why is the cursor drifting when I'm not moving the mouse?**

The Quha Zono X mouse has a motion sensor that detects the mouse movements and transforms these movements into cursor motion. The motion sensor is a highly accurate electric component and is affected by ambient temperature among other things. If the cursor drifts, do the following:

1. Place the turned on device on a table so that it keeps still
2. Wait until the cursor stops moving
3. The mouse functions now without cursor drift

### **Why are gestures not working?**

Make sure that gestures are turned on in the Quha Zono X setup software. You may also need to practice the head movements a little to learn how to perform the gestures. Keep the buzzer on while you are learning.

## **What if can't/don't want to use the gestures and I find it difficult to center the cursor?**

Readjusting the mouse by pushing the mouse cursor off screen is a standard way of repositioning the mouse cursor. If you notice your head is not in a comfortable position when the cursor is in the middle of the screen, turn your head so that the cursor moves to the edge of the screen and beyond.

## **Why has the mouse cursor suddenly stopped/disappeared?**

You may have unintentionally performed the pause gesture and the cursor has been paused. You need to make the exit gesture to release the cursor.

## **Why are the mouse cursor movements jumpy/not smooth?**

The mouse may be so far from the computer that the wireless signal is disrupted and some mouse movements are lost. Try using the high performance mode (p. 17).

## Technical Information

### Quha Zono X mouse

Measurements:	Length 44 mm / 1.7 inches, width 25 mm / 1.0 inches, thickness 13 mm / 0.5 inches, weight 12 g / 0.4 ounces
Operating Temperature:	+10 °C - +35 °C / +50 °F - +95 °F
Storage temperature:	+0 °C - +45 °C / +32 °F - +113 °F
Bluetooth range:	Up to 10 meters / 30 ft
Bluetooth transmit power:	+4 dBm in Best performance mode, +3 dBm in Medium mode, +3 dBm in Best battery life mode
Battery:	LiPo (Lithium Polymer) 90 mAh. Active usage time is up to 18 hours depending on power saving/performance mode
Mouse refresh rate:	208 Hz

### Magnetic charging dock

Connection:	USB-C
Charging time:	2 hours

## Safety

The mouse device and the charging dock include sensitive electronic components and are not user serviceable. Device service and battery replacement may only be done by a Quha certified service point.

The mouse cannot be used while its battery is being charged.

The mouse and the charging dock are for indoor use only. Do not allow them to get wet. Before charging, ensure the USB-C port is clean.

Do not let metal objects come into contact with the charging contacts of the charging dock, as that may cause a short circuit and damage the charger.

Do not let children play with the product. The product contains small parts that may pose a choking hazard.

If the mouse swells or gets hot, or its casing is cracked, the battery is probably faulty. Stop using the device immediately and have the device serviced. Report any serious incident to the manufacturer.

If you feel any pain or discomfort during the mouse usage, stop using the mouse for a while.

The mouse may be used only for its intended purpose.

The mouse, the charging dock, the ear clip and eyeglass clip contain magnets. Do not place it close to magnetically sensitive devices such as pacemakers, hearing devices, credit cards, mechanical watches, data storage media etc. as they may get damaged.



Any modifications not approved by Quha could void the user's authority to operate the device.

## **Contraindications**

The device should not be used by persons with a spine or neck injury that prevents them from moving their head safely, or moving their head for long periods could cause harm to them.

## **Cleaning the mouse and the ear clip**

The mouse and ear clip surfaces can be cleaned by wiping its outside plastic surfaces with a mild detergent.

## **Battery replacement**

When the device usage time is significantly shortened from its design length, you can have the battery replaced by a Quha certified service point.

## **Warranty**

2 years (you can find the warranty card in the sales box, see the entire warranty text at our website ([www.quha.com/warranty](http://www.quha.com/warranty)))

## Product Disposal

When the device has reached its end of life it should be recycled as electric and electronic waste.



## European Compliance

The manufacturer of this product conforms that this product complies with the European Union regulations for safety, health, environmental and consumer protection.



## Industry Canada Compliance

This device complies with Industry Canada licence-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

Le présent appareil est conforme aux CNR d'Industrie Canada applicables aux appareils radio exempts de licence. L'exploitation est autorisée aux deux conditions suivantes : (1) l'appareil ne doit pas produire de brouillage, et (2) l'utilisateur de l'appareil doit accepter tout brouillage radioélectrique subi, même si le brouillage est susceptible d'en compromettre le fonctionnement.

## FCC Compliance

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

## Japan radio equipment compliance



R 204-810005

## Taiwan NCC compliance

內含發射器模組:  CCAI18LP2200T2

## KCC South Korea compliance



R-C-ULX-ANNA-B112

## Brazil compliance

 **ANATEL**  
03850-19-05903

“Este equipamento opera em caráter secundário, isto é, não tem direito a proteção contra interferência prejudicial, mesmo de estações do mesmo tipo, e não pode causar interferência a sistemas operando em caráter primário.”

## Copyrights and trademarks

Quha and the Q-mark are registered trademarks of Quha oy.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Quha is under license. Other trademarks and trade names are those of their respective owners.

© 2022 Quha oy. Quha oy is not responsible for possible errors in this user manual. Quha oy reserves the right to revise the content of this user manual.

Quha oy  
Pirkkalaistie 1  
37100 Nokia  
Finland

[info@quha.com](mailto:info@quha.com)  
[www.quha.com](http://www.quha.com)

