Instruction for use

IMM1067_IFU.Version4 2022-12-15

LeanOnMe - Positioning cushions

immedia
by Etac

Index

Illustrations	3
User manual – English	4
Bruksanvisning – Svenska	10
Bruksanvisning – Norsk	16
Brugsanvisning – Dansk	22
Käyttöohjeet – Suomi	28
Gebrauchsanweisung – Deutsch	34
Gebruiksaanwijzing – Nederlands	40
Instruction d'utilisation – Français	46
Istruzioni per l'uso – Italiano	52
Instrucciones de uso – Español	58

See www.etac.com

Illustrations

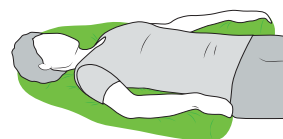
1

2


3


4

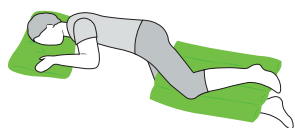
5


6

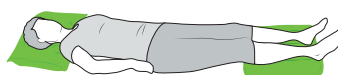
7

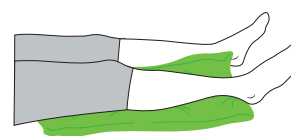8

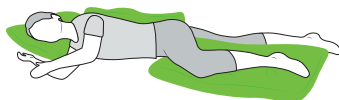9

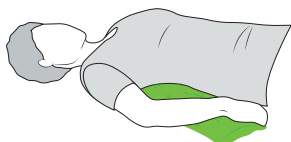
10


11

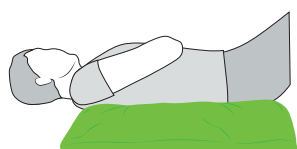12

13

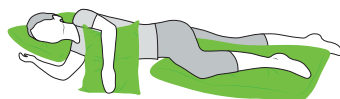
14


15

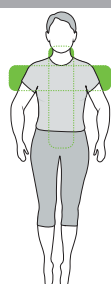16


17

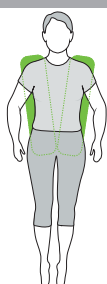18

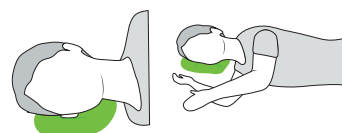19

20

General

Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, please read this manual carefully.

The person described as “the user” in this “Instruction for Use” is the person who is lying or sitting on the product. The carers are the people who manoeuvre the product.

This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the user or carer.

These products comply with the standards applicable for Class 1 products in the Medical Device Regulation (EU) 2017/745 on medical devices.

At Etac we strive to improve our products all the time and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply

to special orders and modifications. If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

Expected lifetime: The device has an expected service life of 1 to 2 years under normal use. The service life of the device varies depending on usage frequency, loads and how often and how it is washed.

For further information on the Etac transfer range, see www.etac.com.

The product can be scrapped in accordance with national regulations.

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Symbols in general

Washing	Disinfect / Wipe off	Warning	Manufacturer
Do not bleach	Never leave on the floor	Production year, month and day	CE-marking
Tumbledry	Risk of sliding down	Batch no./Lot no.	Keep dry
Iron	Keep out of the sun	User mass limit = maximum rated load	Non sterile
Do not dryclean	Read the user manual	The product can be scrapped according to national regulations	Article number
Medical device	Single Patient Multiple use		

Intended purpose	Positioning the user and supporting the body or extremities for users with special needs.
Intended target group	Used for users with special needs; e.g. people with Alzheimer disease, Cerebral palsy, Hemiplegia, Multiple sclerosis, Paraplegia/Quadriplegia, Parkinson disease, Spina bifida, and people with a variety of psychiatric conditions causing emotional dysregulation, anxiety, distressed and/or combative behavior. Not an exhaustive list.
Intended Environment	For use in acute care, long term care, home care
Indications	The device can be used by children, adults and elderly people with a need for positioning or support of body or extremity. The target group for the device is based on individual health and mobility function, and not a specific diagnose or age.
Contra indications	There are no known contra indications

Practical handling

Positioning

Repositioning users with or at risk of pressure injuries and determine the individual's frequency of repositioning is strongly recommended by international experts. Inability to make postural adjustments independently can lead to many negative health implications, e.g. pressure injuries and body shape deformity. LeanOnMe can be used to support the body and extremities, and thus achieve a relieving and calming body position.

Below is only suggestions on how to use LeanOnMe, it should always be the user's special needs that define the choice of cushion/cushions and/or combinations and finally positioning.

Use

Immedia LeanOnMe Roll

The roll can be bent, shaped and adjusted exactly to user needs. It can also be shaken so that it becomes fuller in some areas and thinner on others. In lateral side position it can be shaped to support the head, back and legs. Is very suitable for neurological users, people with decreased body awareness or anxiety. (Illustration 1,2,3,4,5)

For users with brain injury or hemiplegia as it provides a good tactile stimulation.

Immedia LeanOnMe Channel

The channel-sewn cushion is a flexible pillow that protects, supports and relieves. The cushion can be rolled depending on the application. The cushion can be shaped and placed under the leg(s), between the legs, under the calves to relieve the heels, or under an arm to relieve shoulder blades and/or elbows. Support an arm or in lateral position, the cushion can be placed in the back or between the legs to protect exposed areas. In the sitting position, the pillow is placed in the back as a support by e.g. dining. (Illustration 6,7,8,9)

Immedia LeanOnMe Wing

The backrest cushion with wing is used lateral position. The user lies on the wing so that the user's own weight holds the cushion in place. Then the back is supported precisely and stable. The cushion can also be used supine to support the arms or to protect exposed body parts from pressure from bed rails. (Illustration 9,13)

Immedia LeanOnMe Mini

A functional small support cushion, which is placed where smaller gaps occur between body and mattress when a person is positioned. Typically, at the ankle and knee. Applying the cushion has a relaxing effect on the body part, which propagates to the rest of the body. The small cushion is the professional replacement for the duvet clip - but it does not collapse or flatten out. (Illustration 10,11,12)

Immedia LeanOnMe Basic

Traditional positioning cushions in 6 sizes can be used, in lateral and supine positioning for supporting the back, arms and legs, under the knees and ankles, or in supine position to facilitate breathing. The positioning cushions can also be used to protect exposed body parts from pressure from bed rails. (Illustration 6,7,8,9,13,14,15,16,17,18)

Immedia LeanOnMe Ring

The neck cushion is used to provide a stable and relieving support when a normal pillow is not considered appropriate. For users with poor head control, it can also be used in a half-seated bed or armchair. Provides good headrest. The neck Ring can easily be taken on trips to relieve sore neck muscles and vertebrae. (Illustration 19,20)

Notice

Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that carers receive instruction in manual handling. Etac offers advice and training for carers. For further information, contact Etac.

Encourage the user to assist where possible.

To ensure that the user feels safe and that every manual handling is smooth, always plan the manual handling in advance.

The choice of model depends not only on the needs but also on the user's resources.

Always conduct a risk assessment, and ensure that the assistive product can be used for the intended user and, in combination with other devices, is safe for both user and carers.

Never use a product without any cover.

Never leave on the floor.

Important that the cushion do not block the airways

If in any doubt - please contact Etac.

Material and cleaning

Material:

Cover: 75% Lyocell (TENCEL™ + SMARTCEL™ sensitive), 25% Modal (TENCEL™)

Back: 100% Polyester + Polyurethane membrane

Innercushion: 50/50% Cotton/Polyester

Filling: Polyurethane foam + EPP-beans

The product should be checked regularly, preferably each time it is used, and especially after washing.

Check that there is no damage to seams or fabric.

Washing instructions:

Recommended to wash the product before use.

Remove innercushion before washing

Do not use fabric softener

Do not over dry

Do not drain dry

Shake thoroughly after drying

This product can be cleaned with a solvent-free cleaning agent with a pH level between 5 and 9, or a 70% disinfection solution

Never try to repair a product yourself.
Never use a defective product.

Products

Article no.	Product Description	Size (mm)
IM15100	LeanOnMe Mini + Soft-Touch Cover	W150xL300 mm
IM15101	LeanOnMe Basic XS + Soft-Touch Cover	W300xL400 mm
IM15102	LeanOnMe Basic S + Soft-Touch Cover	W400xL600 mm
IM15103	LeanOnMe Basic M + Soft-Touch Cover	W500xL700 mm
IM15104	LeanOnMe Basic L + Soft-Touch Cover	W250xL900 mm
IM15105	LeanOnMe Basic LW + Soft-Touch Cover	W450xL800 mm
IM15106	LeanOnMe Basic XL + Soft-Touch Cover	W550xL900 mm
IM15107	LeanOnMe Wing + Soft-Touch Cover	W450xL800 mm
IM15108	LeanOnMe Channel + Soft-Touch Cover	W750xL750 mm
IM15110	LeanOnMe Ring S + Soft-Touch Cover	Ø400 mm
IM15111	LeanOnMe Ring M + Soft-Touch Cover	Ø500 mm
IM15112	LeanOnMe Roll S + Soft-Touch Cover	W330xL1000 mm
IM15113	LeanOnMe Roll L + Soft-Touch Cover	W330xL2300 mm
IM16100	LeanOnMe Mini + Hygienic Cover	W150xL300 mm
IM16101	LeanOnMe Basic XS + Hygienic Cover	W300xL400 mm
IM16102	LeanOnMe Basic S + Hygienic Cover	W400xL600 mm
IM16103	LeanOnMe Basic M + Hygienic Cover	W500xL700 mm
IM16104	LeanOnMe Basic L + Hygienic Cover	W250xL900 mm
IM16105	LeanOnMe Basic LW + Hygienic Cover	W450xL800 mm
IM16106	LeanOnMe Basic XL + Hygienic Cover	W550xL900 mm
IM16107	LeanOnMe Wing + Hygienic Cover	W450xL800 mm
IM16108	LeanOnMe Channel + Hygienic Cover	W750xL750 mm
IM16110	LeanOnMe Ring S + Hygienic Cover	Ø400 mm
IM16111	LeanOnMe Ring M + Hygienic Cover	Ø500 mm
IM16112	LeanOnMe Roll S + Hygienic Cover	W330xL1000 mm
IM16113	LeanOnMe Roll L + Hygienic Cover	W330xL2300 mm

Accessories

Article no.	Product Description	Size (mm)
IM10500	Soft-Touch Cover for IM15100	W150xL300 mm
IM10501	Soft-Touch Cover for IM15101	W300xL400 mm
IM10502	Soft-Touch Cover for IM15102	W400xL600 mm
IM10503	Soft-Touch Cover for IM15103	W500xL700 mm
IM10504	Soft-Touch Cover for IM15104	W250xL900 mm
IM10505	Soft-Touch Cover for IM15105	W450xL800 mm
IM10506	Soft-Touch Cover for IM15106	W550xL900 mm
IM10507	Soft-Touch Cover for IM15107	W450xL800 mm
IM10508	Soft-Touch Cover for IM15108	W750xL750 mm
IM10510	Soft-Touch Cover for IM15110	Ø400 mm
IM10511	Soft-Touch Cover for IM15111	Ø500 mm
IM10512	Soft-Touch Cover for IM15112	W330xL1000 mm
IM10513	Soft-Touch Cover for IM15113	W330xL2300 mm
IM10600	Hygienic Cover for IM16100	W150xL300 mm
IM10601	Hygienic Cover for IM16101	W300xL400 mm
IM10602	Hygienic Cover for IM16102	W400xL600 mm
IM10603	Hygienic Cover for IM16103	W500xL700 mm
IM10604	Hygienic Cover for IM16104	W250xL900 mm
IM10605	Hygienic Cover for IM16105	W450xL800 mm
IM10606	Hygienic Cover for IM16106	W550xL900 mm
IM10607	Hygienic Cover for IM16107	W450xL800 mm
IM10608	Hygienic Cover for IM16108	W750xL750 mm
IM10610	Hygienic Cover for IM16110	Ø400 mm
IM10611	Hygienic Cover for IM16111	Ø500 mm
IM10612	Hygienic Cover for IM16112	W330xL1000 mm
IM10613	Hygienic Cover for IM16113	W330xL2300 mm

Service information

Inspection

First inspection:

- Is the packaging intact?
- Read the label on the packaging and check the article no. and product description
- Check that Short Instruction is enclosed – updated User Manual/Instruction for use can be downloaded from www.etac.com or contact your Etac Customer Service or the local dealer.
- Check the label on the product - does it include article no., product description, lot/batch no., cleaning instructions and supplier name?

Periodic Inspection:

- Make sure that your Manual transfer system always is in a perfect condition.
- After wash – always check materials, stitching, handles, buckles.
- If the product shows signs of wear and tear, it must be removed from service immediately.

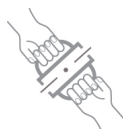
Visual inspection/Check the product:

Ensure that the materials, seams, stitching, handles, buckles are intact/faultless.

Stability test:

Try to bend the product verify that it feels firm, stable and solid.

Mechanical load or stress/Test handles:

Draw hard in the handles in opposite directions and control the material, seams and stitching.

Test buckles:

Lock the buckle and draw in opposite directions, control the buckle and the stitching.

Test low/high friction:

Place the product on a firm surface or a bed, place your hands on the material and test the friction by pushing your hands down into the product.

Low friction – slides effortlessly

High friction – no sliding/moving

Check:

Manual transfer aids with handles, straps and buckles:

Manual transfer aids with for sitting and lying transfer and with high or low friction:

Production year, month and day

The product can be scrapped in accordance with national regulations

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Accessories

Article no.	Product Description	Size (mm)
IM10500	Soft-Touch Cover for IM15100	W150xL300 mm
IM10501	Soft-Touch Cover for IM15101	W300xL400 mm
IM10502	Soft-Touch Cover for IM15102	W400xL600 mm
IM10503	Soft-Touch Cover for IM15103	W500xL700 mm
IM10504	Soft-Touch Cover for IM15104	W250xL900 mm
IM10505	Soft-Touch Cover for IM15105	W450xL800 mm
IM10506	Soft-Touch Cover for IM15106	W550xL900 mm
IM10507	Soft-Touch Cover for IM15107	W450xL800 mm
IM10508	Soft-Touch Cover for IM15108	W750xL750 mm
IM10510	Soft-Touch Cover for IM15110	Ø400 mm
IM10511	Soft-Touch Cover for IM15111	Ø500 mm
IM10512	Soft-Touch Cover for IM15112	W330xL1000 mm
IM10513	Soft-Touch Cover for IM15113	W330xL2300 mm
IM10600	Hygienic Cover for IM16100	W150xL300 mm
IM10601	Hygienic Cover for IM16101	W300xL400 mm
IM10602	Hygienic Cover for IM16102	W400xL600 mm
IM10603	Hygienic Cover for IM16103	W500xL700 mm
IM10604	HygienicCover for IM16104	W250xL900 mm
IM10605	Hygienic Cover for IM16105	W450xL800 mm
IM10606	Hygienic Cover for IM16106	W550xL900 mm
IM10607	Hygienic Cover for IM16107	W450xL800 mm
IM10608	Hygienic Cover for IM16108	W750xL750 mm
IM10610	Hygienic Cover for IM16110	Ø400 mm
IM10611	Hygienic Cover for IM16111	Ø500 mm
IM10612	Hygienic Cover for IM16112	W330xL1000 mm
IM10613	Hygienic Cover for IM16113	W330xL2300 mm

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Etac Immedia A/S
"Etac Ltd. Unit D4A, Coombswood Business Park East,