

Instructions for Use

INTENDED USE / CLINICAL BENEFIT:

The Flo-Form® Positioner uses a fluidized medium that maximizes contouring and support. The product is designed to aid in providing developmental positioning for neonatal, paediatric and adult patients while redistributing pressure on sensitive skin.

WARNINGS / CAUTIONS:

- Do not use if positioner has been torn or punctured.
- Clean with hospital approved disinfectant
- Place a barrier, such as a cover or sheet, between the positioner and the patient's skin
- Do not use in any manner that might restrict the patient's airway

CLEANING INSTRUCTIONS:

- The Flo-Form Positioner is a cleanable and reusable Multi Patient product.
- Disinfect by wiping the casing with a hospital-approved disinfectant, allow to dry.
- Can be cleaned with Antichlor plus 1000ppm clinell universal, and alcohol up to 70%
- Can withstand Hypochlorite 10,000ppm
- Can be cleaned using Tristel Fuse, and HPV or UV After disinfection, wipe down using a clean cloth moistened with clean water.
- Allow positioner to dry fully before use.
- Straight bleaching, scrubbing, machine washing, tumble drying, autoclaving not recommended. For Flo-Form cloth covers (Available Accessory):
- Machine wash and dry at temperature up to 71°C.
- Non-chlorine bleach is recommended.

INSTRUCTIONS FOR USE:

- The Flo-Form Positioner effectively redistributes pressure over a greater surface area and promotes neutral spine and limb alignment.
- The positioner is mouldable and can be adjusted to accommodate each patient's unique clinical needs.
- Take the positioner out of the plastic bag. Flo-Form positioners can be safely / evenly prewarmed to the ambient incubator temperature by placing it in the end space of the baby's incubator (or on an unoccupied pre-warming admit bed) before positioning the baby.
- Place the pre-warmed fluidized positioner with cover under the patient.
- Gently stabilize the baby on the positioner with one hand, then slowly push the perimeter of the positioner towards the baby.
- Remould as needed for medical equipment and position changes.

WARRANTY:

1 year warranty against manufacturer defects.

STORAGE AND DISPOSAL:

No special requirements

Material:

Not made with natural latex rubber. DEHP free.