Rolko
AIRpad

	Bedienungsanleitung.....	Seite 02 - 07
	Instruction manual.....	Page 08 - 13
	Mode d'emploi.....	Page 14 - 19
	Gebruiksaanwijzing.....	Pagina 20 - 25
	Betjeningsvejledning.....	Page 26 - 31

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1-Kammer-Luftkissen: 75% Polyester, 25% Thermoplastisches Polyurethan

Schaumstoff: 100% Polyurethan memory foam

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Dealer: This instruction manual has to be handed out to the customer. Please make sure that the user of the **Rolko-AIRpad** cushion fully understands this product information.

User: Read this instruction manual carefully for safe use and full benefit. Keep this instruction manual at hand for re-use of the seat pad.

Thank you very much for buying the **Rolko-AIRpad** and your trust in our products. Please pay attention to all safety instructions before using the cushion the first time. Check package/article for completeness.

Delivery content:

1 seat cushion 
1 removable standard cover
1 instruction manual
Optional accessory: incontinence cover

Min. user weight: 30 kg

Max. user weight: 150 kg

standard cover

seat cushion

Product description / suitability:

Rolko-AIRpad provides the feeling of firm and stable sitting in a wheelchair and other seat devices. It prevents soreness and helps the healing process of decubitus. **Rolko-AIRpad** can replace other already existing wheelchair cushions.

The combination of visco-elastic foam and adaptive air-chamber provides pressure relieve and comfort for sensible areas at prolonged sitting. The cushion can be used for prophylaxis and for decubitus therapy up to stage II, in accordance with EPUAP. The full-contour air-chamber of the cushion with adjustable air volume will provide a custom fit for every user. User weight is evenly distributed over the entire seat area, reducing soreness in sensitive areas.

At the same time, the memory-foam inside the air bag has a stabilizing effect, supporting a proper posture for the user.

The two-way-valve allows at any time a quick simple adjustment of the cushion to the user. The cushion is self-inflating; no air-pump required!

Select the size of the **Rolko-AIRpad** cushion in accordance with the size of the seat face. Please refer to „Technical Data“, Sizing Chart“ for proper selection.

User weight is limited to max. 150kg!

Contra Indications:

Above 150 kg or under 30 kg, the seat pad may not be used. A person who is unable to sit may not use the cushion. **Rolko-AIRpad** should also not be used in combination with other cushions or covers. Users with incontinence should only use the cushion together with optional incontinence cover.

As soon as you realize symptoms of skin-rash or other irritations, which do not disappear within a short time after removing the cushion, stop using it. Contact your medical doctor, therapist or your health care supply store.

Safety Instructions / Warnings:

- Check regularly for skin rash and other irritations especially in the area of the ischial tuberosity and the coccyx. If the skin-rash or irritation persists, please contact your medical doctor immediately!
- Check the seat cushion regularly while using it!
- Do not use the seat cushion in combination with other cushions!
- Do only use original seat pad covers and only in correct size!
- Do not use the seat cushion as floating aid!
- Do not use a pump for inflation; - the cushion is self inflating!
- Respect user weight limitation!
- Keep the seat cushion away from heat sources, hot ashes, cigarettes and flames! Do not expose the cushion to direct sunlight for an extended time!
- Do not expose the cushion to extreme low temperatures!
- Keep the cushion away from sharp and pointed objects, to avoid damage of the air chamber!
- Do not carry the cushion at the valve stem!
- Do not put any other objects between seat face and AIRpad nor between AIRpad and user. This will impair proper function of the AIRpad!
- The **Rolko-AIRpad** cushion should only be used with the proper air volume:
 - Under-inflation may cause injury to the soft tissue. Remove any weight from the cushion and open the valve to allow air-chamber to fill again. Re-adjust firmness of the cushion thereafter.
 - Over-inflation prevents the user from sinking into the cushion, which strongly reduces the positive effect of the cushion and may result in injuries of skin and soft tissue.
- If the cushion is used at an altitude above 2000 m, the air pressure inside the air chamber has to be adjusted. Remove any weight from the cushion and open the valve, to have the cushion in the self-inflating mode. Thereafter, adjust the pressure in the air chamber again to the user.
- Follow the cleaning instructions for seat pad and cover. Keep the valve closed at any time during the cleaning process!

Application:

Remove all cushions and potential other items from the wheelchair seat or other sitting device. Place the **Rolko-AIRpad** with the grey, anti-slip bottom side onto the seat. The breathable netlike top side of the cover points upward. Turn the **Rolko-AIRpad** until the valve points to the front and make sure it shows through the opening in the seat cover.

 **Both of the grey surfaces - seat pad and cover - have to point downwards! In this case, by looking at the valve stem, the white dot at the seat pad is on the left.**

Adapt the seat pad:

1. Open valve

Valve open / inflating

Put the seat cushion without load on the seat. Open the valve by turning it clockwise. **1** The white marker on the valve cap now points to the top and the cap noticeably snaps in position. Make sure that the cover is not caught by the valve cap!

The cushion now inflates itself. It is fully inflated, once the hissing noise at the valve stops.

2. Close valve and seat

Valve closed

Now turn the valve counter-clockwise to close it. **2** Once fully closed, the white marker at the valve cap points towards the white dot of the cushion and the cap snaps into position.

Sit down, or have the user sit down on the cushion, while the valve is still closed. Make sure, arm- and foot-rests of the wheelchair are properly positioned.

3. Comfortable adjustment

Deflating

Open the valve again by turning the cap clockwise, until you overcome a noticable resistance. If you now slightly turn the valve cap further, some of the air escapes from the cushion. **3**

Let only so much air escape, until the seating position feels stable and comfortable on the cushion. Any bone part of the pelvis should not touch the hard seat surface. Recommended optimal distance between lowest point (usually ischial tuberosity) and bottom side of cushion is approximately 1,5- 2 cm.

As a safety precaution check proper air volume of the cushion regularly:

As a simple measure, put your hand between the cushion and the lowest point of your buttocks. If you can move your fingers just a little, you have the proper inflation.

In case you have to re-adjust firmness of the cushion, always start again with point 1 of the instructions.

 ***Check regularly proper filling of the air cushion. Make sure the cushion offers enough padding to avoid any pressure point sores, but is also not too firm when over-inflated!***

Removing and fitting the cover:

Removing the cover:

1. Turn the valve clockwise to have some of the air escape the **Rolko-AIRpad** cushion.
2. Open the zipper and pull the cover carefully over the cushion. Avoid twisting or bending the valve!

Bottom of cover and cushion lie on top of each other

Slot for valve in cover

Pocket for zipper puller

Assemble cover:

1. Turn the valve slightly clockwise to have some of the air escape the cushion and open the zipper of the cover.
2. Open the zipper and insert the cushion with the valve ahead into the cover. Make sure, the grey side of the cushion is facing the grey anti-slip side of the cover. **a**
3. Pull the cover fully over the cushion and put the valve through the slot of the cover. **b**
Close the zipper and put the puller into the protection pocket at the end. **c**
4. Now open the valve to allow the seat cushion to self-inflate. Check if the cover is fitted properly and without wrinkles.

Cleaning and disinfection:

For cleaning and disinfection remove the cover from the cushion as explained above.

Cushion:     

Always close first the valve before you start cleaning the cushion, to avoid that any water is getting inside the cushion. The valve is closed when white dot and white mark on the valve face each other.

Little dirt of the seat cushion can be cleaned with a damp cloth, if necessary by use of a mild detergent (e.g. mild soap). For heavy dirt the cushion can be hand-washed. For disinfection use a microfibre cloth, soaked with a mild alcohol based disinfectant solution. Alternatively, an alcohol based spray disinfectant can be used.

Please let the cushion dry at ambient.

Don't use bleach, softener or any chemical dry-cleaning agents!

 ***Take care, that the valve is always closed before cleaning!***

Standard cover:     

The standard cover can be machine washed at 60°C with a gentle wash cycle and by using a mild detergent. For disinfection use a microfibre cloth, soaked with a mild alcohol based disinfectant solution. Alternatively, an alcohol based spray disinfectant can be used.

To protect other pieces of cloth during the wash cycle we recommend to close the zipper of the cushion and use a string bag. Do not use bleach, softener or any chemical dry cleaning agents!

Thereafter, the cover can be dried at ambient or tumble dried at low temperature.

After all components are completely dry, fit the cover again to the cushion as explained under **removal / fitting of the cover**.

 ***Consider the wash label at the cover!***

Maintenance, re-use and service life:

To maintain full functionality of the seat pad, let the cushion relax over night once a week with opened valve. Check the seat cushion regularly for tightness and proper functionality of the valve.

If the cushion cannot hold the air, the air chamber may be punctured; you may also notice a hissing noise. Check the entire contact area of the seat cushion top and bottom fabric and of the valve regularly. Look for damages to the surface, for abrasion and small holes. Check periodically all seams and for a proper fit of the cover to avoid the development of folds.

The seat cushion can be re-used. Before using the seat cushion for another patient, please perform the cleaning and disinfection steps according the *cleaning and disinfection procedure*. We recommended to use a new cover for each new user. Please make sure, that the AIRpad is passed on to the next user with all the components. This instruction manual is part of the the package and must be forwarded together with the cushion to the next user.

 ***Stop using a seat cushion with visible damage!***

Technical data:

Used materials:

Single-chamber air-cushion : 75% Polyester, 25% Thermoplastic Polyurethane

Foam: 100% Polyurethane memory foam

Valve: 40% Polyurethane, 30% Nylon, 30% Polyoxymethylene

Standard cover: 100% Polyester

Anti-slip-bottom: 70% Polyester, 30% Polyurethane

Optional accessories:

Incontinence cover: 80% Polyester, 20% Thermoplastic Polyurethane

Permissible environmental conditions:

Ambient temperature: -25 up to 110°C

Relative humidity: max. r.H. 50%

Max. pressure in the cushion corresponds to ambient air pressure

Dimensions:

Article number	Width (valve side)	Depth	Height	Net weight
SK-AIR-380x400x80-S	38 cm	40 cm	8 cm	680 g
SK-AIR-400x400x80-S	40 cm	40 cm	8 cm	720 g
SK-AIR-430x400x80-S	43 cm	40 cm	8 cm	760 g
SK-AIR-430x430x80-S	43 cm	43 cm	8 cm	830 g
SK-AIR-450x400x80-S	45 cm	40 cm	8 cm	800 g
SK-AIR-450x450x80-S	45 cm	45 cm	8 cm	860 g
SK-AIR-480x430x80-S	48 cm	43 cm	8 cm	880 g

CE-labelling:

As an accessory for medical products, this product wears the category 1 label according to EG regulation 93/42/EWG. This product is in compliance with the applicable entire German and European Safety Regulations. A conformity declaration is available at our website: www.rolko.com.

Operating Instruction  / Revision status - 2014-06 / © Rolko Kohlgrüber GmbH /
Industriestraße 14 / 33829 Borgholzhausen / Tel.: +49 (0) 5425 94020 / info@rolko.com

Any reprint, in full or in part, requires our prior permission. All rights reserved.

We reserve the right to make changes for technical reasons. All published information are based on nominal data and are subject to design and manufacture related tolerances.

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Brugsanvisning  / Revisionsstatus 2014-06 / © Rolko Kohlgrüber GmbH /