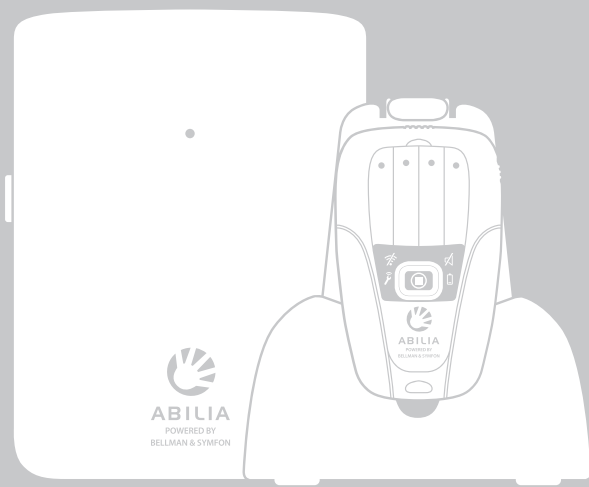
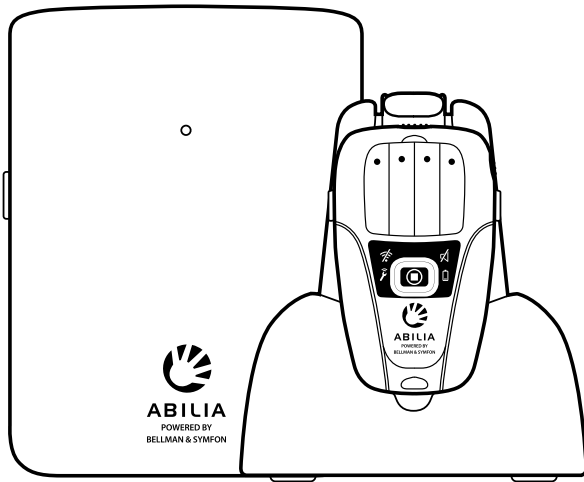
All In One

EN	User manual	4
NO	Brakerhåndbok	30
SE	Bruksanvisning	56

Contents

Chapter 1: Install transmitter

- 6 Transmitter overview
- 7 Install SIM card
- 8 Link transmitter to mobile phone
- 9 Key to alarms on mobile phone
- 10 Pair wireless alarm unit
- 12 Fit transmitter
- 13 Connect sensors to transmitter

Chapter 2: Install Pager

- 14 Pager overview
- 15 Install pager
- 16 Pair transmitter to pager
- 17 Key to pager signals

Chapter 3: Maintenance

- 18 System test and replacing batteries
- 19 Handling and cleaning

Chapter 4: Troubleshooting and other information

- 20 Troubleshooting
- 22 Technical information
- 24 Personalised label in pager
- 25 Important information

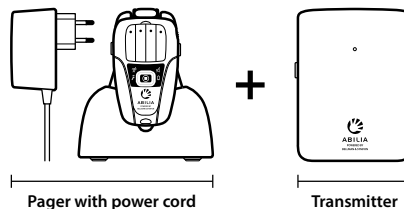
Find your way around the manual

All In One is available with various accessories that require individual installation. To find the right section in the user manual, remove all products from their packaging and check them against the descriptions below.

EN


All In One basic configuration

Go to page 12


All In One with module for wireless alarm

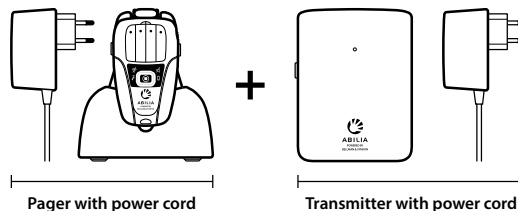Go to page 10


All In One with GSM module

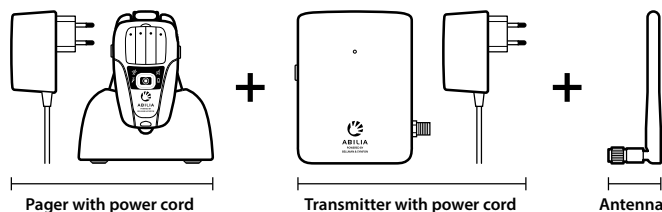Go to page 7

Transmitter overview

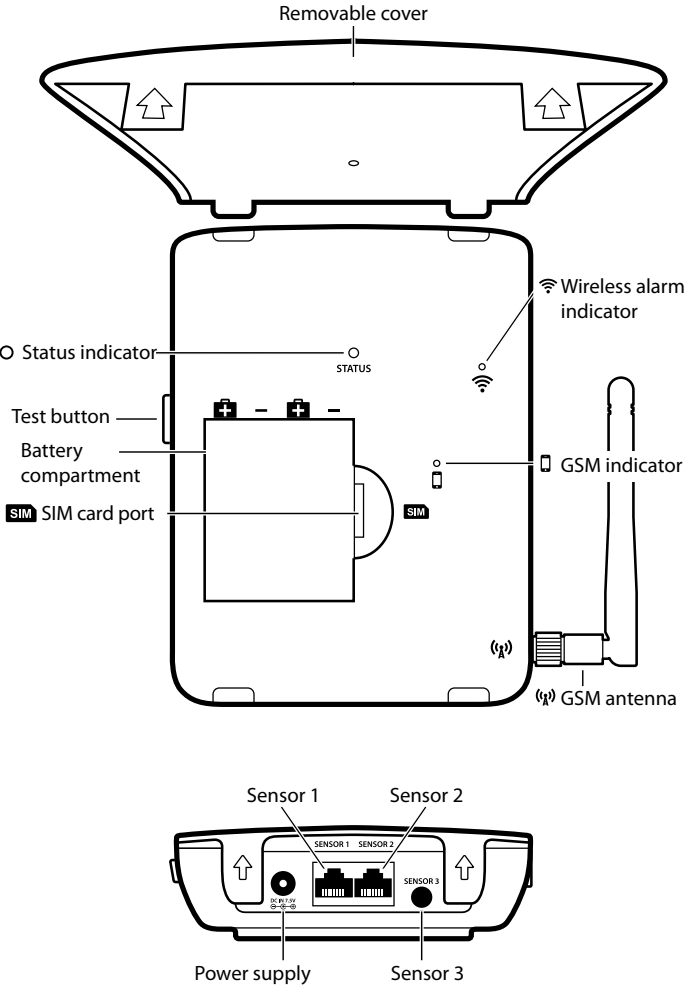Install SIM card

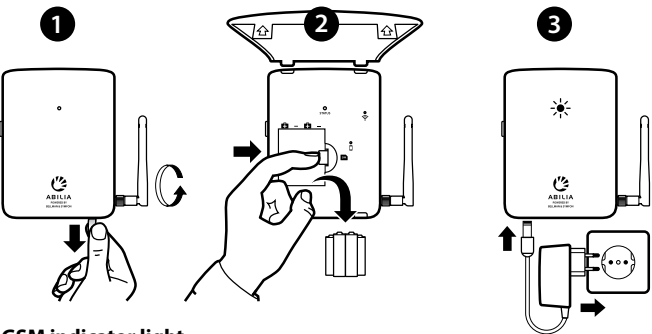
With a GSM module, you can also receive alarms on a mobile phone, very practical when you are out of the building. You will need a **Micro SIM** card for the transmitter (not a prepaid card). The SIM card shall either have the **PIN set to code '0000'** or have no PIN code, see the operator's instructions to change PIN code.

How to install a SIM card

Note: Disconnect the transmitter from its power supply when installing a SIM card.

- 1 Attach the GSM antenna and pull out the battery tab on the transmitter.
- 2 Open the cover and remove the batteries. Insert the SIM card into the holder in the battery compartment marked with a **SIM** symbol. The symbol indicates card direction.
- 3 Reinstall the batteries and close the lid. Connect the power cord to the transmitter and plug into a mains socket.

When the transmitter is connected to the GSM network, the status indicator light will flash green every 15 seconds. If it does not start flashing green within 2 minutes, go to **Troubleshooting** on p. 20.

GSM indicator light

The transmitter has an indicator light, showing current GSM status, located inside the cover and marked with a  symbol (see overview on left). This indicates whether the transmitter is linked to the GSM network and when it is transmitting.

If the indicator is	It means that
Flashing every 3 s	• The transmitter is connected to the GSM network.
Constantly lit	• The transmitter is texting and calling an assistant.
Flashing every 1 s	• The transmitter has lost contact with the GSM network.
Off	• The transmitter has no GSM module/ it's out of order.

Link transmitter to mobile phone

To receive alarms via GSM, the transmitter must be linked to the assistant's mobile phone. A transmitter can handle up to two assistants. They are designated **Assistant 1** and **Assistant 2**.

How to link a transmitter

- 1 Send a text message with Assistant 1's number to the transmitter's mobile number according to the principle below. Repeat for Assistant 2.

Note: Use space before and after the mobile number.

Assistant 1
ALERT 'MOBILE NUMBER' A1


For example:
ALERT 070123456789 A1

Assistant 2
ALERT 'MOBILE NUMBER' A2

For example:
ALERT 070123456789 A2

- 2 To test the link: press the transmitter's test button. The transmitter will send a text message and call the assistants in the order stated on the next page. If no text or call is received, see **Troubleshooting** on page 20.

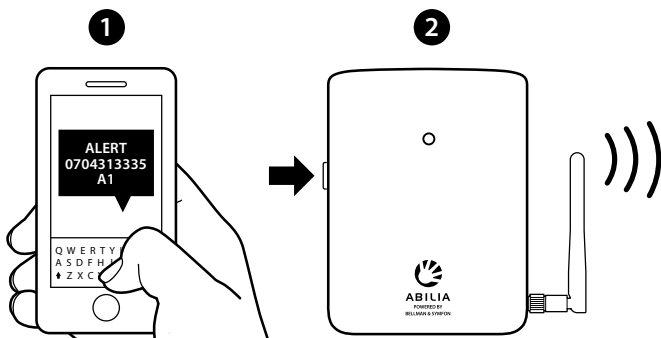
Repeat steps 1-2 to link a *new* transmitter. Send the message with the mobile numbers of the assistants to the new transmitter's mobile number.

Note: You can switch assistant by texting the new number to the transmitter according to the **ALERT 'NEW MOBILE NUMBER' A1** principle. To clear all assistant numbers, text **DELETE** to the transmitter. This action cannot be cancelled.

How it works

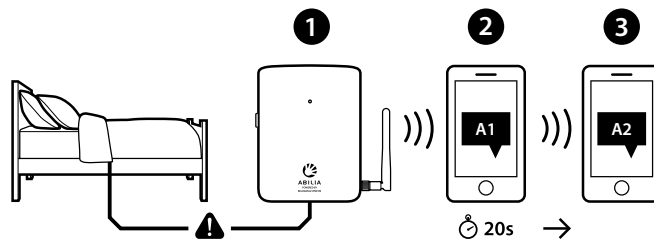- 1 When a sensor is activated, the transmitter sends an alarm via text message to **Assistant 1**. As a safety precaution, it also makes a call (tone only).
- 2 Assistant 1 has **20 seconds** to respond to the call. To prevent confusion with voice mail, the call has to be concluded within **10 seconds**.
- 3 If Assistant 1 fails to answer or hang up in time, the text message and call are redirected to **Assistant 2**.

How to read alarms

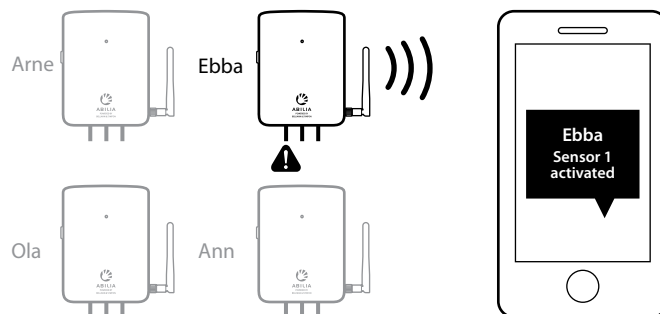
- 1 To quickly be able to see which transmitter is sending an alarm, you need to enter a contact in your mobile contact list for each transmitter on the system.

Note: Name the transmitter according to the resident's name or room number.

- 2 When a sensor is activated on any of the transmitters, you will receive a text message with the following:

- **Transmitter name** (or *mobile number* if it's not in your address book)
- **Activated sensor no.** 1-4

The example below shows that sensor 1 in Ebba's room has activated.


Pair wireless alarm unit

A wireless alarm unit is often carried on the wrist or around the neck, but also comes in the form of a wireless door and movement sensor. The wireless alarm unit needs to be paired to the transmitter before it can work with the system.

Linking a wireless alarm unit

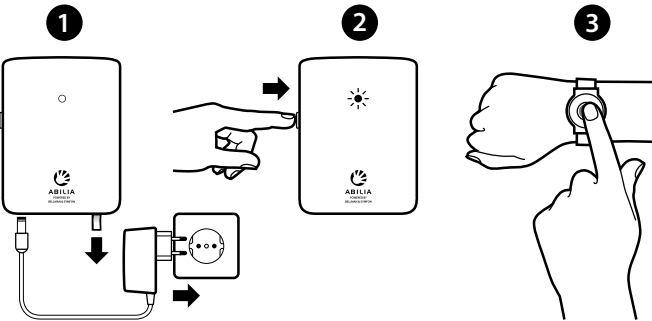- 1 Pull out the battery tab to start the transmitter. Connect power cord and plug into a mains socket.
 - 2 Press and hold the transmitter's test button. Release the button when the transmitter's status indicator light begins to flash blue.
 - 3 Activate the alarm unit within 10 seconds to pair the units. The transmitter status indicator light will flash green three times when ready.
- Repeat steps 2-3 to link a new alarm unit. Max. 4 units can be linked.


Indicator light for wireless alarm unit

There is an indicator under the cover that shows the current alarm status. It is marked with a  symbol (see Overview on p. 6). Here you can see if the alarm unit is linked and when the transmitter receives an alarm.

If the LED is	It means that
Lit / blinks in yellow	• The transmitter is receiving an alarm / heartbeat.
Lit in orange	• The transmitter is ready to be connected to the alarm unit.
Off	• The transmitter has no wireless alarm module. • The module is out of order.

Delete wireless alarm units

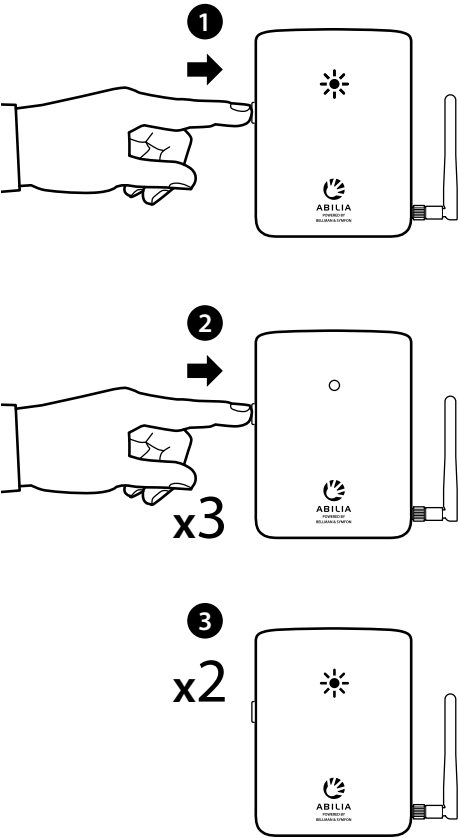- 1 Press and hold the transmitter's test button. Release the button when the transmitter's status indicator light begins to flash blue.
 - 2 Press the transmitter test button three times in quick succession.
 - 3 The transmitter's status indicator will show that all linked alarm units have been deleted by flashing green twice.
- Note:** This action deletes pairings permanently and cannot be cancelled.

Mounting a transmitter

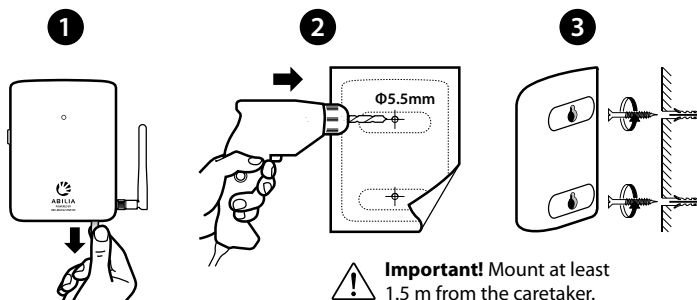
Note: A transmitter with a GSM module or wireless alarm unit needs a certain amount of preparation before mounting.

- Before installing a transmitter with **GSM** module, read page 7-9.
- Before installing a transmitter with **wireless alarm unit** module, read page 10.

How to mount a transmitter

- 1 Pull out the battery tab to start the transmitter.
- 2 Mark and then drill the holes using the drilling template.
- 3 Leave 2 mm distance between screw head and wall when screwing the screws in. Hang the transmitter on the wall and check that it is securely in place.

Repeat steps 1-3 to mount a new transmitter.

Transmitter status indicator light

The indicator light on the front shows the current status. It can for instance tell you when the transmitter sends an alarm or if an error has occurred.

Note: If an error occurs, check the indicator lights located under the cover.

If the LED is	It means that
Blinking in green	• The system is working normally.
Blinking in yellow	• The transmitter is activated by a sensor and sending an alarm.
Blinking in red	• The transmitter's battery level is low, see p. 18.
Blinking in purple	• The batteries are fitted incorrectly/wrong type, see p. 18.
Blinking in white	• The GSM module is searching network or out of order, see p. 20.
Blinking in blue	• The wireless alarm unit is ready to be connected / error, see p. 20.

Connect sensors to transmitter

You can connect up to 3 different wired sensors to a transmitter, e.g. epilepsy alarm, incontinence alarm and door sensor.

How to connect a sensor

- 1 Connect the sensor on the under side of the transmitter according to the figure below. Select input for SENSOR 1 or SENSOR 2 if it has an RJ11 jack and input for SENSOR 3 if it has a 3.5 mm mono telephone jack. Check that the jack is securely in place and that the cable is secure.
- 2 Verify that the connection works by activating the sensor. The status indicator light will flash yellow to show that an alarm has been sent.

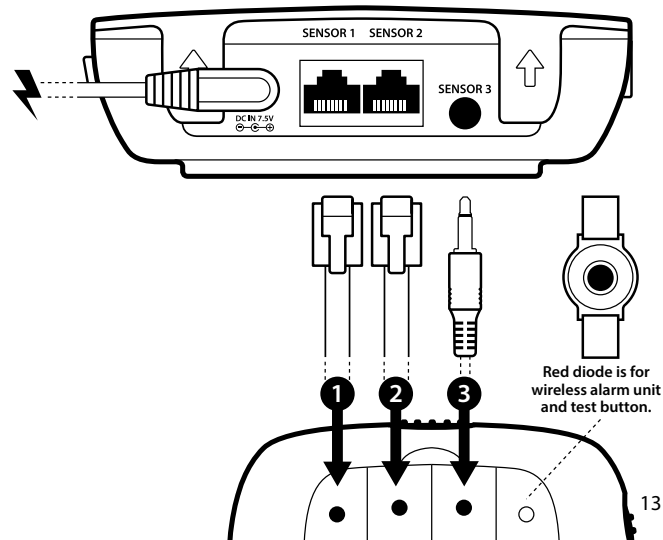
Repeat steps 1-2 to connect a new sensor.

Note: Each sensor is allocated its own LED light on the pager as follows:



- SENSOR 1 - Orange diode
- SENSOR 2 - Green diode
- SENSOR 3 - Yellow diode
- The red diode is reserved for wireless alarm unit and test button.

If you have installed **SEVERAL** transmitters, the pager will signal differently, see p. 17.

Note: Detailed information on how to install and use the various sensors can be found in the respective sensor's user instructions.


Overview of pager and charger

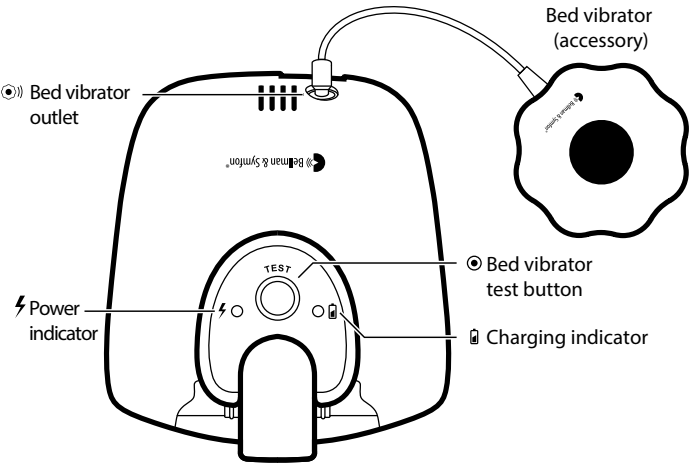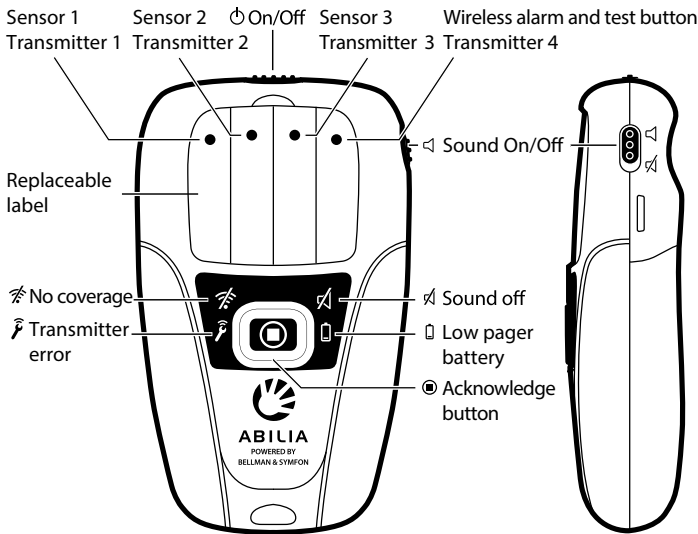Install pager

- 1 Pull out the battery tab on the pager and charger to start the units.
- 2 Plug the charger into a mains socket. The power LED ⚡ will light up to show that it is connected. Place the pager in the charger and check that it clicks into place. The charging LED 🔋 will blink/light up to show that it's charging.

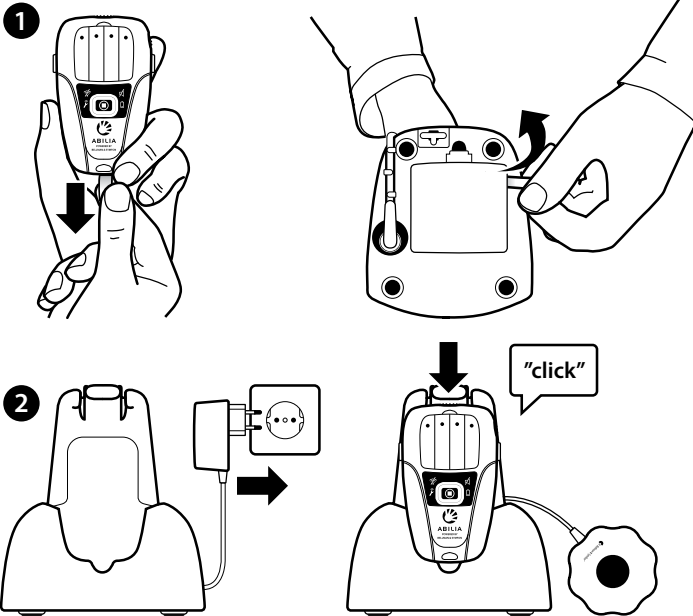Note: Remember to charge the pager for 6 hours before using it for the first time. When fully charged, the charging LED 🔋 will turn off.


How to use the pager

When a sensor is activated, the pager will sound and vibrate. The diode colour and flashing pattern shows which transmitter/sensor has been activated.

If you want to	Do this
Switch off the pager alarm	• Press the pager Acknowledge button .
Mute the alarm	• Push the pager Sound switch up/down.
Switch the pager on/off	• Push the pager On/Off switch right/left.

Pair transmitter to pager

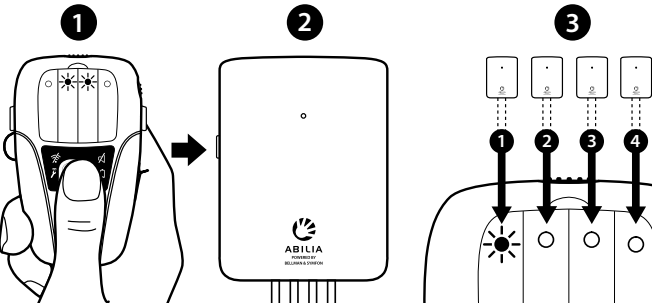
- 1 To link a transmitter, press and hold the pager acknowledge button. Release the button when the two centre diode lights start to flash.
- 2 Press the transmitter test button within 10 seconds to pair the units.
- 3 A diode light on the pager will come on to show that the transmitter is linked:
 - Orange diode - Transmitter 1
 - Yellow diode - Transmitter 3
 - Green diode - Transmitter 2
 - Red diode - Transmitter 4

Repeat steps 1-3 to link a new transmitter. You can link up to 4 units.

Delete linked transmitter

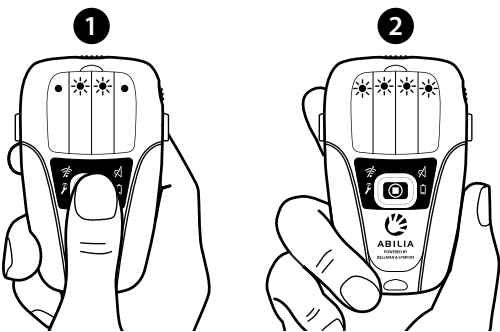- 1 Press and hold the pager OK button until the two centre diodes begin to flash. Press the button three times in quick succession.
- 2 All the diodes will flash twice to show that the linked transmitters have been deleted. This action cannot be cancelled.

Key to pager signals

The pager shows which sensor and transmitter has activated by using various diode colours and flashing patterns.

Note: The pager diode colours and flashing pattern differ depending on whether you have **ONE** or **SEVERAL** transmitters linked to the system.

System with ONE transmitter - Each sensor has a dedicated diode colour

Activated transmitter	Activated sensor	Colour and flashing pattern
Transmitter 1	Sensor 1	Orange
Transmitter 1	Sensor 2	Green
Transmitter 1	Sensor 3	Yellow
Transmitter 1	Wireless alarm unit	Red
Transmitter 1	Test button	Red

System with SEVERAL transmitters - Each transmitter has a dedicated diode colour
- Sensors are identified by flashing pattern

Activated transmitter	Activated sensor	Colour and flashing pattern
Transmitter 1	Sensor 1	Orange
Transmitter 1	Sensor 2	Orange
Transmitter 1	Sensor 3	Orange
Transmitter 1	Wireless alarm unit	Orange
Transmitter 1	Test button	Orange
Transmitter 2	Sensor 1	Green
Transmitter 2	Sensor 2	Green
Transmitter 2	Sensor 3	Green
Transmitter 2	Wireless alarm unit	Green
Transmitter 2	Test button	Green
Transmitter 3	Sensor 1	Yellow
Transmitter 3	Sensor 2	Yellow
Transmitter 3	Sensor 3	Yellow
Transmitter 3	Wireless alarm unit	Yellow
Transmitter 3	Test button	Yellow
Transmitter 4	Sensor 1	Red
Transmitter 4	Sensor 2	Red
Transmitter 4	Sensor 3	Red
Transmitter 4	Wireless alarm unit	Red
Transmitter 4	Test button	Red

System test

It is important that system function is checked at least once a week. Perform a full system test to include activating all sensors and test all transmitter alarm functions.

Important!

- More frequent test intervals can be required if specified by the sensor user manual.
- More frequent test intervals may be required in special circumstances, if the system configuration is modified or if any of the units are dropped.

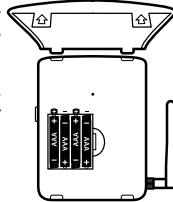
Changing batteries

Check battery performance regularly on all units to ensure that the system is fully functional. Please note that rechargeable NiMH batteries should be replaced approx. every 2 years. Always keep batteries out of reach of children. Dispose of batteries according to your local environmental laws and guidelines.

Replacing batteries in the transmitter

Open the cover of the transmitter. Remove the old batteries and replace with new ones. Instructions are above the battery compartment.

Important! If the transmitter is battery powered, use alkaline AAA batteries only. If the transmitter is mains powered, use rechargeable NiMH AAA batteries only. NEVER mix NiMH batteries with alkaline batteries as the batteries and charger can overheat.

Replacing batteries in the pager

Open the battery cover on the back of the pager. Remove the old battery and replace with a new one, see the instructions in the battery compartment.

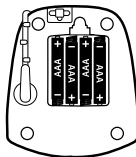
Important! Only rechargeable NiMH AAA batteries shall be used in the pager.

Replacing batteries in the pager charger

Open the battery cover under the pager charger. Remove the old batteries and replace with new ones. Instructions are in the battery compartment.

Important! Only rechargeable NiMH AAA batteries shall be used in the pager charger.

Handling and cleaning

Warning! Failure to follow the instructions for handling and cleaning can damage the products and void the warranty.

Using jacks and outlets

Always check that the jack fits the connector and that it is correctly aligned. Never force a jack into a connector. If it is difficult to plug in the jack, check that the connector is not blocked in any way and make sure that you are plugging it into the right connector.

Using the products

Do not use or store the products near a source of heat. Only use in dry environments when the temperature is between 15°-35° C. Do not use the pager in the bath or shower. If the pager or transmitter gets wet or is exposed to moisture, it should no longer be regarded as reliable and should be replaced. If you do not plan to use the product for an extended period, remove the batteries.

Cleaning

Before cleaning the product, disconnect it from any power source. Use a soft, lint-free cloth. Avoid any moisture getting into the openings. Never use window cleaner, detergents, sprays, solvents, alcohol, ammonia or products that contain abrasives.

Guarantee and service

If the product is either damaged or out of order, follow the instructions in the user manual. If the product continues to function incorrectly, contact your local dealer for details on guarantee cover and service.

Other important information

Information on how to dispose of the product and other important information on instructions and safety can be found in the section on **Important information** on p. 28-29.

Troubleshooting

Most problems with the system can be quickly solved by following the instructions below. There are more tips and information on troubleshooting on our website.

Transmitter

If	Try this
Transmitter status indicator light flashing red	The transmitter battery level is low. Replace the batteries if the transmitter is battery powered. Charge the batteries if it is mains powered. Important!  If the transmitter is battery powered, use alkaline AAA batteries only. If the transmitter is mains powered, use rechargeable NiMH AAA batteries only. NEVER mix NiMH batteries with alkaline as the unit can overheat.
Transmitter status indicator light flashing purple	Batteries in transmitter are wrong type or fitted incorrectly. See above for correct configuration and fitting of batteries.
Transmitter status indicator light flashing white	- The GSM module is indicating an error. Open the cover and check the GSM LED above the -symbol (see Transmitter overview on p 6). - If the LED is blinking every second, no GSM network has been detected. Check that the SIM card is fitted correctly and that you have GSM coverage in your area, see Installing a SIM card on p 7. When the transmitter is connected to a GSM network the GSM LED will blink every 3 s. - If the problem persists, contact your GSM provider. - If the LED is off, the unit has no GSM module or the module is out of order. Contact your dealer for service.
Transmitter status indicator light flashing blue	- The transmitter is ready to be connected to a wireless alarm unit. - The alarm unit is faulty, see separate user manual. If the problem persists, contact your dealer for service.

Pager

If	Try this
Pager  symbol is lit whilst the pager is vibrating and beeping	Pager has lost radio contact with a transmitter. The LED position on the pager will indicate which transmitter is at fault if you have several transmitters. Move to within range of the transmitter.
Pager  symbol is lit whilst the pager is vibrating and beeping	A transmitter in the system is indicating an error. The LED position on the pager will indicate which transmitter is at fault if you have several transmitters. See Troubleshooting transmitter on p. 20.
Pager  symbol is white, whilst the pager is vibrating and beeping	Pager has less than two hours of battery life left. Charge batteries by putting the pager in its charger. A full charge takes 6 hours.
Pager  symbol is red, whilst the pager is vibrating and beeping	Battery in pager is wrong type, incorrectly fitted or flat. Check how the battery is fitted or replace, see Changing batteries on p. 18.
Pager not emitting any sound whilst the  symbol is lit	Pager sound switched off. Slide sound switch upwards to restore sound. Switch is on right side of pager (see overview).

Pager charger

If	Try this
Pager cannot be charged	- Check that the pager is correctly placed in the charger. The LED by the -symbol will blink/light up to show that it's charging. - Check that the charger is connected to mains power. The LED by the -symbol is lit to show that it is connected. If the LED blinks, it is using the backup battery.
Reserve battery in charger not working and charger  symbol off	Check batteries are correctly fitted and charged. Charge or replace batteries, see Changing batteries on p. 18.

Technical information

All In One Transmitter

Dimensions and weight	h 140 mm x w 105 mm x d 30 mm, 210 g incl. batteries
Mains electricity	Input: 100 - 240 V ~ 50/60 Hz 0.3 A Output: 7.5 V --- 1000 mA Use the supplied adaptor model SSA-10W-12 EU 075100 by SIL
Batteries	With mains electricity: 4 x 1.2 V AAA rechargeable NiMH batteries. Use the supplied rechargeable batteries model HFR-AAA900 900 mAh by Shenzhen Highpower Technology Co. Ltd., or model 5703 1000 mAh by Varta. Without mains electricity: 4 x 1.5V AAA alkaline batteries.
Operating time	365 days without GSM module or wireless alarm unit 4 h reserve with GSM module or wireless alarm unit
Electricity consumption	0.3 mA in basic configuration
Connections	1 x 7.5V power cord 2 x RJ-11 inputs 1 x 3.5 mm tele jack 1 x SMA jack for GSM antenna
Activation	Via test button Via sensors connected to the 3 sensor inputs Via wireless alarm unit (module BE9230 required)
Radio frequency	869.2375 MHz
Range	Up to 500 m line of sight. The range is reduced by walls and large objects that obstruct radio signals. The range can also be affected by other radio transmitters such as TVs, PCs, mobile phones or tablets.
Environment	For indoor use only Temperature range: 15° - 35° C Relative air humidity: 5 to 95%, non-condensating Shall be installed at least 1.5m from the caretaker
Regulatory	Fulfil WEEE, CE, RoHS and R&TTE
Accessories	BE9220 Module for GSM BE9230 Module for wireless alarm units

All In One Pager

Dimensions & weight	h 91 mm x b 55 mm x d 23 mm, 60 g incl. battery
Battery power	1 x 1.2 V AAA rechargeable NiMH battery. Use the supplied rechargeable battery model HFR-AAA900 900 mAh by Shenzhen Highpower Technology Co. Ltd.
Operating time	24 h
Charging time	Up to 6 h
Electricity consumption	20 mA
Activation	Via radio
Radio frequency	869.2375 MHz
Range	Up to 500 m line of sight. The range is reduced by walls and large objects that obstruct radio signals. The range can also be affected by other radio transmitters such as TVs, PCs, mobile phones or tablets.
Environment	For indoor use only temperature range: 15° - 35° C Relative air humidity: 5 to 95%, non-condensating
Regulatory	Fulfil WEEE, CE, RoHS and R&TTE

All In One pager charger

Dimensions & weight	h 110 mm x w 93 mm x d 110 mm, 195 g incl. batteries
Mains power	Input: 100-240V~50/60Hz 0.5A, Output: 7.5V --- 1600mA. Use the supplied adaptor model SFP0751600P made by Helms-Man.
Battery power	4 x 1.2 V AAA rechargeable NiMH batteries. Use the supplied rechargeable batteries model 80AAAHC 800 mAh by GP.
Power consumption	200 mA
Connections	7.5V power cord 3.5 mm tele jack for bed shaker Via the test button that activates a bed shaker
Activation	Via the test button that activates a bed shaker
Environment	For indoor use only. Temperature range: 15° - 35° C Relative humidity: 5% - 95%, non-condensing
Regulatory	Fulfil WEEE, CE, RoHS and R&TTE
Accessories	BE1270 Bed shaker

 **CAUTION!** Explosion hazard if battery is replaced by an incorrect type. Dispose of used batteries according to the recycling instructions.

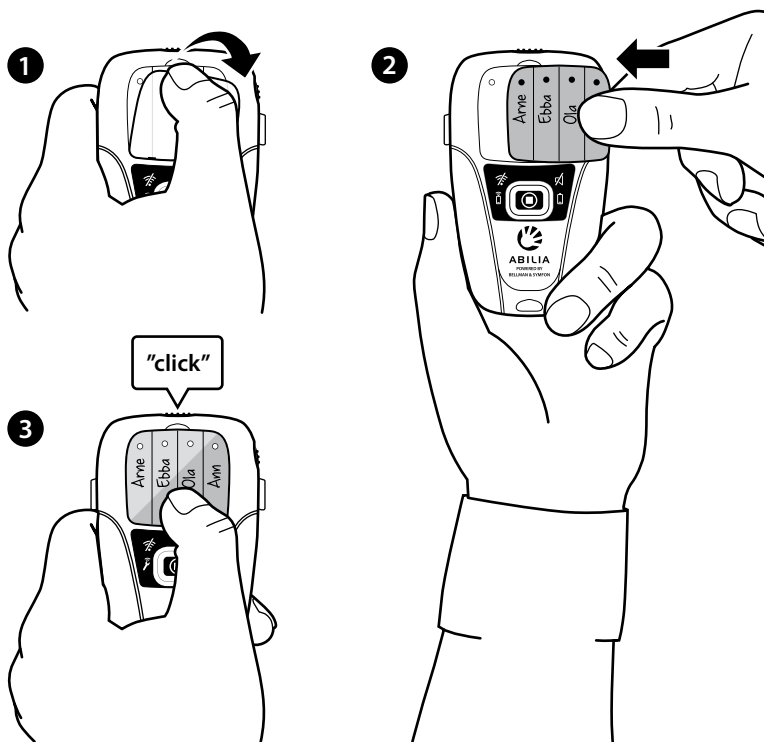
Personalised label in pager

A personalised label makes it easier to identify alarms on your pager. Depending on configuration, you can write the name of residents or the sensors linked. An extra label is included in the package.

Replacing labels in the pager

- 1 Remove the transparent plastic cover protecting the panel with coloured diodes. Avoid damaging the plastic.
- 2 Replace the label.
- 3 Snap the plastic cover back into place.

Note: You can download and editable PDF version on our website.

Important information

This section contains important information about safety, handling, disposal, recycling and warranty.

Read the manual and all safety information before using All In One. Keep the manual for future use.

Warning!

- There is a potential risk of fire, electrical shock, or damage to the product if the safety instructions are not followed.
- Do not use or store this product near any heat sources such as radiators, naked flames, ovens or any other heat output.
- Protect the cables from any potential source of damage.
- Use only the power supply and accessories specified in this manual.
- Disconnect the power supply from the mains during thunderstorms or if the product is not in use for a long period.
- If the product is dropped always test it before reusing.
- Please note that the deliberate or accidental switching off of the volume or power switch on the Pager can result in a missed alarm.
- Be aware that alarms can be missed if batteries run out, connectors become disconnected, or mobile systems are out of operation. Only refer damaged or faulty products to qualified service personnel.
- Do not dismantle the product; there is a risk of electric shock. Tampering with or dismantling the product will void warranty.
- The product is intended to be used as a care alerting system but should not be used as the only security in life-threatening situations. This product does not replace the need for personal supervision of patients/users.
- The product has no power switch. In order to disconnect the product, the charger and power supply must be disconnected from the wall outlet. Make sure that these are easily accessible.
- The product is designed for indoors use only.
- Do not expose the product to moisture.
- Protect the product from shocks during storage and transport.

Model, type and classification

Information is available at the back of the All In One Pager and on the charger for the Pager.

Recycling of old products

The symbol means that according to local laws and regulations it should not be disposed of with household waste. When the product is no longer of use, hand it in to a recycling centre for electrical waste and electronic equipment. By recycling used products you will help to conserve natural resources and protect health and the environment.

Recycling of used batteries

This symbol means that the product contains batteries that according to local laws and regulations must not be disposed of with household waste. Take used batteries to a battery disposal facility or a recycling centre.

The European Union - declaration of conformity


CE 0359 Hereby, Bellman & Symfon Europe AB, declares that this product is in compliance with the essential requirements and other relevant provisions of R&TTE 1999/5/EC, LVD 2006/95/EC, MDD 93/42/EEC and EMC 2004/108/EC directives. The declaration of conformity may be consulted at Bellman & Symfon Europe AB, Södra Långebergsgatan 30, 421 32 Västra Frölunda, Sweden.

My notes

Use the illustration below to note details of your installation of the All In One system.


Note. Examples of notes include a reminder of which sensors you have linked to the transmitter, which transmitter is linked to the pager and contact details for assistants.

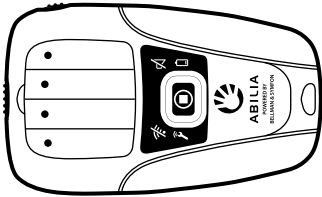If you need more pages, go to our website for a PDF version under **My notes**.


Pagers

Orange diode:	
Green diode:	
Yellow diode:	
Red diode:	

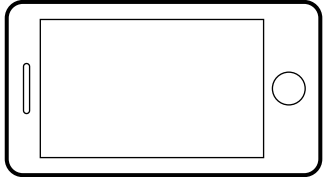
Assistant 1	
Name:	
Mobile no:	

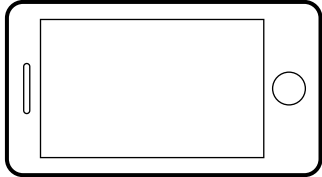
Assistant 2	
Name:	
Mobile no:	


Transmitter 1	
Mobile no:	
Sensor 1:	
Sensor 2:	
Sensor 3:	
Wireless alarm:	

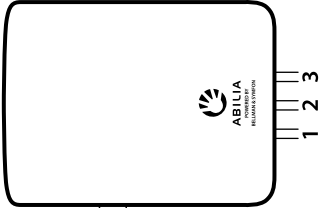Transmitter 2	
Mobile no:	
Sensor 1:	
Sensor 2:	
Sensor 3:	
Wireless alarm:	

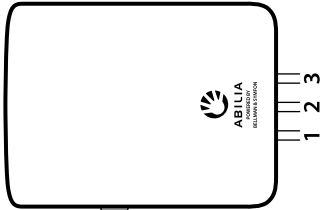Transmitter 3	
Mobile no:	
Sensor 1:	
Sensor 2:	
Sensor 3:	
Wireless alarm:	

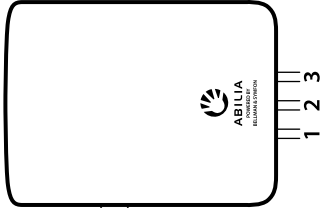Transmitter 4	
Mobile no:	
Sensor 1:	
Sensor 2:	
Sensor 3:	
Wireless alarm:	

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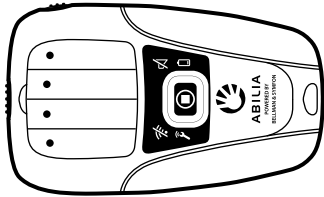
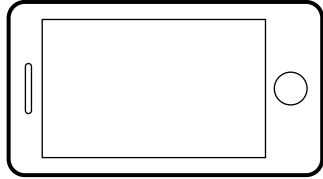
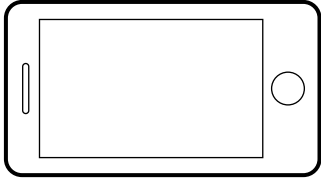
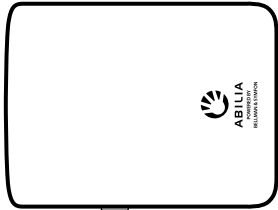
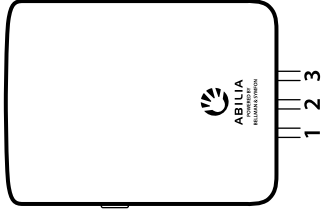
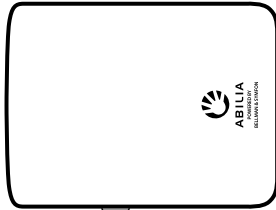
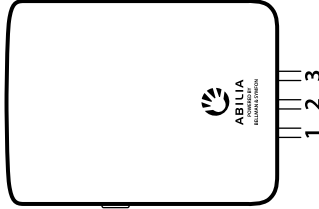
Technical information

BE2310 Transmitter

Mains electricity Input: 100 – 240 V ~ 50/60 Hz 0.3A
Output: 7.5 V $\equiv$ 1000 mA
Use the supplied adaptor model SSA-10W-12 EU 075100 by SIL

Battery power With mains electricity:
4 x 1.2 VAAA rechargeable NiMH batteries.
Use the supplied rechargeable batteries model HFR-AAA900 900 mAh by Shenzhen Highpower Technology Co., Ltd. or model 5703 1000 mAh by Varta.
Without mains electricity: 4 x 1.5V AAA alkaline batteries

Receiver category Wireless alarm receiver module (BE9230 accessory)
is a category 2 receiver

BE2340 Pager

Battery power 1 x 1.2 V AAA rechargeable NiMH battery.
Use the supplied rechargeable battery model
HFR-AAA900 900 mAh by Shenzhen Highpower
Technology Co., Ltd.

Receiver category Pager is a category 2 receiver

BE1250 Pager charger

Mains electricity Input: 100 – 240 V ~ 50/60 Hz 0.5A
Output: 7.5 V $\equiv$ 1600 mA
Use the supplied adaptor model SFP0751600P
made by Helms-Man

Battery power 4 x 1.2 V AAA rechargeable NiMH battery
Use the supplied rechargeable batteries model
80AAHC 800 mAh by GP

CAUTION: Explosion hazard if battery is replaced by an incorrect type.
Dispose of used batteries according to the instructions in the leaflet.

**POWERED BY
BELLMAN & SYMFON**