

DESIGNED AND DEVELOPED IN NORWAY

# Letsip® V-Series Pack:

## Letsip® V-Series Pack:

Letsip® V0 mouthpiece

Letsip® V1 mouthpiece

Letsip® V2 mouthpiece

Letsip® V3 mouthpiece

Letsip® V4 mouthpiece

Letsip® V5 mouthpiece

Letsip® rechargeable handle



Letsip® V-Series reaches deep into the musculature with its sonic vibration.



The Letsip V-series is designed to give muscles more stimulus, so that they develop better, giving a broader range of higher or lower muscle activity.



[www.letsip.com](http://www.letsip.com)

[www.letsip.com/1](http://www.letsip.com/1)



Scan me

Exercising the muscles in and around the mouth

# www.letsip.no

## Letsip® V-Series Pack:

### Letsip® V-Series:

The V-series is a versatile product for exercising the lips, cheeks and tongue. It reaches deep into the musculature with its sonic vibrations.

Before use; charge the battery for 2 hours.



**Start the handle** by pressing the on/off button.

Turn off by pressing the button for 2 seconds

Press on/off repeatedly to change the mode settings

Mode: Strong – Medium – Sensitive - Pulse 1 - Pulse 2

Motor: 48000 Pulses /minute, in strong mode

Battery: Rechargeable Li-ion battery with 3,7V 600mAh 2.22Wh  
USB cable, 5V / 1A

### Cleaning and disinfecting:

We recommend that you lightly wash all products before use, excluding the Letsip® handle piece. All the pieces, excluding the handle, are dishwasher safe, up to 158°F / 70°C. The Letsip® handle piece is water resistant and can be disinfected with wipes or a light spray. Disinfecting your Letsip products is best done with LifeClean. We recommend using LifeClean, as it does not cause any visible damage to Letsip products. LifeClean administers the highest level of disinfection, coming into full effect within two minutes of proper application.

### NOTE!

*Cleaning Letsip products with spirits or washing the plastics in high temperatures, can change the shape and strength of the product over time. Disinfecting with alcohol may also over time, cause the discolouration of the handle. Follow the cleaning guide at [www.letsip.com](http://www.letsip.com)*

## WARNINGS & PRECAUTIONS

Keep out of the reach of unsupervised children. Caution is recommended when using Letsip products around the mucous membranes in the mouth, and on exposed and/ or irritated skin. Vibration on the throat is not recommended. Always check the product before use and discard the product at the first signs of damage or weakness.



**DESIGNED AND DEVELOPED IN NORWAY**

# 2 year guarantee



## Letsip AS

Langlandsvegen 51, N-6010 Aalesund Norway  
Phone +47 480 09 500 [post@letsip.com](mailto:post@letsip.com) [www.letsip.com](http://www.letsip.com)

**PATENTED** All products are protected by the international patent application



Material ABS Dow USA. Made in P.R.C.