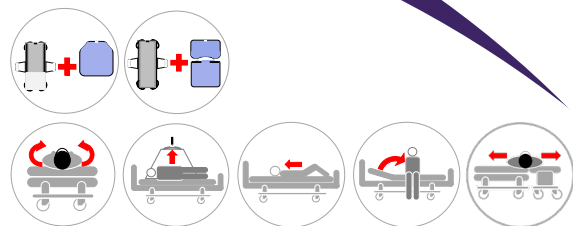
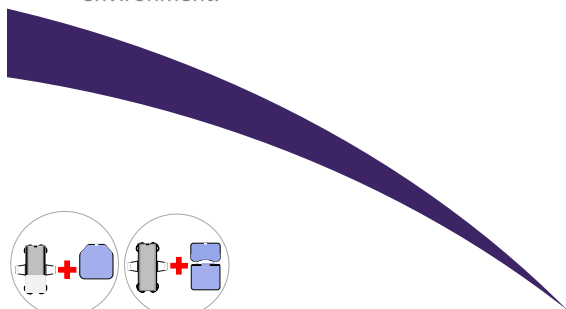
MADE IN EUROPE

This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(DK) Forordning MDR 2017/745 for medicinsk udstyr klasse 1
(UK) Regulation MDR 2017/745 for medical devices class 1

ISO 12 31 03 21

The Original Nordic
Human Handling Concept

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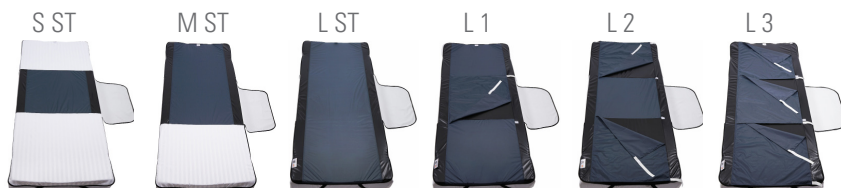
Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.

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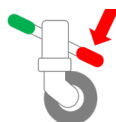
(UK) Never leave the user on the product when lying on the edge of the bed.

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(UK) Never leave the product on the floor.

(DK) Sørg for, at alle relevante bremsere er låst, inden forflytning foretages.

(UK) Make sure that all relevant brakes are activated before transferring the user.

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(UK) Always make sure that the user is not caught in the straps.

(DK) For mere information, brugervejledning, videoer mm., se
(UK) For more information, User Manual, videos etc. visit

www.master-care.dk

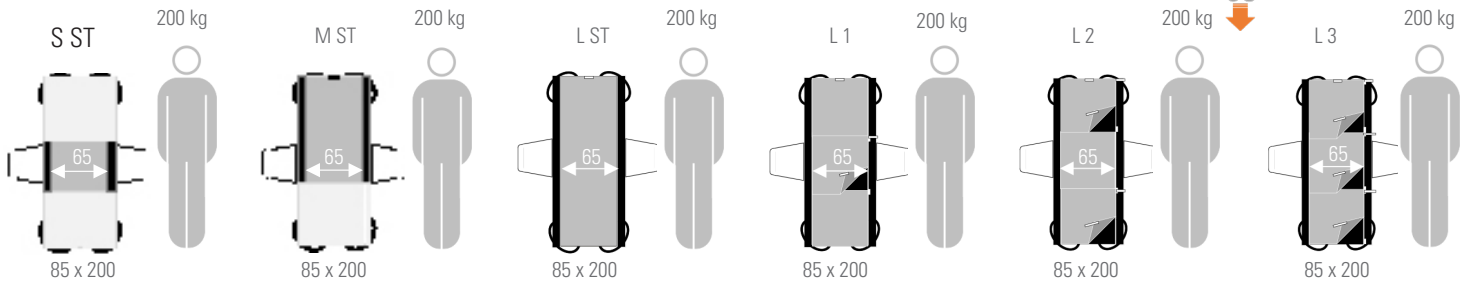
Developed &
Manufactured by:

master-care.dk


Vario Cover

The **Original** Master Turner System


(DK) Monter Vario Cover / (UK) Mount the Vario Cover

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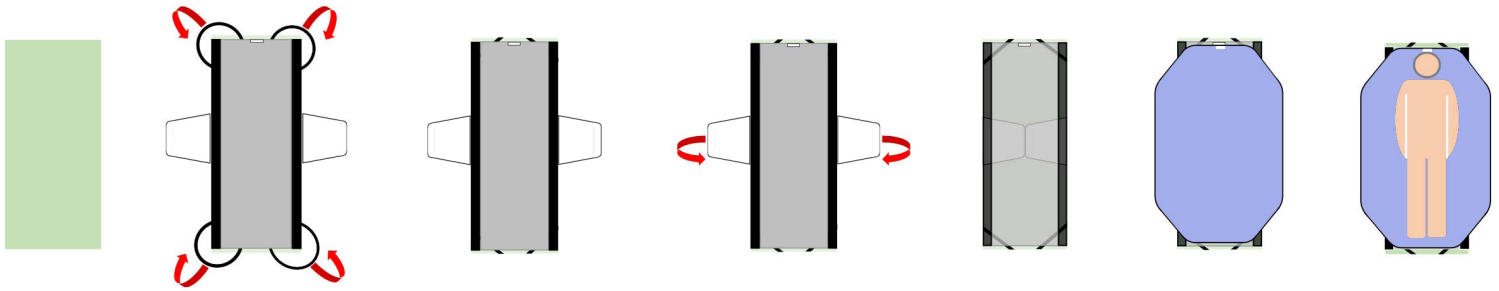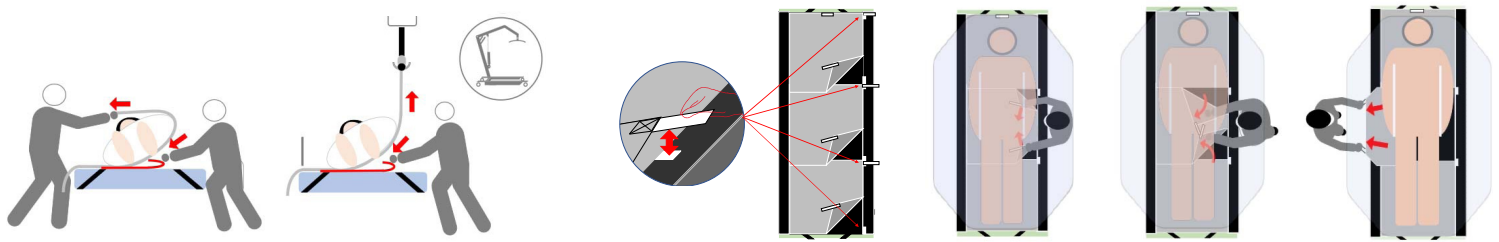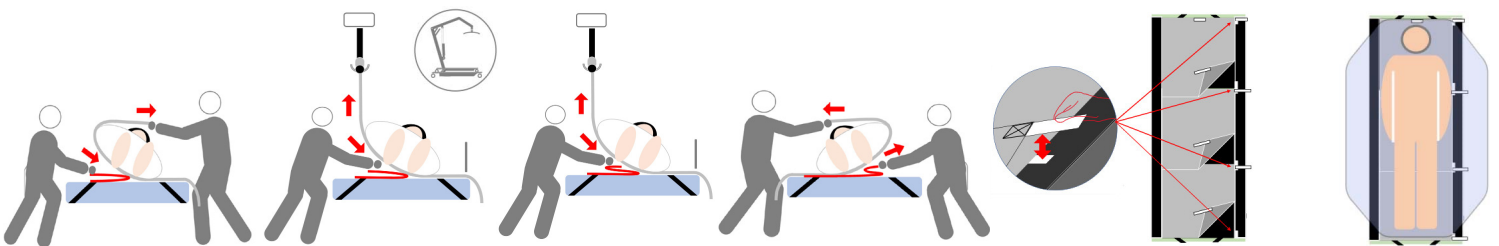
(UK) Products with hook and loop tape should be washed at max. 60 °C.

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(UK) Gentle washing and dry tumbling significantly prolongs the durability.

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(UK) Never use fabric softener as this will reduce the sliding properties significantly.

(DK) Luk altid burrebånd inden vask.

(UK) Always combine the hook and loop tape before washing.

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(UK) Reservations are made for printing errors.

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master care A/S, Sofienlystvej 3, DK 8340 Malling, Danmark