

Quickguide Basic badetaburet - rund med drejepude

Assembly and general recommendations for use

Step 1. Place seat upside down on a flat surface.

Step 2. Fix crossed base tubes with seat using four wing screws with plastic and metal washers. Screw paler wing screw with washers in the middle.

Step 3. Join base tubes with legs.

HEIGHT ADJUSTMENT

- To adjust the height of the shower chair, press metal pin, select appropriate height of leg, and lock the position of the tubes relative to each other with the use of pin. The legs are adjustable in 2.5 cm increments.

Note! The pins must be set on the same level in each leg.