


3. Assembly and disassembly



To mount the wheels on your chair:

1. Press the push button on the quick release axle;
2. Check that you have the same color on the wheel disc and that of the adapter centering pin. (NOTE: There are dedicated Left and Right wheels) (**image 1**)
3. Insert the axle of the drive wheel into the positioning socket (axle receiver) of the wheelchair;
4. Turn the wheel until the adapter centering pin enters one of the holes in the daisy chain; (**image 2**)
5. Check that the push button has decompressed. Pull on the wheel to check that it is securely locked in the frame.

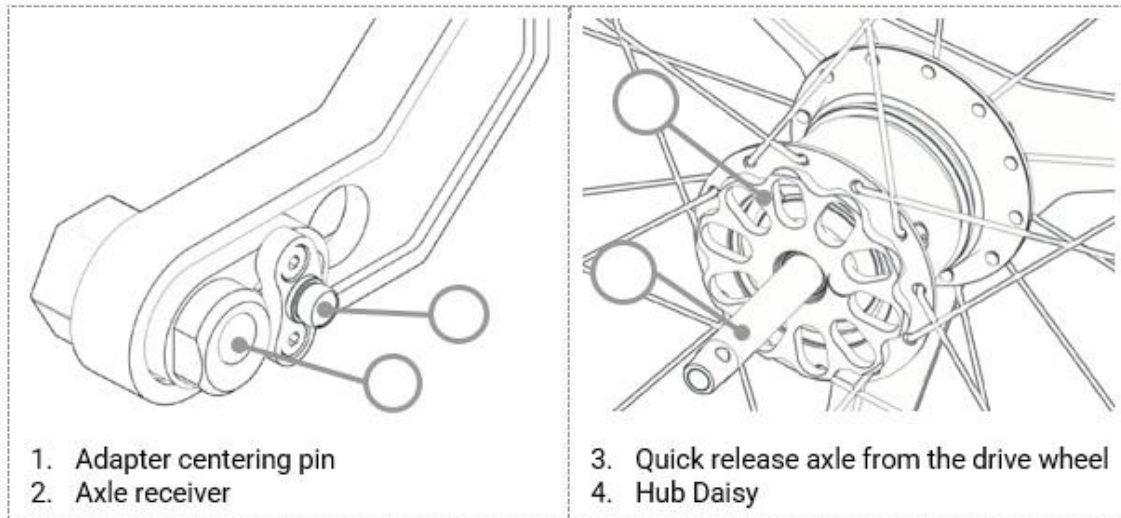